

Swiss Quality for Dental- & Medical Technology

Universally to Success!

TCM Endo III



Universal device for root canal preparation !

- ✓ best price/performance ratio
- ✓ usable for all established NiTi instruments
- ✓ highest quality for a gentle and trouble-free using
- ✓ practically no waste heat at the handpiece
- ✓ handy design ensures ergonomic working
- ✓ torque delimitation for optimum protection against instrument-breaks
- ✓ easy handling and maintenance

NOUVAG AG
St. Gallerstr. 23-25
CH-9403 Goldach
Switzerland

Phone +41 (0)71 846 66 00
Fax +41 (0)71 846 66 70
info@novag.com
www.nouvag.com



NOUVAG
Dental and Medical Equipment



NOUVAG GmbH
Schulthaisstrasse 15
D-78462 Konstanz
Germany

Phone +49 (0)7531 1290-0
Fax +49 (0)7531 1290-12
info-de@novag.com
www.nouvag.com

Please visit our new website:

www.nouvag.com

TCM Endo III

Universal device for root canal preparation !

The TCM Endo III improves a fast and qualitative root-canal preparation. The torque level can be selected with pre-defined buttons.

The turning speed is pre-selectable depending on own needs and will be constantly held to the maximum with each load till the adjusted torque is reached. A constant rotation of the root channel instruments will be reached and the quality of the preparation will be substantially increased.

The new developed torque-safety reduces instrument-breaks.

When the instrument reaches the pre-selected torque level, the motor will reverse for two revolutions, then return to the forward direction to finish the root canal preparation.



Art. No. 1542

Included in Delivery:

- Control unit (Art. No. 1520)
- Electronic motor Endo (Art. No. 1823)
- Contra-angle 8:1 (Art. No. 5057)
- On/Off Foot pedal (Art. No. 1488)

Technical Data:

Voltage:	115V/60 Hz or 230V/50 Hz
Max. power:	40 VA
Speed range micro motor:	1'200 – 16'000 min ¹
Speed range contra-angle:	75 – 2'000 min ¹
Torque range:	2 bis 20 Nmm
Type of instrument interface:	Schaft Ø 2.35 mm (ISO 1797, Typ 1)
Dimension (W x D x H):	100 x 170 x 90 mm
Control unit weight:	2 kg

NOUVAG AG
St. Gallerstr. 23-25
CH-9403 Goldach
Switzerland

Phone +41 (0)71 846 66 00
Fax +41 (0)71 846 66 70
info@novag.com
www.nouvag.com



NOUVAG GmbH
Schultheissstrasse 15
D-78462 Konstanz
Germany

Phone +49 (0)7531 1290-0
Fax +49 (0)7531 1290-12
info-de@novag.com
www.nouvag.com

Please visit our new website:

www.nouvag.com