

MU_011_GB	19/04/13
Rev.2	Page 1 of 24

Operator's manual

lamp for operating theatre

ORION 60

Mobile (LC001LRB)
Single Ceiling (LC002LRB)
Double Ceiling (ORION 60+ORION 40) (LC003LRB)
Double Ceiling (ORION 60+ORION 60) (LC004LRB)



MU_011_GB	19/04/13
Rev.2	Page 2 of 24

Introduction

Dear user,

You are kindly invited to read this manual carefully before proceeding to use the Product in order to safeguard yourself and other people from any injuries.



This appliance is a Class 1 medical device pursuant to European Directives on medical devices (MDD) 93/42/EEC, Annex IX, and 2007/47/EC.

Conformity

The manufacturer declares that this product is in compliance with Annex I (essential requirements of Directive 93/42/EEC and certifies such conformity by affixing the CE mark.

The Product is classified in risk group 1 according to IEC 62471 standard (Photobiological Safety of Lamps).

Validity of manual

This operator's manual refers to the following Products:

- ORION 60 single-ceiling version
- ORION 60 double-ceiling version (ORION 60+60 / ORION 60+40)
- ORION 60 mobile version

Customer service

The customer service is at your disposal in case of Product details, information concerning its use, identification of spare parts being required and for any other queries you might have concerning the appliance, for ordering spares and for matters relating to assistance and warranty.

- TECNO-GAZ
- Strada Cavalli, 4
- I-43038 Sala Baganza Parma -
- Tél.: +39 0521 83.39.26
- Fax: +39 0521 83.33.91
- e_mail: info@tecnogaz.com

Copyright

The contents of this Manual may be amended by TECNO-GAZ, without prior notice or any further obligations, in order to make changes and improvements. The reproduction, including partial, or translation of any part of this manual is forbidden without the written permission of TECNO-GAZ.



MU_011_GB	19/04/13
Rev.2	Page 3 of 24

Right to make changes TECNO-GAZ reserves the right to change, cancel or otherwise amend the data contained in this document at any time and for any reason without prior notice inasmuch as TECNO-GAZ is constantly seeking new solutions which lead to product evolution. TECNO-GAZ therefore reserves the right to make changes to the supplied Product in terms of shape, fittings, technology and performances.

Translations

With regard to translations into languages other than Italian, reference shall always be made to the Italian edition of this operator's manual.



MU_011_GB	19/04/13
Rev.2	Page 4 of 24

Manufacturer's declaration of conformity C€

The company:

TECNO-GAZ S.P.A. Strada Cavalli n. 4 - CAP 43038 - Sala Baganza - Parma - ITALY

Declares under its own responsibility that the Product (Medical lighting device for surgical and diagnosis use):

OR	ION	60
OIV		υu

APPLICARE ETICHETTA

made by TECNO-GAZ S.P.A., complies with Annex VII of Directive 93/42/EEC dated 14/05/1993, and subsequent amendments (including Directive 2007/47/EC dated 05/09/2007) and the following standards:

IEC 60601-1 (Part 1: General requirements for safety)
 IEC 60601-2-41 (Part 2: Particular requirements for safety)
 IEC 60601-1-2 (Part 1: General requirements for safety - EMC)

Classification with reference to article 9 and Annex IX of Directives 93/42/EEC and 2007/47/EC

DURATION: Short term (Par.1 "Definitions", art.1, sub-section 1.1, annex IX)

DESCRIPTION: Non-invasive medical device (Par.1 "Definitions", art.1, sub-section 1.2, annex IX)

Active medical device (Par.1 "Definitions", art.1, sub-section 1.4, annex IX)

CLASS: I (Par.3 "Classification", art.1, sub-section 1.1 Rule 1, annex IX)

Name: Paolo Bertozzi Position: Managing Director

Sala Baganza, 19-04-2012





MU_011_GB 19/04/13

Rev.2 Page 5 of 24

Contents

1	Important information for the user	7
1.1	User qualification	7
1.2	Precautions for safe appliance operation	8
2	Precautions for the appliance operator	8
2.1	Technical safety specifications	8
2.2	Personnel training obligation	8
2.3	Warranty and liabilities	9
2.4	Structural changes or variations	9
2.5	Disposal after use	9
3	Importance of personal safety	9
3.1	Intended use	9
3.2	Environmental conditions	10
3.3	Use in combination with other medical products	10
3.4	Technical safety conditions	11
3.5	Other safety conditions (secondary effects)	11
3.6	Graphic symbols used in this manual	12
3.7	Other graphic symbols used on the Product	12
4	Product description and operation	13
4.1	Product description	13
4.2	Description of operation	14
5	Cleaning and disinfecting	15
5.1	Cleaning the Product	15
5.2	Disinfecting	15
5.3	Sterilizing handpieces	16



MU_011_GB	19/04/13
Rev.2	Page 6 of 24

5.4	Yearly inspections by operator	17
5.5	Repairs	17
6	Adjustments	17
6.1	Setting the rocker arm	17
6.2	Adjusting the braking force	18
6.3	Troubleshooting	18
6.4	Routine maintenance	19
6.5	Spare parts list	19
7	Technical properties	20
8	EMC compliance	22
Not	tes	24



MU_011_GB	19/04/13
Rev.2	Page 7 of 24

1 Important information for the user

Product

The ME (Medical-Electrical) EQUIPMENT to which this manual refers is a LAMP for Operating Theatre or SYSTEM of LAMPS for Operating Theatre. For ease of description, this ME EQUIPMENT will be referred to in this manual with the name of "Product".

1.1 User qualification

Personnel

The Product and these operating instructions are intended for use by medical personnel and qualified technicians working in hospitals and medical surgeries who have acquired working skills by undergoing medical training and who are in possession of necessary authorisation where required.

Personal safety

Importance of personal safety. Before using the Product, read the safety precautions in paragraphs 2.1, 3.4 and 3.5

Adjustments

The operations described in Chapter 6 "Adjustments" must be performed by a qualified technician of the Product operator in accordance with the safety rules and precautions indicated in this operator's manual.

Cleaning

Product cleaning can only be done by duly trained personnel.

Importance of manual

This manual is an integral part of the Product according to the provisions of the European Directives 93/42/EEC and 2007/47/EC.

Always keep this operator's manual close to the Product so as to be able to refer to it in case of doubts relating to lamp use, safety matters and other important information.

Never transfer the Product to another user or to other premises without its being accompanied by this operator's manual.

This manual must always accompany the Product.

These operating instructions must always be easily accessible to any Product user.

You are invited to carefully read this operator's manual before using the Product. This way, you can make best use of Product potential and protect yourself and others from any injuries.



MU_011_GB	19/04/13
Rev.2	Page 8 of 24

1.2 Precautions for safe appliance operation

Correct installation

This operator's manual is only valid after the correct installation of the Product, made in compliance with the valid installation instructions and with the correct startup by a professional installer. This operator's manual does not replace the obligation to instruct the user to carry out operations important for safety, operating, using and looking after the Product.

Safety provisions

The Product is made according to the current state of the art and its operation is safe, as long as it is used in compliance with all operating instructions and safety precautions.

Use of the Product can nevertheless be dangerous, especially if it is used by unqualified or inexpert persons or in an incorrect way, without abiding by the safety precautions contained in this operator's manual or in a way not in compliance with intended use.

To only be used in compliance with intended use

The Product is only designed to be used for the purposes indicated in this operator's manual. Any other use could cause mortal danger and/or hazards for the Product and the other material assets of the operator.

2 Precautions for the appliance operator

2.1 Technical safety specifications

Cleaning personnel

The Product cleaning and disinfecting operations described in Chapter 5 must only be performed by duly trained personnel.

Servicing personnel

The inspection and maintenance operations described in Chapter 6 must only be performed by professional technical personnel.

2.2 Personnel training obligation

Instructing users

Instruct personnel according to the operating instructions as regards controlling, cleaning and looking after the lamp.

The operator must provide such personnel with written instructions based on this manual.



MU_011_GB	19/04/13
Rev.2	Page 9 of 24

2.3 Warranty and liabilities

TECNO-GAZ disclaims all liability as regards unreliable Product operation in the following cases:

- assembly, changes and repairs not being made by a technician who has attended a training course on the Product organised by the manufacturer or by a professional technician,
- the Product not being used for the purposes for which it was intended, in compliance with the operating rules and instructions.

2.4 Structural changes or variations

Arbitrary changes

For safety reasons, no arbitrary structural changes or variations to the Product are acceptable. In case of changes or transformations of this kind, the manufacturer's Product warranty shall be invalidated. The manufacturer thus disclaims all liability for any damage or injuries caused by any arbitrary structural modifications or variations made or the use of non-original spare parts.

Only use original TECNO-GAZ spare

The use of parts not supplied by TECNO-GAZ or its dealers shall invalidate the warranty.

2.5 Disposal after use

Disposal at the end of life cycle

The used Product contains valuable materials which can be recycled.

Dispose of the used Product in an environment-friendly way and in compliance with applicable national directives on waste disposal.

3 Importance of personal safety

3.1 Intended use

Use in compliance with

standards The Product is made to light up the area occupied by the patient undergoing

surgery or observation and has been designed for use in operating theatres

or medical surgeries.

Field of work The Product correctly lights up the field of work from a distance of about 70

- 140 cm from the point of operation.



MU_011_GB	19/04/13
Rev.2	Page 10 of 24

Single lamp:

Definition

In compliance with the IEC60601-2-41 standard, a single lamp (ORION 60) is a secondary scialytic lamp for surgery and can only be used in operations where the interruption of lighting does not cause risks for the patient.

System of operating lamps:

Definition

In compliance with the IEC60601-2-41 standard, a system of lamps made up of several lamp units can be used to locally light up the patient's body without any limitation. It is also suitable for continuous function.

It enables the surgeon to operate also in the most difficult conditions of visibility. It is intended to make treatment and diagnosis possible and to be used in operating theatres.

Undesired effects caused by superimposition of light fields If the light fields of several lamp units are superimposed, there will be an increase in heat in the patient area with consequent dehydration of tissues and, above all in the case of prolonged operation and reduced blood supply, considerable damage to tissues.

If reduced blood supply or the start of tissue dehydration occurs, reduce the light intensity.

3.2 Environmental conditions

- The Product is not suitable for use in explosion-risk areas.
- The Product is not suitable for use in the presence of inflammable mixtures of anaesthetics with air, oxygen or NO₂ (laughing gas).
- During operation, the ambient temperature must be between 10°C and 40°C.
- Relative humidity must be between 30% and 75%.
- Atmospheric pressure must be between 700 and 1060hPa.

3.3 Use in combination with other medical products

- the Product can be equipped with appliances of other manufacturers.
 Refer to the operating instructions for such appliances.
- Only fit medical devices (e.g., LCD monitors) bearing the CE mark.



MU_011_GB	19/04/13
Rev.2	Page 11 of 24

3.4 Technical safety conditions

The safe use and proper operation of the Product is ensured if:

Safe fastening The lamp is safely fastened to the ceiling/wall from a static viewpoint and a

static stability test exists,

Wiring systems The wiring systems of the premises involved are in compliance with

applicable local regulations,

Authorised personnel Changes to the lamp or maintenance jobs are performed by personnel

trained by TECNO-GAZ or by a professional technician

Correct assembly

and start up

Original spares

The Product has been installed following currently valid installation

With regard to assistance, repairs, structural changes and additional

instructions and has been started up by a professional installer,

accessories, only original TECNO-GAZ spare parts are used.

3.5 Other safety conditions (secondary effects)

Optical safety - Do not direct the light source into the patient's and/or operator's eyes.

- Obligation to adequately protect the patient's eyes.

Failure to follow such precautions could cause glare and potential damage to

the retina.

Incorrect use - Never place and/or hang anything on the Product.

Unless this precaution is taken, positioning will not be reliable and the

danger exists of such objects falling in the operating area.

- Never hang on the Product with the body weight of a person.

Unless this precaution is taken, mechanical parts of the Product could be

damaged.

Covering the heads - Never cover the head of the Product during operation.

Failure to comply could prevent heat exchange with the environment and the

Product could overheat.

Knocks - Avoid knocking the rocker arms and Product head.

A violent knock could damage the Product and pieces of paint could chip off

and fall onto the operating field in the patient area.



MU_011_GB	19/04/13
Rev.2	Page 12 of 24

3.6 Graphic symbols used in this manual

In these operating instructions and on the lamp itself, important indications are marked by means of symbols and notice words.

Notice words such as HAZARD, CAUTION or IMPORTANT indicate the classification of the risk of suffering injuries.

HAZARD indicates an immediately hazardous situation which could result in death or serious injuries.

CAUTION indicates a potentially hazardous situation that could result in death or serious injuries.

IMPORTANT indicates a potentially hazardous situation which could result in moderate or light injuries.

The following triangular symbol together with the explanation alongside indicates the type of hazard to be dealt with:



Electric shock, Mechanical hazard from sprung masses (quick break of a damped arm during installation)

3.7 Other graphic symbols used on the Product

Below are the symbols to be found on the Product:



Type B appliance. Indicates the degree of protection against direct and indirect contacts.



Graphic symbol certifying product CE marking



Symbol indicating the date of manufacture (month and year)



Fuses used in the device



MU_011_GB	19/04/13
Rev.2	Page 13 of 24

4 Product description and operation

4.1 Product description

Versions The Product is available in different versions:

mobile version

- single ceiling version

double ceiling version (scialytic lamp system)

See drawing 134 MOBILE version: base with rollers (1), power supply unit (2), stand cover (3),

lower stem (4), upper stem (5), swing arm (6), fork (7), lamp head (8),

function control keyboard (9), sterilizable grip (10), power socket (11).

See drawing 109 SINGLE CEILING VERSION: ceiling cover (1), ceiling anchor tube (2),

power supply unit (3), horizontal arm (4), swing arm (5), fork (6), lamp head

(7), function control keyboard (8), sterilizable grip (9).

See drawing 110 DOUBLE CEILING VERSION: ceiling cover (1), ceiling anchor tube (2),

power supply unit (3), double horizontal arm (4), swing arm (5), fork (6),

lamp head (7), function control keyboard (8), sterilizable grip (9).




MU_011_GB	19/04/13
Rev.2	Page 14 of 24

4.2 Description of operation


Control panel

The function control keyboard is fitted to the Product fork. This keyboard makes it possible to switch the lamp on and off by means of the membrane key I/O (1), light intensity can be adjusted by pressing the keys with the sun symbol (2) with display of intensity reached by means of 5 green position micro LEDs. The colour temperature between the two values 4500°K and 5000°K can be selected by pressing the key with the letter K (3) while the set value is displayed by means of the lighting up of one of the 2 green micro LEDs. The "Endoled" function can only be switched on when the lamp is off, by means of the key with the letter E (4). Finally, the light range can be adjusted by means of the key (5) which extends or narrows the range. Display is by means of the lighting up of one of the 2 green micro LEDs:

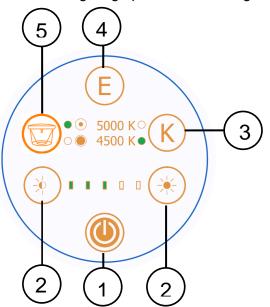

Wall control panel

On request, the Product can be equipped with a wall panel for controlling functions. Such control also permits controlling lamp functions by means of a supplementary keyboard featuring the same functions already previously shown with regard to the standard control panel fitted to the Product structure.

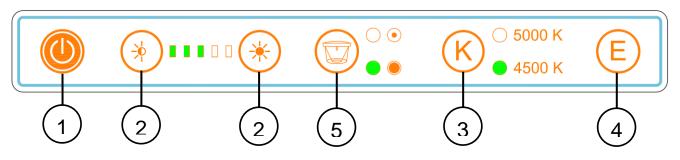



MU_011_GB	19/04/13
Rev.2	Page 15 of 24

5 Cleaning and disinfecting

5.1 Cleaning the Product



CAUTION – Electric shock hazard

Switch the Product off by means of the operating theatre main switch and make sure it cannot be switched back on.

Protect the Product against water spray and do not clean it/disinfect it with liquids.

IMPORTANT

Leave the lamp body to cool down. Only clean the lamp body when it is cold. Clean with appropriate detergent with low alkaline content and chlorine free.

Do not use abrasive products, petrol, paint thinners, alkaline detergents, acids, containing alcohol or aldehydes;

Dose the detergents so no liquids penetrate into the lamp bodies and into the support arm system.

Clean the Product with a damp but not wet cloth.

5.2 Disinfecting



CAUTION – Electric shock hazard

Switch the Product off by means of the operating theatre main switch and make sure it cannot be switched back on.

Protect the Product against water spray and do not clean it/disinfect it with liquids.

Leave the lamp body to cool down. Only disinfect the lamp body when it is cold.

CAUTION

Disinfectants can contain substances which are harmful for the health: only use disinfectants in accordance with the rules on hygiene established by the hospital.

The Product operator must comply with the rules established by the national commission for hygiene and disinfection.



MU_011_GB	19/04/13
Rev.2	Page 16 of 24

IMPORTANT

To prevent damaging parts in stainless steel or aluminium, only use disinfectants which are chlorine and halogen free.

To prevent the plastic parts becoming fragile, use only disinfectants with low alcohol content.

Dose the disinfectants so no liquids penetrate inside the lamp bodies and into the support arm system.

Clean the Product with a damp but not wet cloth.

5.3 Sterilizing handpieces



CAUTION – Hazard for the patient

Replace the handpieces as soon as these become cracked or deformed, as these could fall in the wound area.

The Product operator must comply with the rules of the national commission for hygiene and disinfection.

Handpiece fitting / removal:

- press the handpiece safety key and remove the handpiece.
- insert the handpiece up fast and turn it until it fastens on and rotation is blocked.

Cleaning, disinfecting and sterilizing the handpiece:

The handpieces are made of plastic material resistant to heat and knocks (PPSU).

They can be cleaned with a lightly-alkaline detergent free of active chlorine.

To disinfect the handpieces, we suggest using alcohol or aldehyde-based products. The disinfectants must be approved by the manufacturer for use on polyphenylsulfone (PPSU).

Before sterilizing, rinse the handpieces.

The handpieces can withstand about 300 steam sterilization cycles as follows:

steam sterilization at 121°C 1.3bar from 25 to 30 minutes,

or

steam sterilization at 134°C 2.3 bar for 4 minutes.

Position the handpieces straight with open side downwards.

Do not exceed a sterilization temperature of 134°C.



MU_011_GB	19/04/13
Rev.2	Page 17 of 24

Avoid the handpieces coming into contact with other objects during the disinfection process.

Each Product, over time, is subject to a certain amount of wear. Product safety and operation must therefore be checked during inspection and maintenance intervals.

5.4 Yearly inspections by operator



IMPORTANT

Keep to the yearly inspection schedules and inspect the product according to IEC 62353 standard.

5.5 Repairs



CAUTION – Unsuitable repairs

The Product must only be opened and repaired by a technician who has attended a course on the Product organized by the manufacturer or by a qualified technician in possession of the necessary technical skills.

6 Adjustments

6.1 Setting the rocker arm

See drawing 111

The Product is sold already balanced and does not require further adjustment. In the event of the swinging arm with spring balancing becoming stiff or loose over time, mechanical intervention is possible by regulating the compression of the internal spring.

Loosen the two stop dowels (1) which secure the cover (2) and move this forward. Fit a pin (3) with max diameter of 7 mm in the holes of the ring nut and turn in the direction indicated by the arrows to increase/decrease the load on the spring.

If the swing arm drops, this means the elastic force of the spring is insufficient:

turn the lever downwards and load the spring.



MU_011_GB	19/04/13
Rev.2	Page 18 of 24

If the swing arm continues to lift up, this means the elastic force of the spring is too high:

- turn the lever upwards and release the spring.

After making adjustments, return the covering to its original position.

6.2 Adjusting the braking force

See drawing 111 The brakes are set during installation. Like all the other mechanical parts,

the brakes are also subject to wear.

If the lamp body does not remain in any position it is set, the braking force

will have to be adjusted by means of the brake screws.

Horizontal arm brakes Use a suitable slotted screwdriver to increase the brake force, by turning the

arm brake screws (4) and (5) clockwise.

Fork brakes To increase the braking force at the head, turn the two brake screws (6 and

7) clockwise using a slotted screwdriver.

6.3 Troubleshooting

no.	Problem	Solution
1	The Product does not remain in position	Make sure the plate fitted on the wall (wall) is perfectly flat, that the stem is flat on the base (mobile) and that the tube secured to the ceiling (ceiling) is level. Further tighten the brakes on the joints so as to increase friction.
2	The Product fails to work	Make sure fuses have been fitted inside the terminal board. Make sure the electrical connectors are fitted. Make sure there is power voltage in the lamp head (18/26VDC)
3	The fuse continues to burn out	Check the specifications of the fitted fuses T2A (primary) and T10A (secondary) with 230VAC for ORION 60 T4A (primary) and T10A (secondary) with 100VAC for ORION 60 T1A (primary) and T6.3A (secondary) with 230VAC for ORION 40 T2A (primary) and T6.3A (secondary) with 100VAC for ORION 40 T4A (primary) and T10A (secondary) with battery ORION 60
4	The light flickers and produces a stroboscopic effect	Contact the after sales service.
5	The light beam on the operating field is not focalised (defective meeting of light fields)	Contact the after-sales service.
6	The Product does not switch on	Check the supply power voltage and check the fuses. The electronics are faulty: contact the after-sales service.



MU_011_GB	19/04/13
Rev.2	Page 19 of 24

6.4 Routine maintenance

no.	Period	Job
1	Every 6 months	Inspect all the lamp joints and make sure they do not squeak. If they do, add white grease to the clutches involved. If the Product does not maintain the position, adjust the clutches. To determine which clutches to adjust, see point 6.2 .
2	Once a year	Make sure the Tiges retention screws are tightened properly. Also check the 6 horizontal arm retention screws and the 3 swing arm screws. If these are not properly fastened, adequately tighten.
3	Once a year	Check the integrity of the leads from the PCB to the board and that these are fastened properly. If they are not, proceed to correctly tighten. INTERRUPT THE POWER SUPPLY BEFORE PERFORMING THESE OPERATIONS.
4	Once a year	Make sure the line voltage is correct. Make sure 24V are reaching the board.
5	Once a year	Check the condition of the Product paint. Make sure there are no paint pieces that could fall on the operating field.

6.5 Spare parts list

Description	Order code
Sterilizable grip	Z200518
Electronic board ORION 60/ORION 40	Z300632-PL81/ Z300632-PL30E
Keyboard ORION 60/ORION 40	Z300227



MU_011_GB	19/04/13
Rev.2	Page 20 of 24

7 Technical properties

Technical details of light	ORION 60	ORION 60+ORION 40
Illumination E _c at 1 m distance ± 10% (5,000°K) [Lux]	160,000	160,000 + 130,000
Illumination E _c at 1 m distance ± 10% (4,500°K) [Lux]	160,000	160,000 + 80,000
Double selection colour temperature (±5%) [K]	4.500) / 5.000
Colour rendering index R _a at 4,500°K / 5,000°K [-]	96 / 96	
R ₉	90	
Number of LEDs	No.81	No.81 + No.30
Focus	Fixed	Fixed + Fixed
Light range variable between – a (cm)	19-32	19-32 ORION60 16-28 ORION40
Light range diameter d ₅₀ [mm]	100	100 + 80
Light range diameter d ₁₀ [mm]	190	190 + 160
Lighting depth L1+L2 [cm]	26 + 17	26 + 17 (34 + 15)
Max irradiation [W/m²]	392	392 + 299
Irradiation / Illumination [mW/m²lx]	2.45	2.45 + 2.3
Max irradiation in UV [W/m²]	0.002	0.002 + 0.002
Grip focalization	No (electronic focalization)	
Power connection details		
Primary alternate voltage [Volt ac]	100 ÷ 240	
Secondary direct voltage [Volt dc]	24	24
Frequency [Hz]	50/60	
Power input [VA]	145	145 + 70
LED light source [No.]	No.81 LEDs	No.81 + No.30 LEDs



MU_011_GB	19/04/13
Rev.2	Page 21 of 24

Duration of LED diode light source [hr] (this figure can vary according to a supply voltage higher than that indicated, to power peaks and operating frequency)		50,000
Light intensity control [%]		25-100
General data		
Colour	RAL 9003	
Directive	93/42/EEC and 2007/47/EC	
Standards	IEC 60601-2-41	
Classification of medical device product	Class I	
Protection against direct and indirect contacts	Type-B	
Dimensions		
Diameter of lamp body [cm]	63	63 + 40
Diameter of poly-elliptical reflectors [cm]	No.72x5.5 + No.9x3.2 ORION60	No.72x5.5 + No.9x3.2 OR.60 + No.30x5.5 OR.40
Light emission surface [cm ²]	1,782	1,782 + 712
Weight of mobile scialytic lamp, single ceiling lamp, double ceiling lamp 81+81, double ceiling lamp 81+30E [Kg]	80, 44, 70, 63	
Markings		
C€	In conformity with Directive 93/42/EEC and 2007/47/EC	
All lighting values are subject to a tolerance of \pm 6% reasons.	due to manufacturing	and metrological



MU_011_GB	19/04/13	
Rev.2	Page 22 of 24	

8 EMC compliance

The Product has been tested in accordance to EN60601-1-2 to ensure proper electromagnetic compatibility. Portable and mobile RF-communications equipment can affect the Product. Other products used in the vicinity of Product should also comply with this standard.

The Product is intended for use in the electromagnetic environment specified below. The customer or the user of the Product should assure that these are used in such an environment.

Emissions test	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The Product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The Product is suitable for use in all establishments other than domestic, and may be used in domestic establishments and those
Harmonic emissions IEC 61000-3-2	Class A	directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes, provided the following warning is heeded:
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Compliance	Warning: This equipment/system is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the Product or shielding the location



MU_011_GB	19/04/13
Rev.2	Page 23 of 24

Immunity test	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment-guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 6 kV contact +/- 8 kV air	+/- 6 kV contact +/- 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrical fast transient / burst IEC 61000-4-4	+/- 2 kV for power supply unit +/- 1 kV for input/output lines	+/- 2 kV for power supply lines +/- 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	+/- 1 kV differential mode +/- 2 kV common mode	+/- 1 kV differential mode +/- 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U _T (>95% dip in U _T) For 0,5 cycle 40% of U _T (60% dip in U _T) For 5 cycles 70% of U _T (30% dip in U _T) For 25 cycles <5% U _T (>95% dip in U _T) For 5 sec	<5% U _T (>95% dip in U _T) For 0,5 cycle 40% of U _T (60% dip in U _T) For 5 cycles 70% of U _T (30% dip in U _T) For 25 cycles <5% U _T (>95% dip in U _T) For 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Product requires continued operation during power mains interruptions, it is recommended that the Product be powered from an uninterruptible power supply or battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.



MU_011_GB	19/04/13
Rev.2	Page 24 of 24

Notes

