# SUMMARY

## ENGLISH

QUICK START RAYPLICKER™ DEVICE.....	3
RAYPLICKER™ BEST PRACTICES .....	4
QUICK START DENTIST SOFTWARE .....	5

## FRANÇAIS

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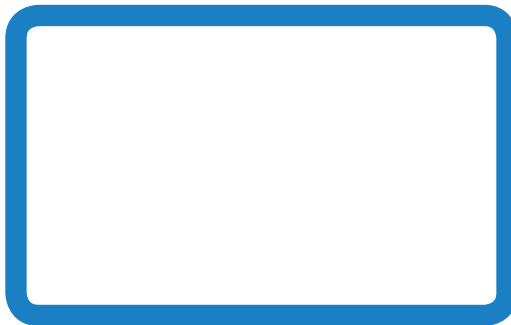
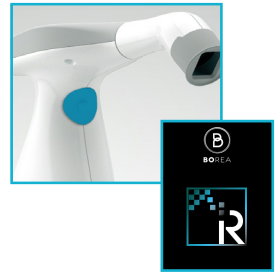
2

Place the protective sheath on the guide and then the calibration tip previously sterilized by autoclave. (134°C during 20 min)


3

Start the unit by pressing the switch.

4

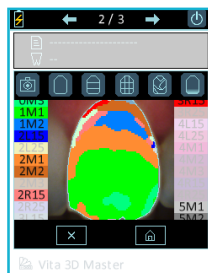Click on the «Camera» icon to launch the acquisition mode then press the button to perform the shade analysis. (see point 3) It is imperative not to move for 1 second, the time of the acquisition.

5

Click on the screen to view the different shade mappings and to delete the file.

6

Transfer the file to your Rayplicker™ software. To do so, connect the device to the computer through the docking station.

## RAYPLICKER™ BEST PRACTICES

1. The tooth must be in the center of the image and without inclination. The tip must be under the lip and make contact with adjacent teeth and gingiva. The gray edges of the tip are parallel to the edges of the image. Hold this position until the end of the acquisition (1 sec)

2. The tip is not well positioned. It must be parallel to the edges of the image, making sure to place the tooth in the center of the image. Also check the correct angle of the acquisition.

3. Open the patient's mouth (he must breathe through the nose during the acquisition). The tongue, lips and teeth opposite to that scanned should not be visible on the screen.

4. The tooth should be centered and the tip should not press too hard on the patient's gums. (To avoid the white coloring).

5. The tip is too far from the tooth and the lip should not appear on the image. Place the tip in contact with the gingiva and the teeth adjacent to the scanned tooth.

# QUICK START DENTIST SOFTWARE

Software downloadable on:

[www.borea.dental/Download](http://www.borea.dental/Download)

## I. MAIN PAGE

The screenshot shows the main interface of the BOREA software. At the top, it displays the user name "Dr. SMITH Jean". Below this is a search bar and a grid of file cards. Each card represents a patient's file, including the patient name, the laboratory name "Laboratoire Borea", and the modification date. The files are color-coded: green for finished orders and black for untreated orders. On the right side of the interface, there are icons for settings, import/export, and cloud synchronization. The BOREA logo is prominently displayed at the bottom center of the window.

Import files from Rayplicker™

Order sent to the lab (Blue File)

Settings

Import / Export

Cloud actualization

Untreated order (Black file)

Finished order (Green file)


## II. ANALYSIS TAB

Referent teeth selection

Color mapping

Image treatment tool:

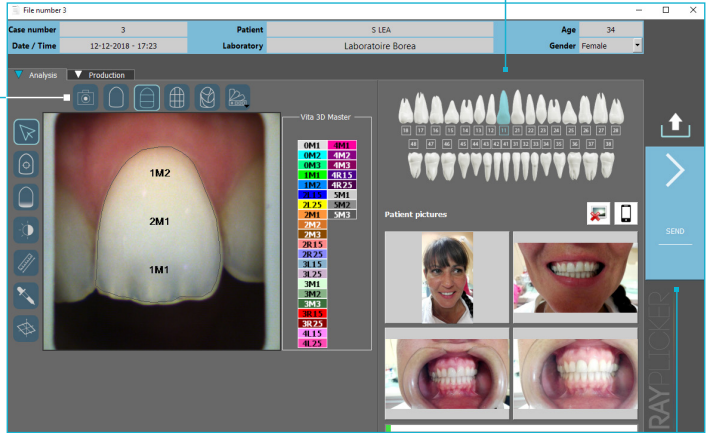- Tooth border
- Translucency
- B&W
- Ruler
- Cie Lab Analysis
- Colorimetric coordinates


Send the file

## III. PRODUCTION TAB

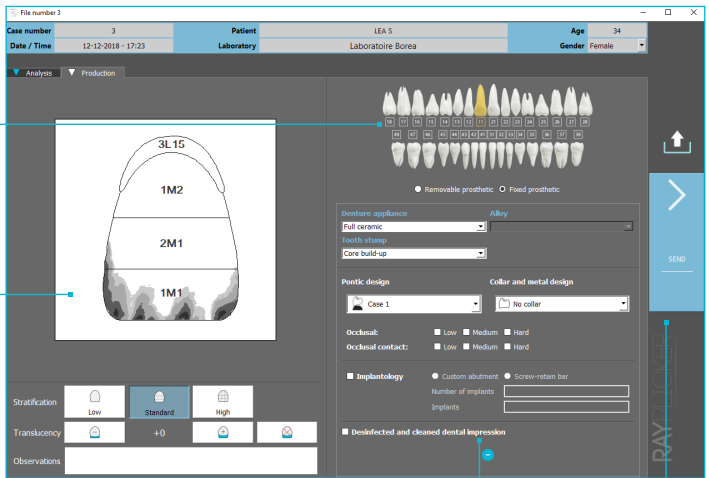Production tooth

Scheme of the production tooth

Information about the production tooth  
<https://stomshop.pro/>

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[www.borea.dental/Telechargements](http://www.borea.dental/Telechargements)

## I. PAGE PRINCIPALE

The screenshot shows the main interface of the BOREA software. At the top, there is a header with the user name 'Dr. SMITH Jean' and a gear icon for settings. Below the header is a search bar and a 'PaperKit' button. The main area displays a grid of patient files, each with a status icon (green for finished, black for not processed, blue for sent) and patient details. Callout boxes point to various features: 'Importer depuis Rayclipker' points to the top-left icon; 'Fiche envoyée (Bleue)' points to a blue file card; 'Paramètres' points to the gear icon; 'Import / Export' points to the 'PaperKit' button; 'Actualisation du Cloud' points to the refresh icon; 'Fiche non traitée (Noire)' points to a black file card; 'Fiche terminée (Verte)' points to a green file card.

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[www.borea.dental/Descargar](http://www.borea.dental/Descargar)

## I. PÁGINA PRINCIPAL

The screenshot shows the main interface of the BOREA software. At the top, it displays the user name 'Dr. SMITH Jean' and a gear icon for settings. Below this is a search bar and a grid of file cards. Each card represents a patient's file with a status icon (green checkmark for 'Pedido finalizado' or blue checkmark for 'Pedido no tratado') and a blue arrow icon for 'Importar / Exportar'. The file cards are arranged in a grid, and the BOREA logo is visible at the bottom of the interface.

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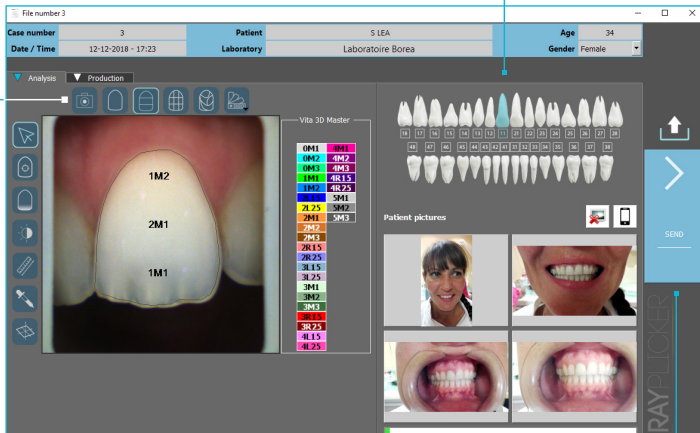
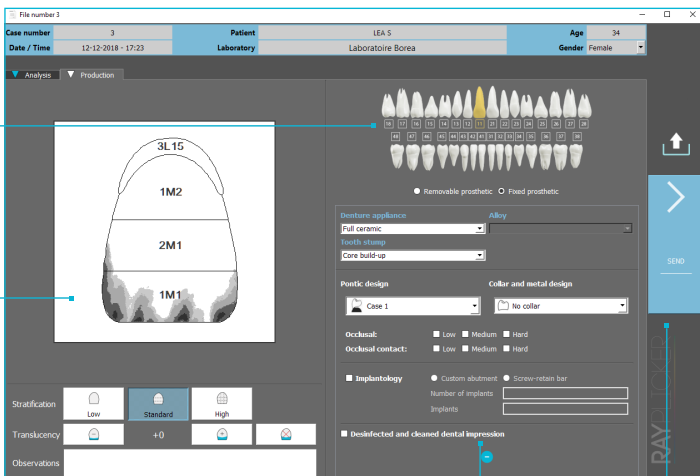
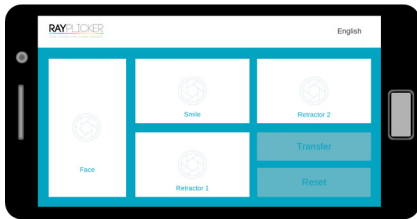
## Pics App Solution


1

First download the Rayclicker application on:

Google Play or Apple Store

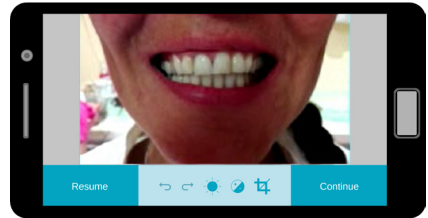
Open the Rayclicker application and take patient pictures.

2


Bottom menu bar allows pictures enhancements :

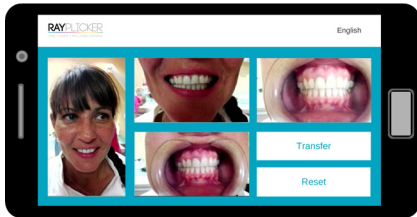Brightness  
Contrast  
Cropping

3

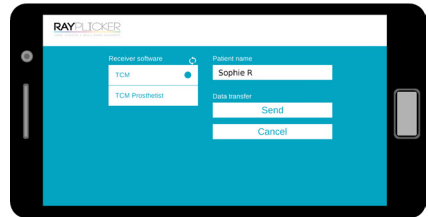
When complete, press «transfer».


4

Write a patient name and then select a software receiver\* before sending. Pictures will be automatically downloaded into the Rayclicker™ software by wi-fi.

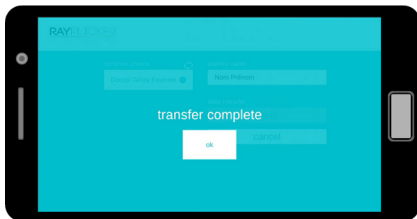\*Important: Make sure both your devices (phone and software) are connected at the same time to the same wi-fi network.

5

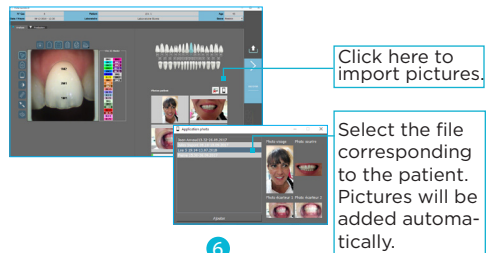6

Import contextual picture to your file on the software.

**BO**REA

CIRE-ESTER Technopole  
12 Rue Gémini  
87068 Limoges Cedex, FRANCE

+33 (0)9 83 71 71 61  
contact@borea.dental

www.**BO**REA.dental