... makes the difference

BASICS III

see it
feel it
test it

devemed

... makes the difference

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devemed GmbH is based in Neuhausen ob Eck by Tuttlingen, a city renowned worldwide for its medical instruments. The company was reborn in 2006, and can look back over more than 100 years of experience in the manufacturing of top quality dental instruments.

devemed unites exclusive design with the best comfort. Functional qualities, like the prevention of sediment build, unwanted slipping of the instruments or the reduction of distracting light reflection are standard implementations in our F-Line products. Micro-surgical instruments have a very important place within our large product range.

Our many years of experience have taught us that innovative and top quality dental instruments are essential criteria for the competence of a dentist. The requirements of the dentist have customized our new and further developed instruments because contemporary and advanced application techniques need high-tech instruments. For developing our instruments, we have the support of world renowned practicing dentists and experts.

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IMPORTANT NOTICE

This catalogue remains the property of devemed GmbH even if a nominal fee has been paid. We can recall the catalogue, at any time, in the event that it has not been used according to our company's interests, i.e. the sale or purchase of devemed products. In this case, we reserve the right to take further action.

The reproduction or copying of this catalogue in whole or in parts is forbidden without our written permission.

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Download:
<http://a-k-i.org/index.php?id=12>

Download:
<http://www.devemed.de/de-DE/download-service/>

Before each sterilisation, the instruments should be manually placed in a disinfection bath and/or ultrasonic cleaning unit/thermal disinfectant. The following instructions from the manufacturer must be kept:

- Use only appropriate cleaning and disinfection material, and under compliance of the dosage instructions
- **Do not use cleaning materials which contain chlorine or chloric substances**, as well as the corresponding chemical bonds of other halogens (**fluorine, iodine, bromine**)
- Instruments with hinges or connectors (e.g. forceps, scissors, or needle holders) should always be cleaned and sterilized in the "open" position.
- **Instruments and equipment made of aluminum** may not be put in the ultrasonic bath or in the disinfectant. Use only aluminum adapted cleaning and disinfection materials.
- Thoroughly rinse the instruments with demineralized water after cleaning. The demineralized water should remove the remaining cleaning materials from the instruments
- Remaining residue and encrustation should be removed with a soft brush (not a metal brush). Such residue can result in corrosion or other damage to the instruments during sterilization
- **Instruments have to be completely dry** before they are introduced into the sterilization process
- We solely recommend the use of autoclaves and steam sterilisation. All other methods carried out are exclusively at the doctors own risk. If hot air sterilisation is used, ensure that temperatures do not exceed 180° C. Higher temperatures can lead to non-removable discoloration and to a loss in hardness of the instruments.
- Instruments made of stainless steel may not be pooled together with other metals or materials during the sterilization process. Disregarding this instruction can cause oxidation spots to occur.
- Sterilized instruments have to be dry, dust-free and stored away from chemical products. Avoid large temperature fluctuations in the location, where the instruments are to be stored.

For further and more detailed information about these procedures, please pay attention to our references in the current brochure "Instrument reprocessing how to do it right" or in our "Guidelines for instrument reprocessing".

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Stains

- Insufficient manual or mechanical cleaning
- Regional water characteristics (e.g. high chlorine content) can lead to staining and damage
- Inappropriate cleaning, care and disinfecting materials
- Dosage instructions of the cleaning materials are ignored
- Sterilization steam is contaminated
- Pharmaceutical residue

Water spots, iridescent surface, discoloration

- Rinse water and sterilization steam contain a high concentration of minerals, heavy metal ions or silicates. Therefore only use demineralized water.
- Temper colors and water spots are not signs of corrosion. Remove these spots promptly by rubbing vigorously
- Inappropriate materials that contain high levels of chlorine or acid must be avoided
- Stainless steel instruments may never be pooled together with rusty instruments or instruments with damaged surfaces, to prevent contact corrosion

Corrosion

- Can also be caused by rust containing water used during cleaning or autoclaving. Only use demineralized water
- Thoroughly clean new autoclaves before initial use

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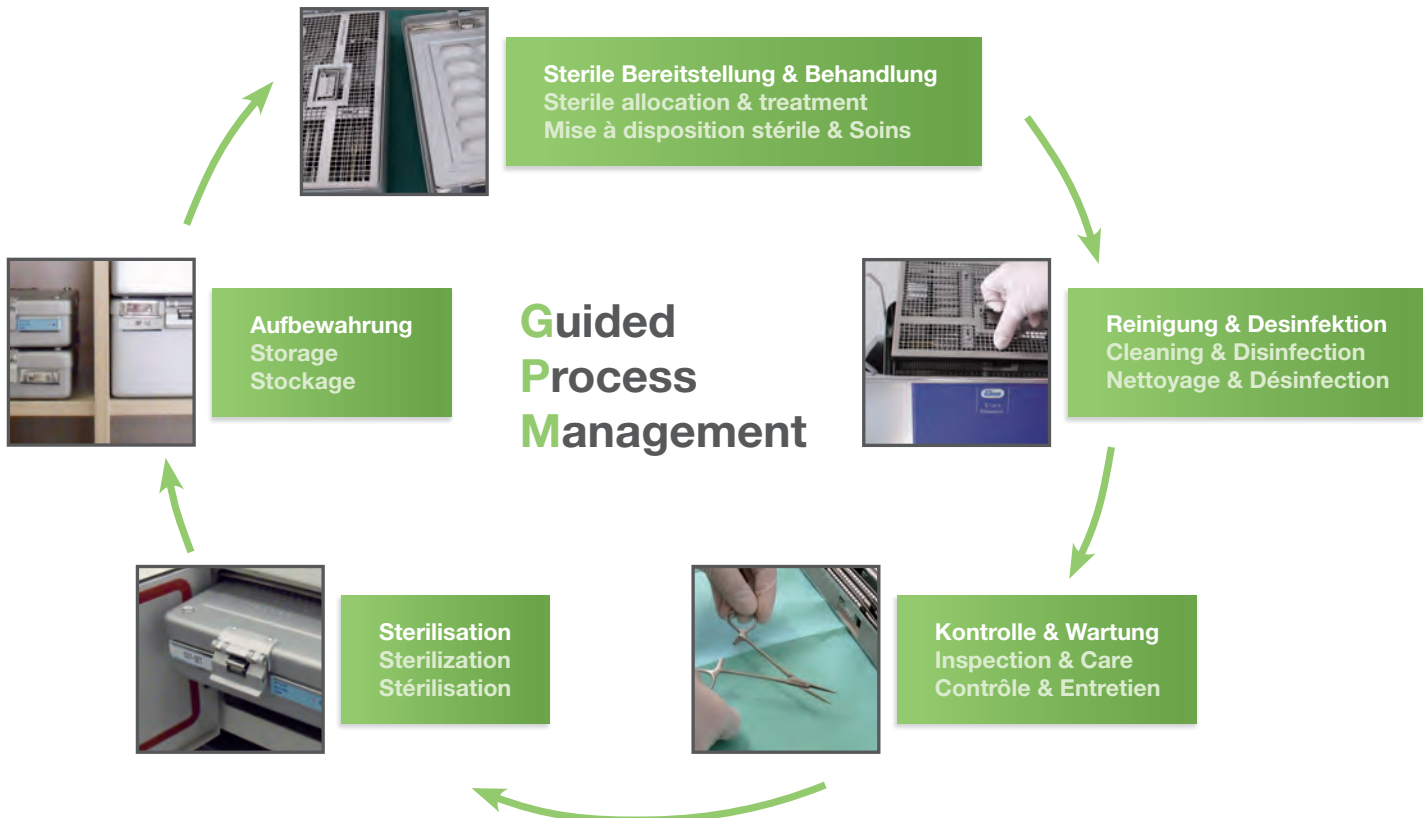
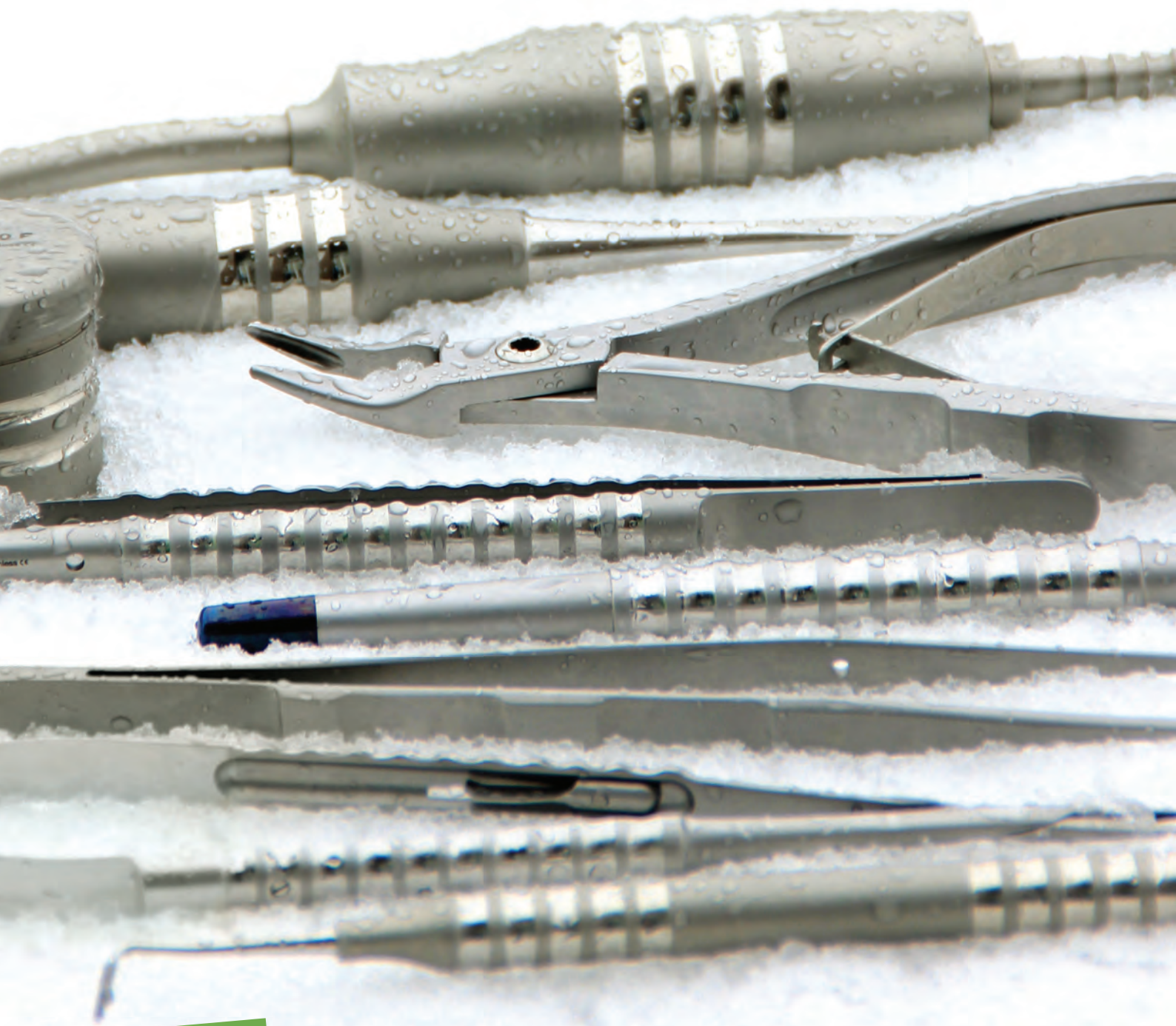
best quality

easy cleaning

consistent design with practical functionality

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The **F-line SUPERLIGHT** round handle has been developed by our engineers in consultation with our worldwide accredited experts. Its special grip profile and its sandblasted surface, gives the handle a unique look.

The hollow handle enables more comfortable, ergonomic and less-impact working. It has a diameter of 9 mm and very thin walls.

The sandblasted surface of the handle minimizes disturbing light reflections while working with magnifying glasses and microscopes. The finish also protects from the accumulation of deposits of dirt in the ridges and grooves.

In order not to exceed a total length of 175 mm, the handle is available in three different sizes, according to the size of the workpieces. This could also be a considerable advantage as it allows you to optimize the space in your wash trays.

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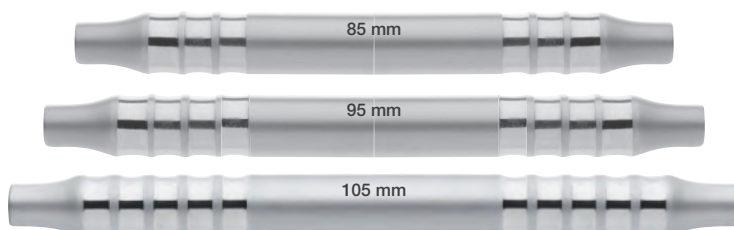
The newly developed solid **F-line ROUND SOLID** handle, which is orientated for pressure and tension, allows more tactile working. The conical form effectuates a direct and controlled power transmission to the tooth.

The ergonomic design avoids uncontrolled slipping and promises perfect comfort and easy cleaning.

The end caps of the handle are colour-coded, which allows easy identification of the instruments.

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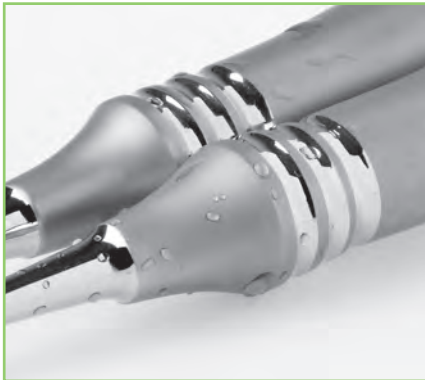
The hollow **F-line PEAR HANDLE** handle was developed for superior handling. Due to its pear-shaped design, it fits perfectly in the hand, ensures a good grip and avoids inadvertent slipping.

The **F-line**-design prevents the deposition of dirt and is easy to clean. Two flat surfaces are incorporated into the lower end of the teardrop handle to prevent the instrument from rolling away.

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The **F-line** PLAIN SURFACE design clearly shows why our FIRST LINE range is outstanding. The ergonomic handle fits comfortably in the hand and is especially suitable for instruments for which constant power transmission is important.

Interfering light reflections are reduced due to the sandblasted surface. The extra large and smooth surfaces, as well as the wide indentations prevent the deposition of dirt, thus enabling simple and clinically clean cleansing.

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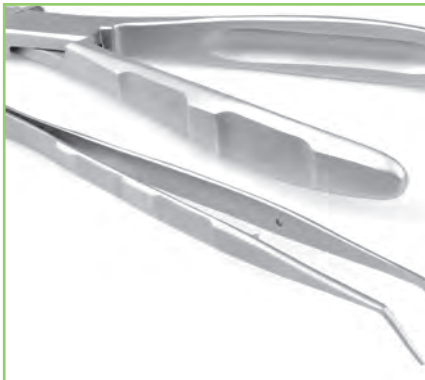
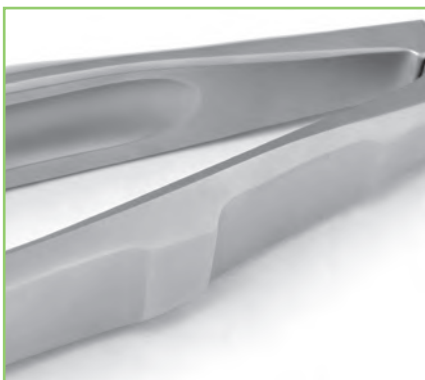
constant power transfer

plain surface

minimizes annoying light reflections

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For the S-tec (S = Standard) series, standardized shapes and dimensions of the workpiece ends have been combined with our **F^{line} ROUND SOLID** handle.

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With our Xperenced-tec tweezer series, those working tip dimensions lie between normal surgical instruments and micro instruments, we want to accompany surgical dentists on their way of the minimally invasive surgery.

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Xperenced-tec tweezers are used as a basis for micro surgery and help dentists to develop precision working. Therefore, use of Xperenced-tec tweezers assists with the optimal transition to our M-tec-forceps.

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Our M-tec instruments are coordinated to the dimensions and structures of micro surgery, and their materials and conditions. The slender workpiece ends facilitate working with magnifying glasses.

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Our U-tec-series with the **F^{line} ROUND SOLID** handle has been developed for dentists who are already familiar in the field of minimal invasive surgery. The ultra delicate workpiece ends are proportionally based on fabric structure and material and allow a new dimension of selective working with magnifying glasses or microscopes.

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Mini-Cut

One cutting edge is finely serrated. This prevents slipping from tissue and sutures.

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Super-Cut

The razor sharp cutting edge cuts effortlessly and softly through thicker material. The serrated cutting edge on the second blade prevents slipping from tissue and sutures. Super-Cut is not practical for wires, cartilage and other hard materials. One handle ring is gold plated to help with easier identification. Scissors in our F-line Design are marked by **SC**-Symbol.

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Wolfram-Carbide

The inner-edge of both cutting blades are wolfram-carbide coated. This prevents slipping. Both handle rings are gold-plated for easier identification.

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Tungsten-Carbide

Tungsten-carbide is incorporated into the blades which enables a precise and gentle incision, as well as a long lifespan. For easier identification, both handle rings are gold plated. Instruments in our F-line Design are marked by **TC**-Symbol.

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Titanium nitride

Due to the titanium nitride hard surface coating, the adhesion of the filling materials is greatly reduced.

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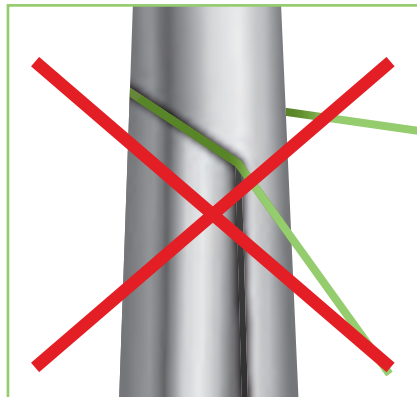
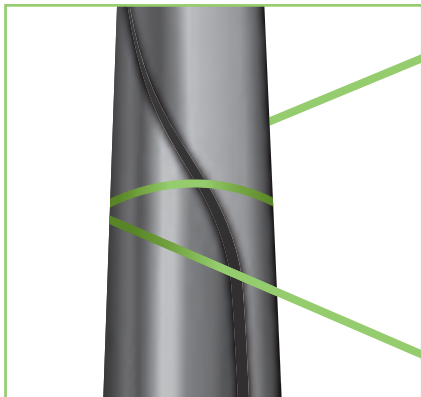
The curved tips of the “pivot-curved” lock of micro needle holders and micro scissors prevent the suture from getting caught and breaking in the instrument. The drilled pivot stabilises the working tips and the round handle eases the rotatory motion in the hand.

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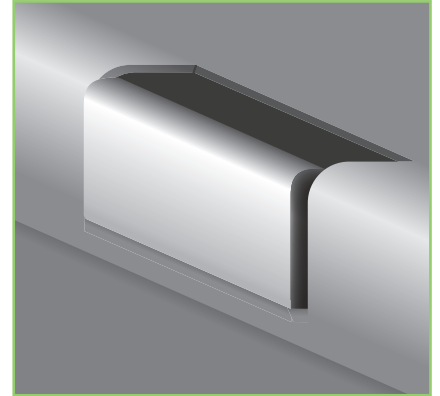
The newly developed pressure cap *easy lock* ensures that the GPM Tray can be opened and closed comfortably and without complications.

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geschlossen
 closed
 fermé

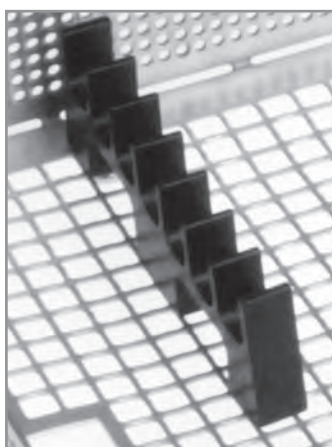
GPM System | Trays

GPM System | Trays
GPM System | Cassettes

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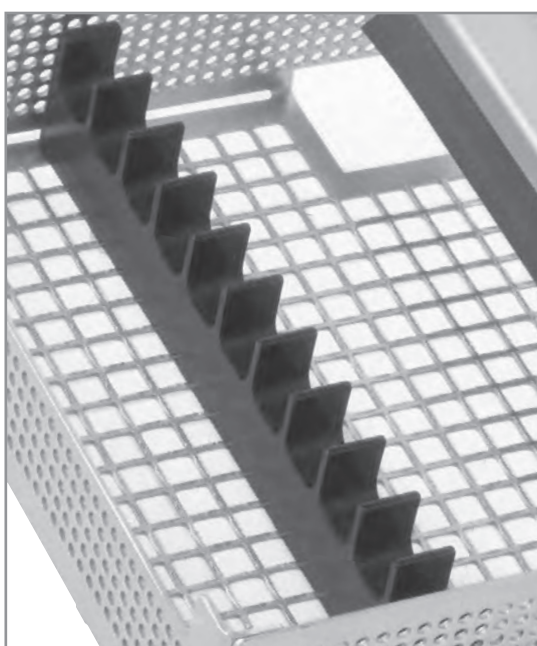
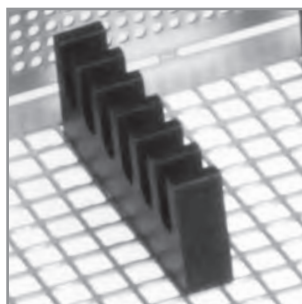
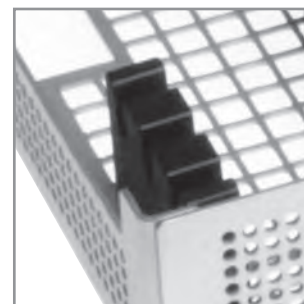
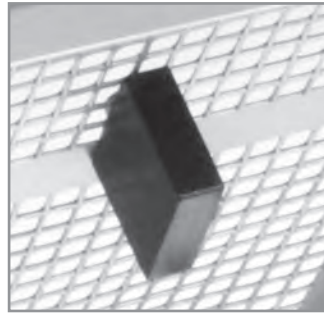
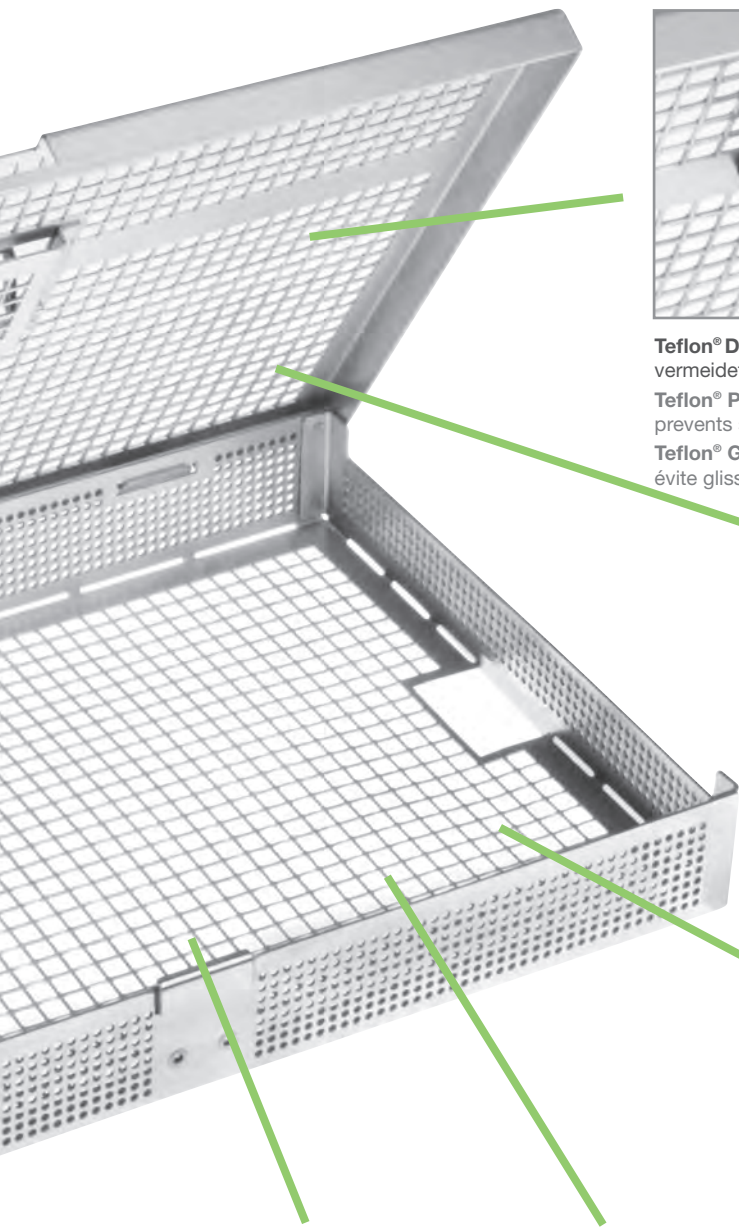
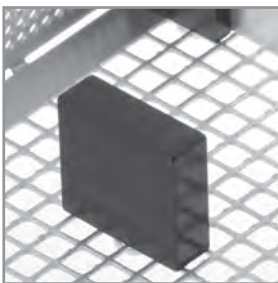
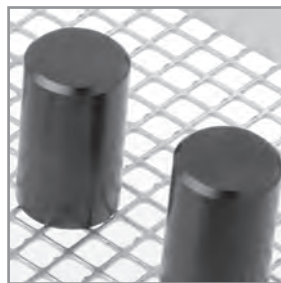
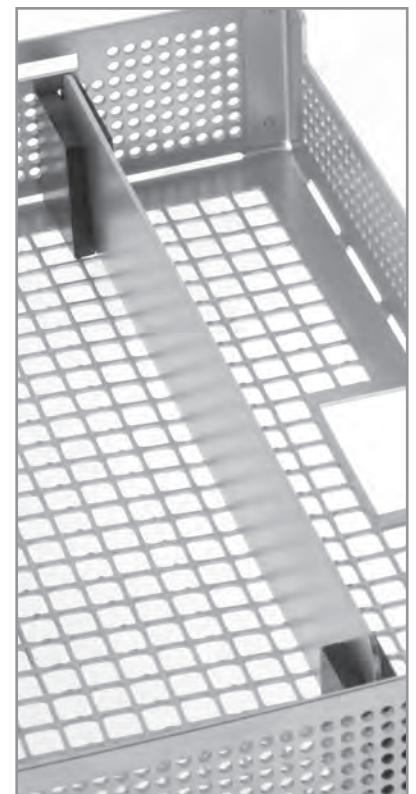
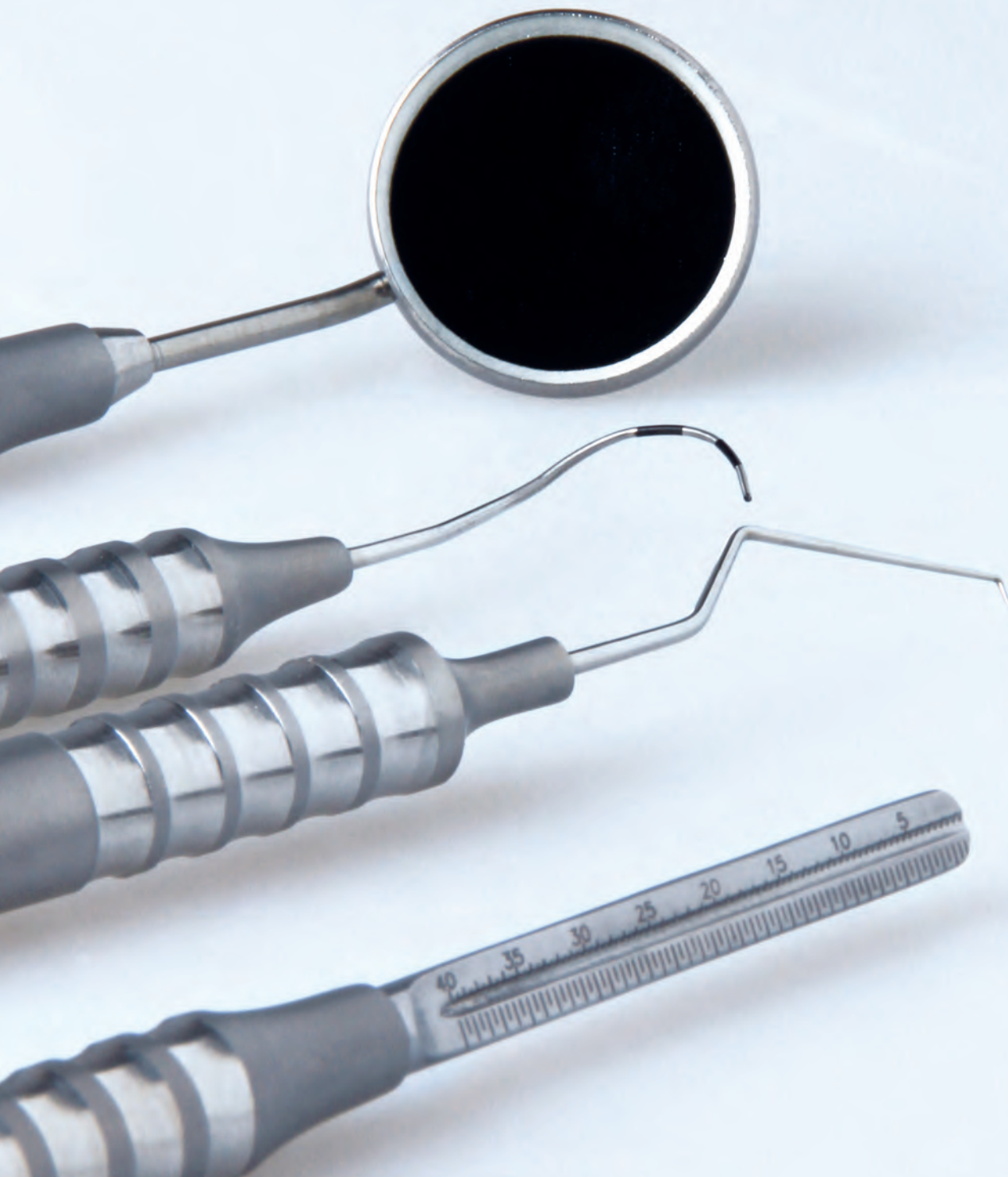
DIAGNOSTIK

DIAGNOSTICS
DIAGNOSTIC

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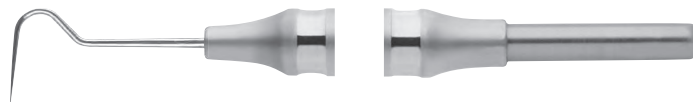
 Probes | single ended
 Sondes | simples

F-line SL

| | |
|------------------|---|
| 1444-05 F |  |
| # 3A | 16,4 cm |

| | |
|------------------|---|
| 1444-10 F |  |
| # 8 | 15,6 cm |

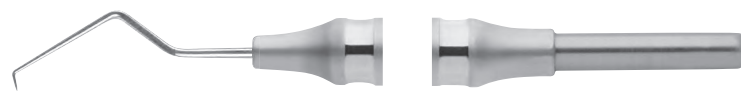
| | |
|------------------|---|
| 1444-15 F |  |
| # 9 | 16,2 cm |

| | |
|---------------------|---|
| 1444-20 F |  |
| # 23 „Schäferhaken“ | 16,0 cm |

| | |
|------------------|---|
| 1444-25 F |  |
| # TU17 | 16,4 cm |

Zahnsonden | doppelendig

 Probes | double ended
 Sondes | doubles

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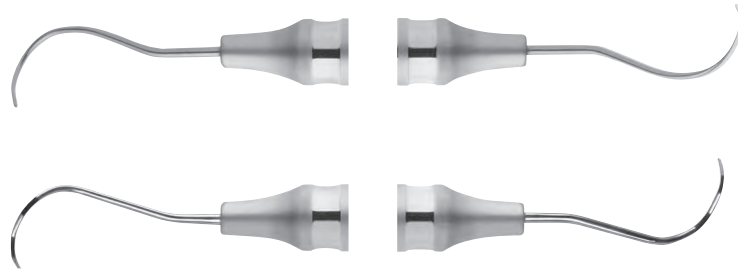
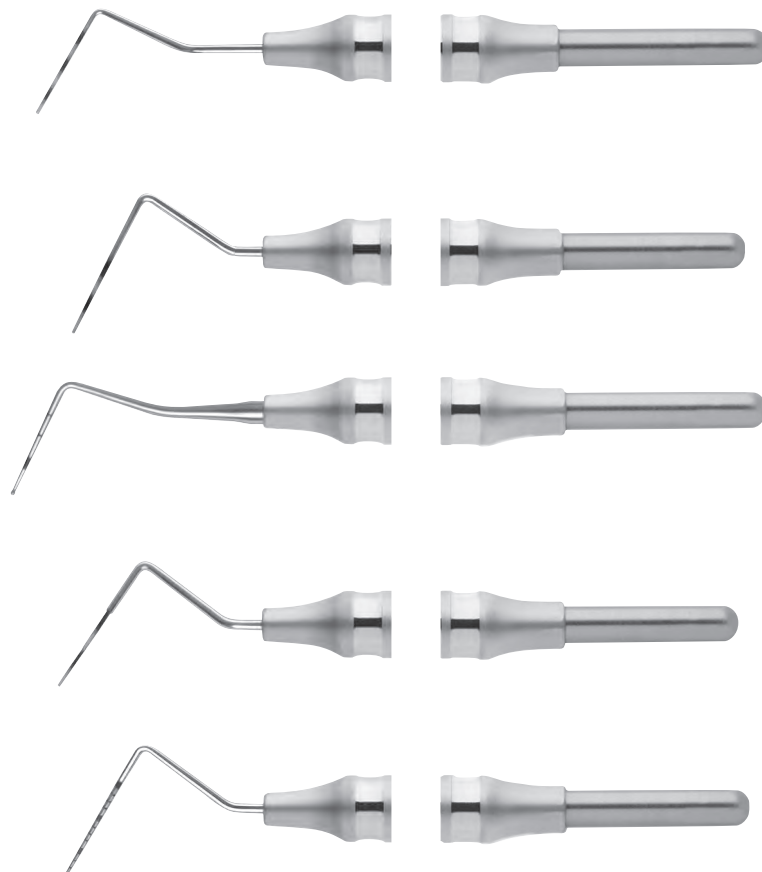
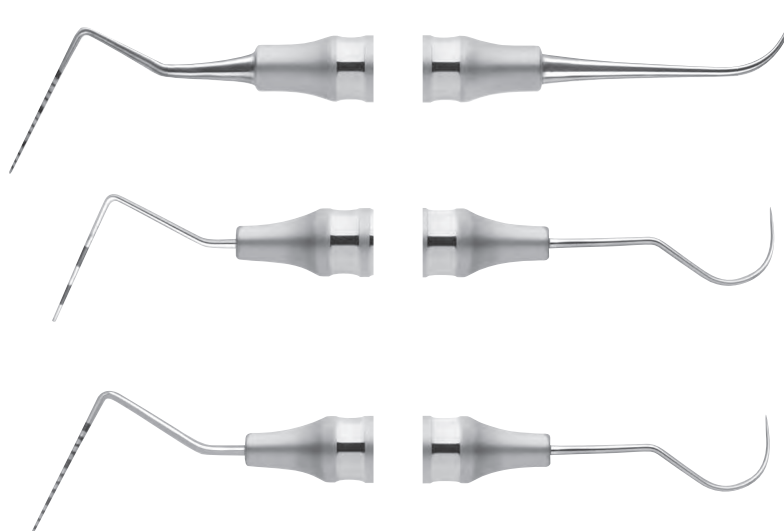
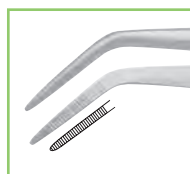
The „College modified“ promises with its wide and ergonomically handle an very well comfort and ensure a lower-impact working during long use. Its special form allows a higher power transmission in contrast to usual tweezers.

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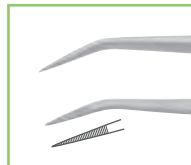
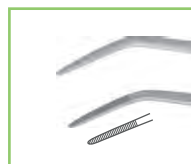
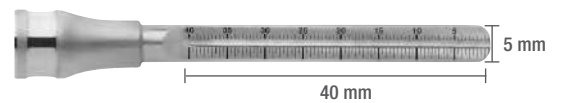
MEGA and MEGAduo

- Pin-sharp accuracy, distortion-free and a high-contrast image
- Utmost degree of optic convenience – fatigue-proof and eyefriendly in all fields of application
- Low-reflection due to semi gloss surface
- Easy cleaning due to Rhodium with non-adhesive effect
- Absolutely resistant against all kinds of disinfection and sterilization
- Comfortable holding back of the cheek because of the wide-shaped handle
- Hygienic as there are no weldings where deposits can form
- slight in design

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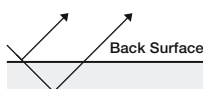
The advantage of rhodium plated front surface (FS – Front Surface) is an undistorted and clear image.

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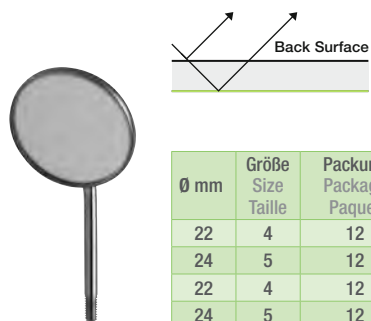
This classic mouth mirror has the reflective coating on the back side of the mirror glass.

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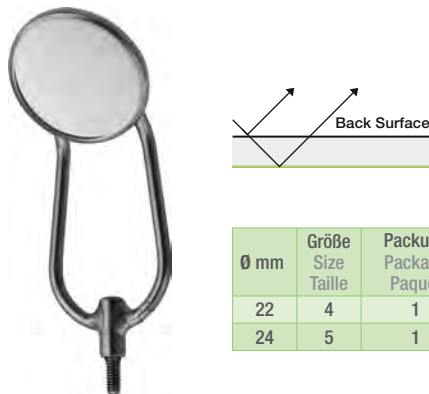
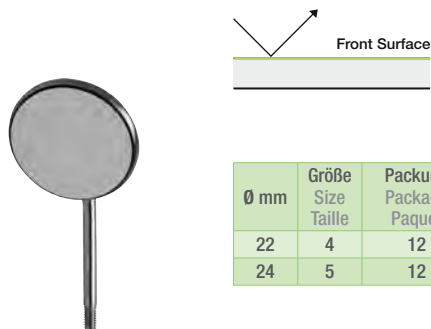
Filling instruments | ball-shaped | Dykal
Instruments à sculpter | globulaire | Dykal

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Titanium nitride

Adhesion of the composite is significantly reduced due to the titanium nitride hard surface coating.

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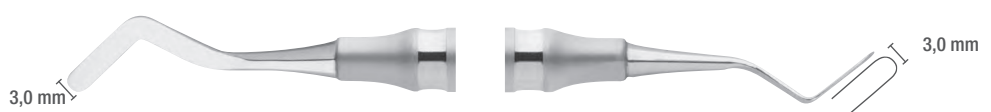
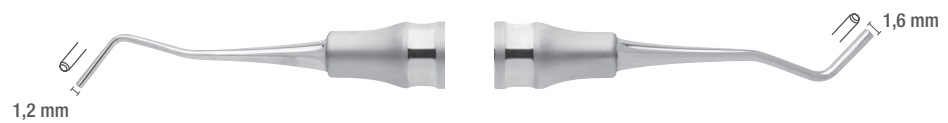
ECI | Composite instruments for aesthetic restorations of teeth

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(ECI=ESTHETIC COMPOSITE INSTRUMENT)

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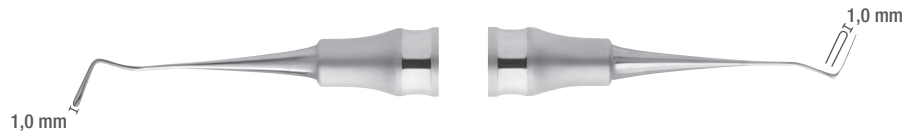
This delicate, flexible and spatulated instrument is used for burnishing and carving in very confined spaces. It can be helpful also when working with very small quantities of plastic-ceramic filling materials.

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This delicate, flexible and spatulated instrument is used for burnishing and carving in proximal areas with more difficult access.

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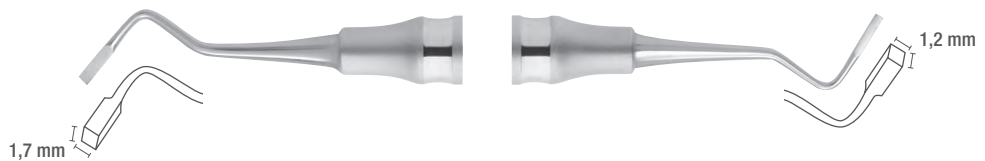
Instrument for the final design of the occlusal convexities.

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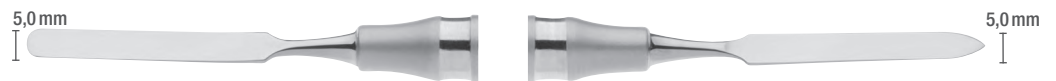
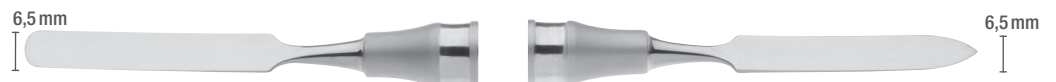
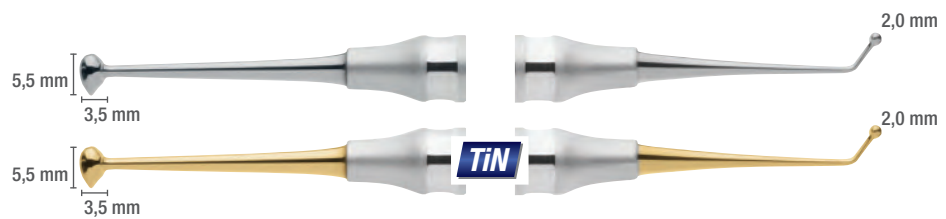
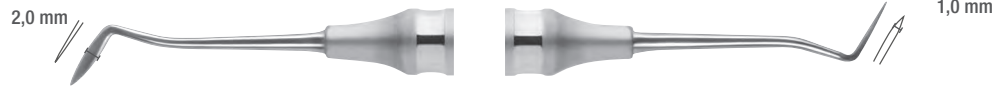
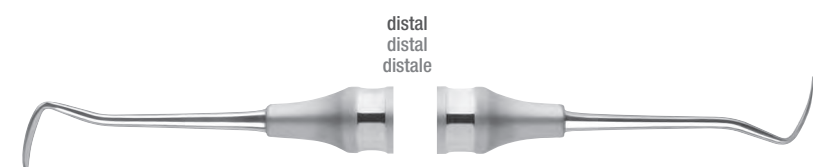
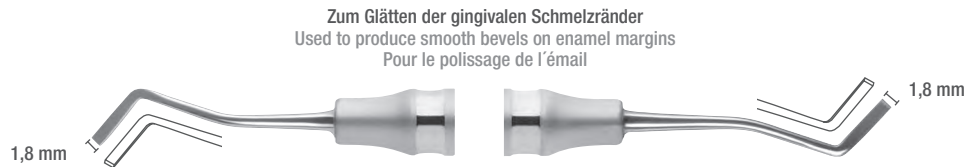
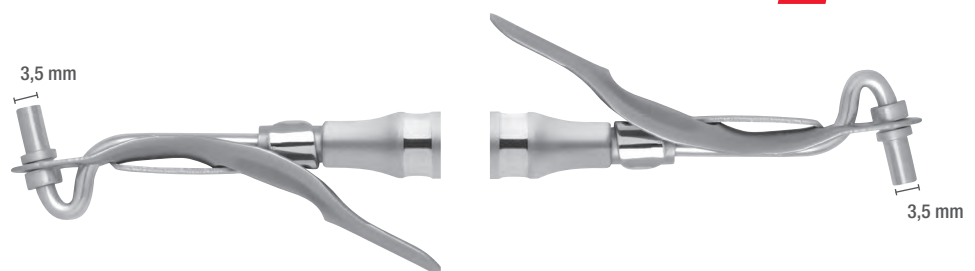
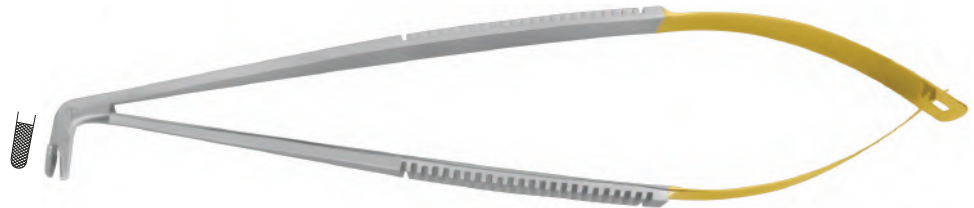
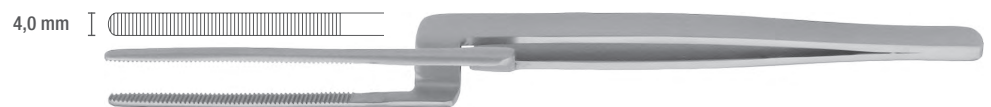
Glass plates for mixing
Plaques de verre

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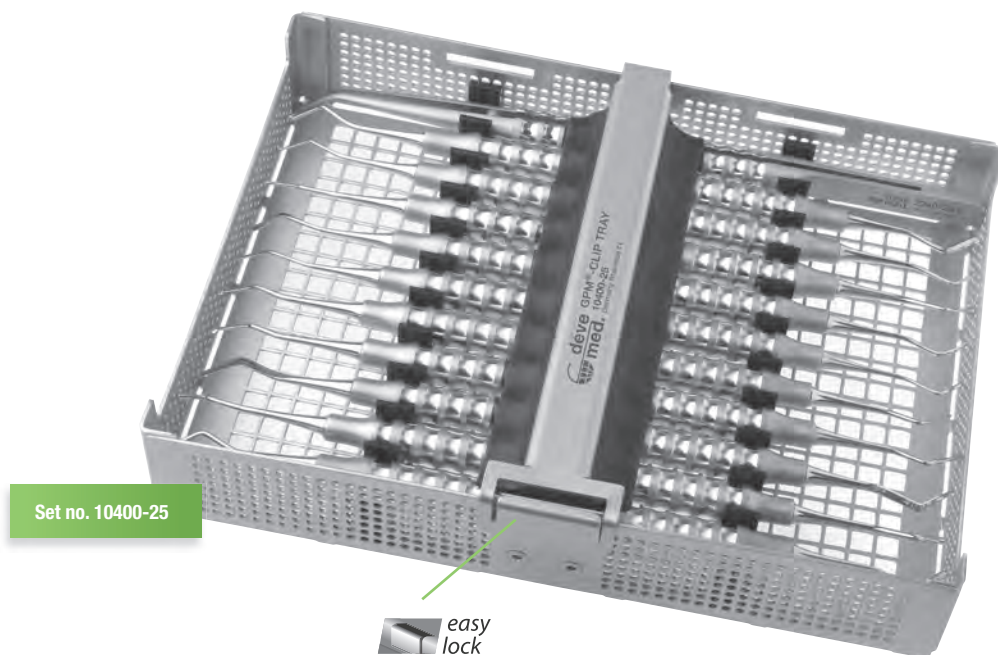
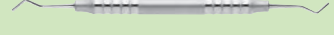
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
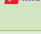
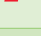
For the removal of supragingival calculus.

These very thin and counter angled scalers are particularly suitable for interproximal surfaces. Due to the slightly angled working ends, the mesial and distal surfaces also can be reached.

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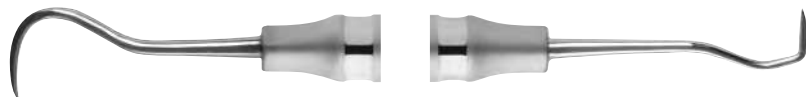
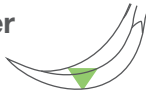
Jacquette

Jacquette scalers are fine instruments for removing supragingival calculus and concretions and also for smoothing the root surface. At each end of the sickle-shaped scaler, there are two tapered cutting edges.

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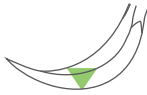
McCall

McCall scalers have tapered, pointed working ends and are highly suitable for interdental cavities.

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One side has a small cleaning curette, the other side a curved sickle scaler for removing necrotic tissue and slight residue from curettage.

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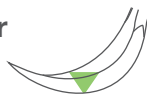
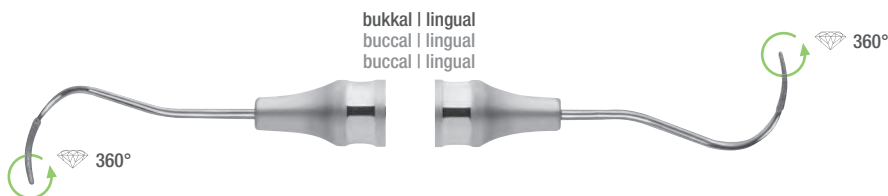
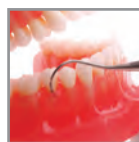
The diamond-coated scaler-files allow gentle and safe scaling of all furcations and root surfaces.

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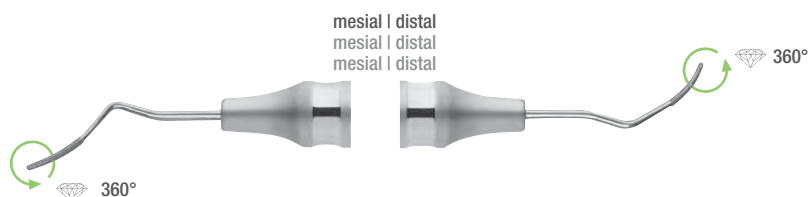
In addition to the classic models 1-2, 3-4 and 5-6, the introduction of the Langer curette 17-18 to the range complets the selection and offers odontologists a diverse product range in the area of curettage.

Langer curettes combine the shank design of the Gracey curettes with the 90° cutting edge of the universal curettes, sharpened on both sides.

Langer curette 1-2 is designed for the treatment of lower bicuspid and molars, the Langer curette 3-4 designed for upper bicuspid and molars and the Langer curette 5-6 is designed for upper and lower front teeth. The model 17-18 is especially adapted for all surfaces in the molar area of upper and lower jaw, especially for smaller mouth openings.

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Mesial-distal curette: # 11-14 and # 12-13
The shanks are angled differently on both working ends. The instrument does not have to be changed to effectively clean all mesial and distal surfaces of the bicuspid and molars.

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The terminal shank of the Gracey curette „Large“ is 3 mm longer in comparison to the Gracey curette „Standard“.

This modification allows for better access to the crown region. The extension of the shank is also optimal for the removal of calculus and for curettage in periodontal pockets with a minimum depth of 5 mm.

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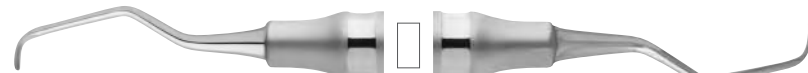
Mesial-distal curette: # 11-14 and # 12-13
The shanks are angled differently on both working ends. The instrument does not to be changed to effectively clean all mesial and distal surfaces of the bicuspid and molars.

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The terminal shank of the Gracey curette „Small“ is 3 mm longer and the work endpieces are 50% shorter.

This modification allows better access to the narrow root surfaces. A reduction in the widening of the gingival tissue when inserting the instrument is also an advantage of the shortened blade. Perfect for the front tooth region and for better access to all mesial and distal surfaces.

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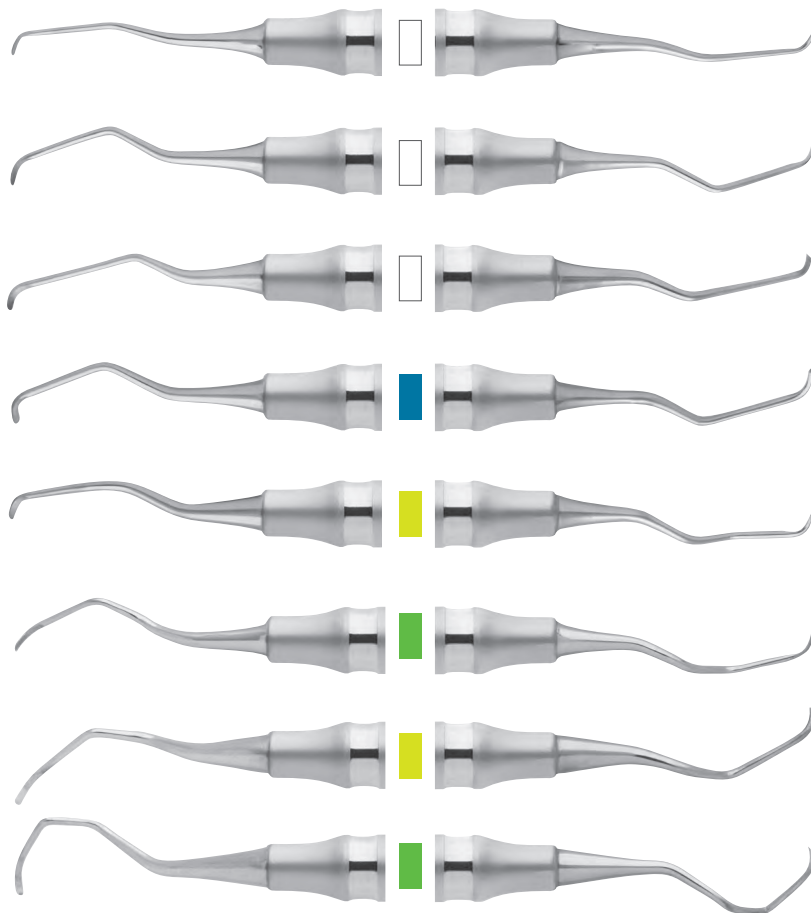
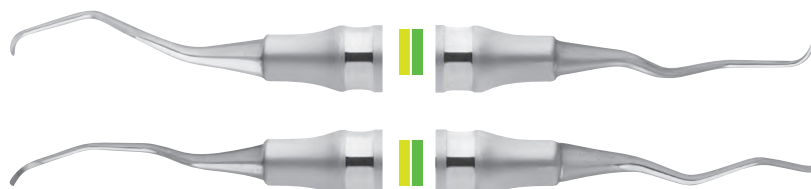
Mesial-distal curette: # 11-14 und # 12-13
The shanks are angled differently on both working ends. The instrument does not to be changed to effectively clean all mesial and distal surfaces of the bicuspids and molars.

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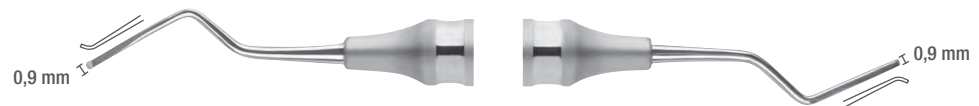
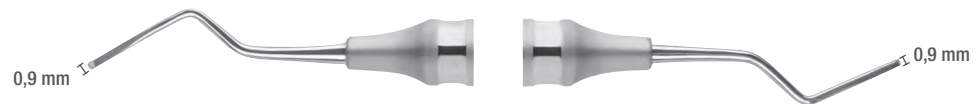
Used for scaling root concavities and furcations

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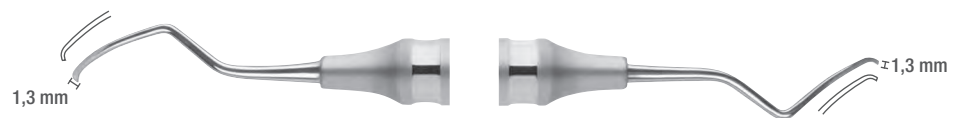
Hold the instrument with the working part face down and place the skewer on the facial surface. Due to the magnet, the plastic rods align automatically. The grinding angle can be read.

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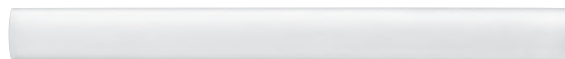
Grind the curettes after applying oil to the stone. Wipe the stone clean with a dry, clean cloth. The sanding dust remains in the oil and the stone is clean again. If you grind your curettes when dry, or by using water, then the dust penetrates the stone. The grinding stone cannot be thoroughly cleaned and therefore after a period of time must be disposed of.

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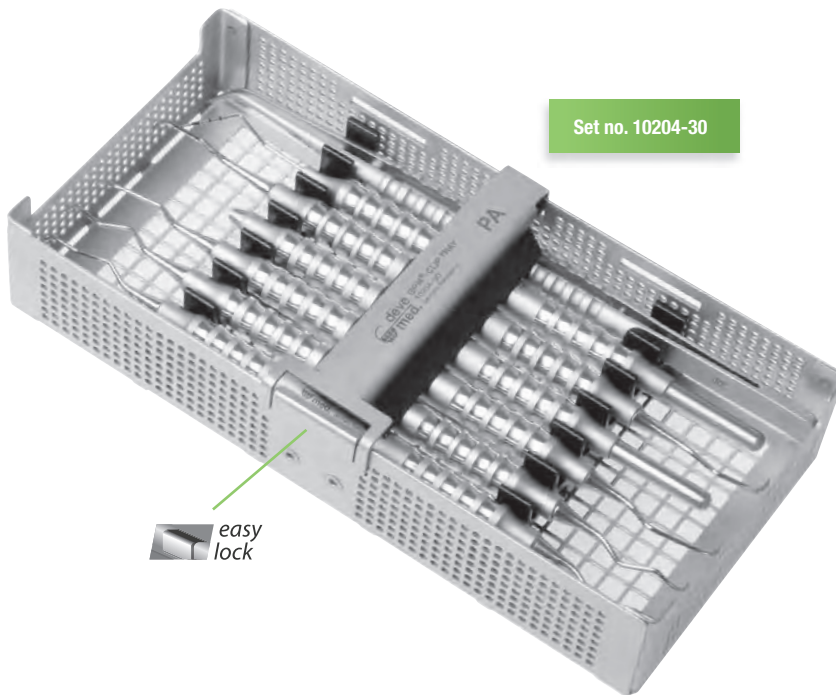
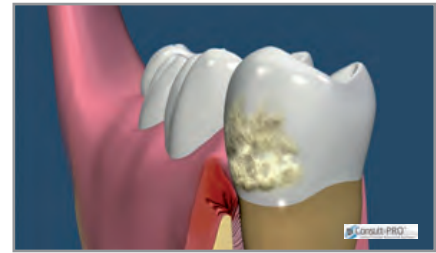
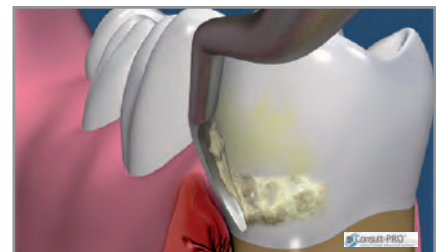
Due to insufficient oral hygiene, plaque and calculus deposits form on the teeth. These accretions can cause inflammation as well as swollen and bloody gingival.

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Several different curettes and scalers can be used to remove these sub- and supra-gingival accretions.

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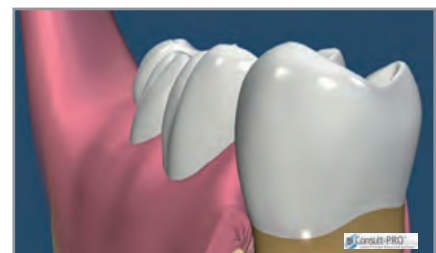
The teeth are smoothed with a polishing cone, so that new plaque cannot settle.

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As long as the teeth can be kept clean and free from plaque, the soft tissue can regenerate and its pink colour will return.

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ENDODONTIE

ENDODONTIC
ENDODONTIE

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Probes | single ended
Sondes | simples

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Exkavatoren | doppelendig

Excavators | double ended
Excavateurs | doubles

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The slender tips of the forceps allows a grasping of small root/bone splinters, fractured endodontic instruments, etc.

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Suitable for gripping round or square objects of any kind. For example, root canal instruments, crowns or rotary instruments and for the safe removal of covers (e.g. from syringes, canulas, etc.).

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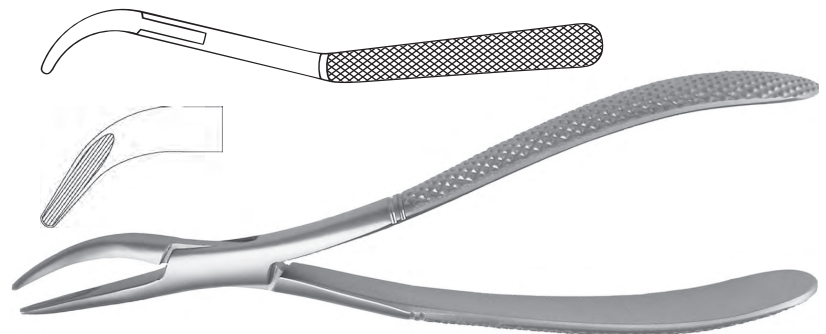
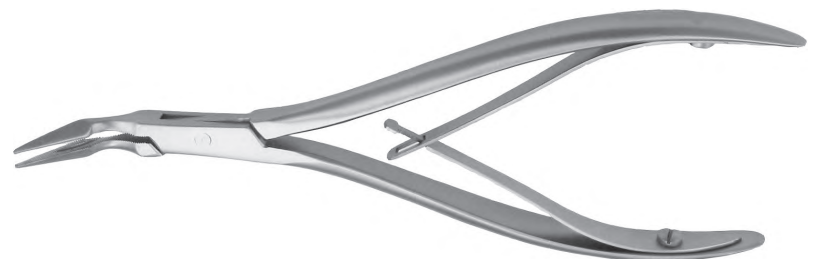
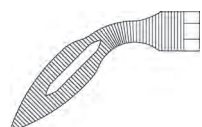
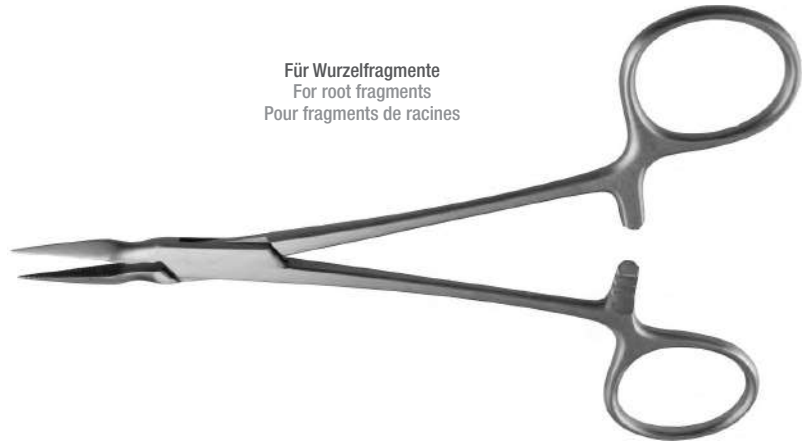
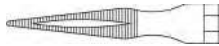
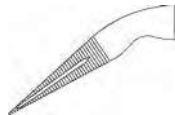
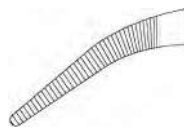
dev-lux Instrumente

 dev-lux instruments
 instruments dev-lux

saving hard and soft tissue
modern tooth extraction
tactile working
direct and controlled power transfer

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With the dev-lux-range you get a modern set of instruments for dental extraction, appropriate for the principle minimal invasiveness.

Increasingly today, the focus in modern dentistry is on implantology. Therefore it is necessary to conserve the hard and soft tissue during extraction. The dev-lux-instruments have been developed with this in mind.

The new, handle design **F-line ROUND SOLID**, aligned to pressure and tug, allows tactile working without destroying the adjacent structures. Direct and controlled power transfer is possible with the new handle.

This is the perfect instrument for the careful and faultless removal of non-preservable teeth.

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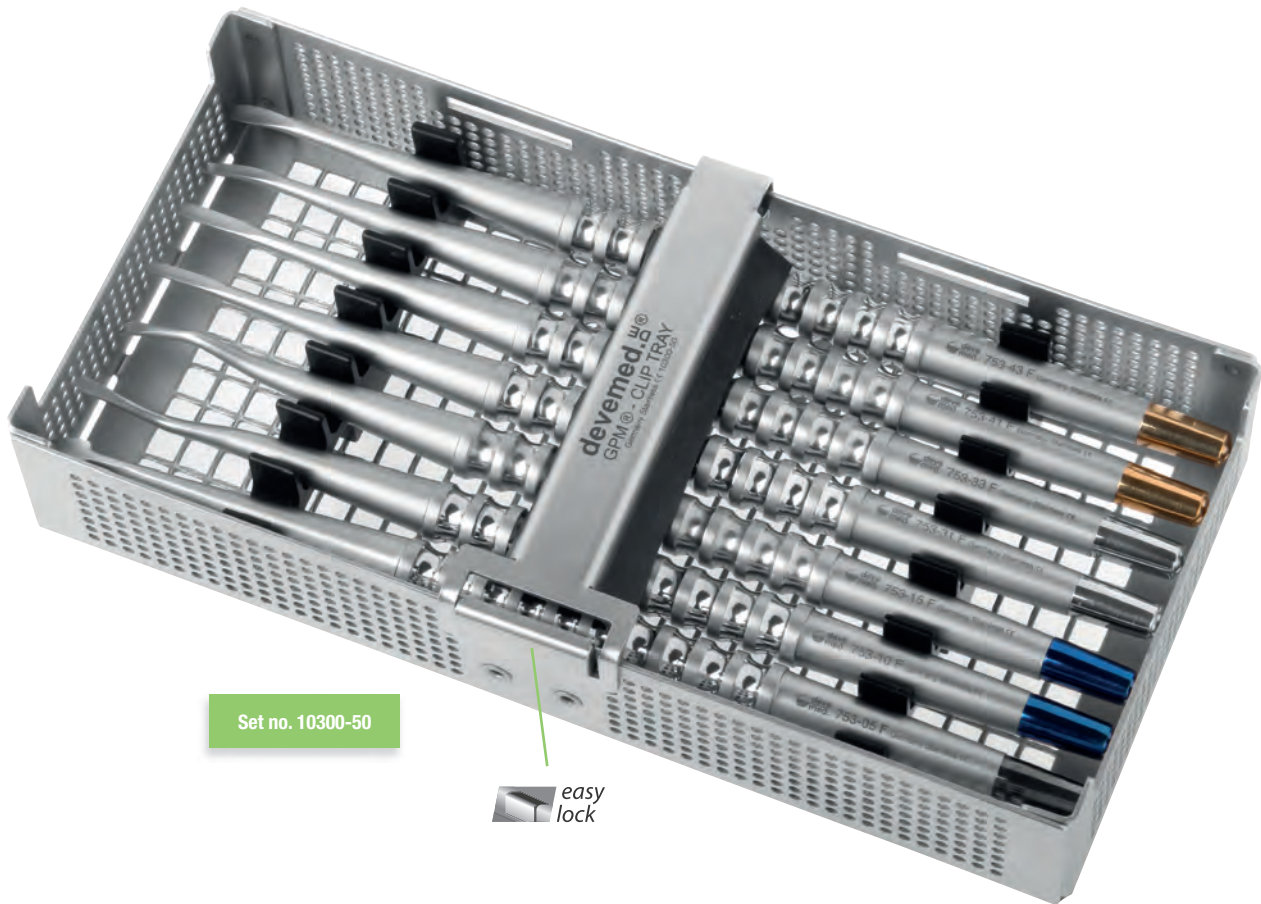
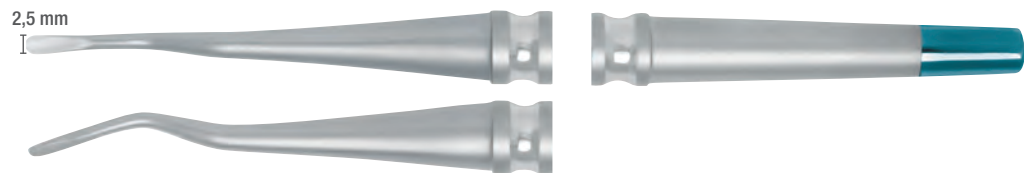
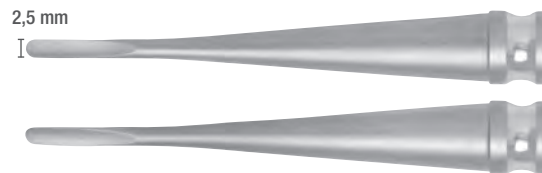
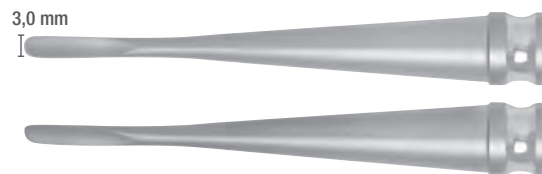
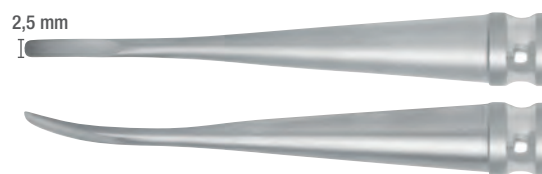
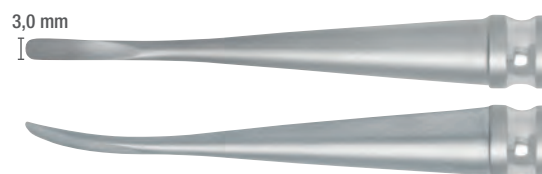
dev-lux instruments should be sharpened regularly (after 5-6 applications). Particularly suitable: abrasive 9500-60 and 9500-30 (Pages C.16)

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751-00
14,0 cm

Today's modern implantology demands that immediate implantation takes place after gentle and atraumatic tooth extraction. Therefore it is increasingly important that during the tooth extraction the hard and soft tissue structures are already handled very gently and atraumatically. Periolux esare a further development of periotomes and thus enable unique mesial and distal manipulation of the root structures. This prevents fracturing of the buccal and labial bone lamella, which is extremely important for the implant insertion. The Periolux is an ideal complement to the conventional extracting instruments for the gentle extraction of non-restorable teeth:

- Light handle design for fatigue-free working
- The handle size allows for careful gentle and controlled operation as well as direct power transmission to avoid tooth and root fractures
- Atraumatic removal of teeth without damaging nearby structures

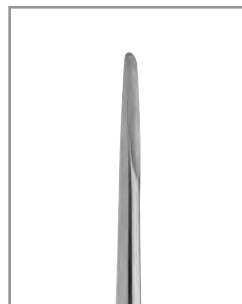
The alveole is atraumatically expanded by the special flat worktips, and the periodontal ligaments are gently severed by slight rotary motion.

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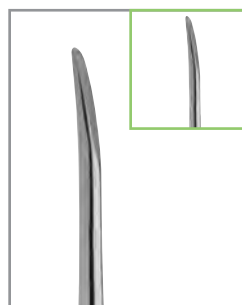
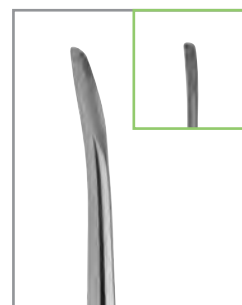
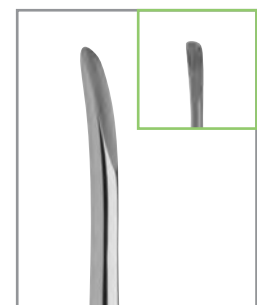
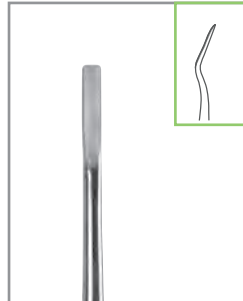
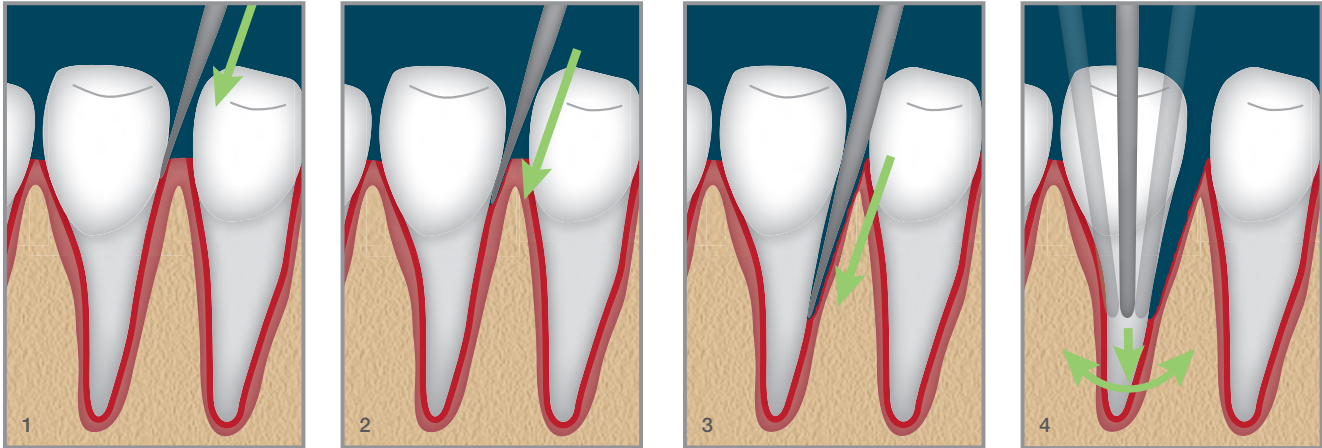
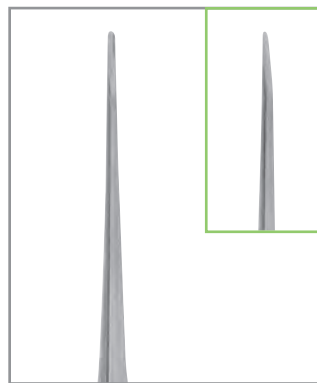
Position the Periolux to the side of the tooth (1), enlarge the alveole (2), sever the ligament (3) and remove the tooth with a rotating circular motion from its socket (4). This results in minimal tissue damage.

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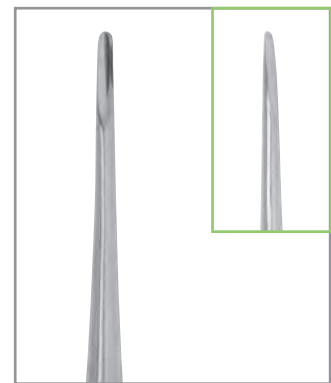
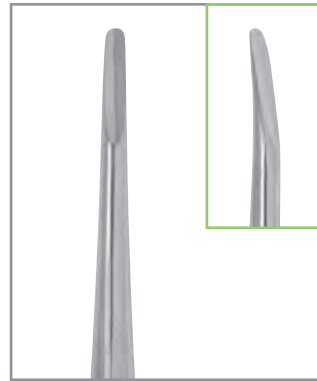
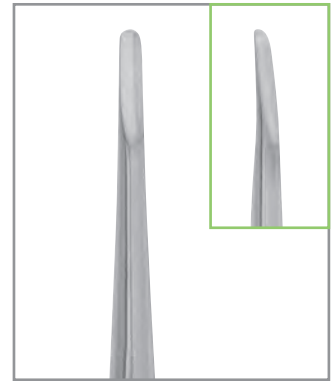
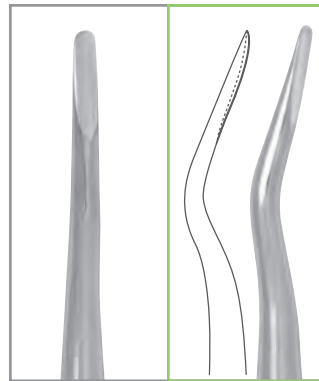
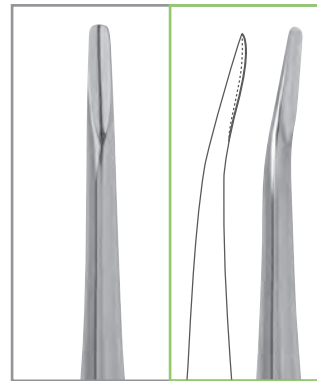
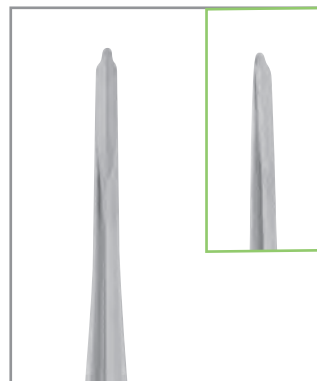
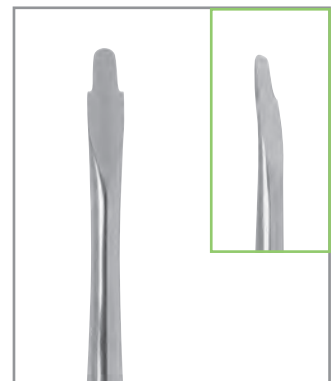
Periotome | einendig

Periotomes | single ended
Périotomes | simples

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The periotomes are ideally adapted for the careful detachment of the periodontium. The resulting minimal damage to the surrounding alveolar bone leads to a gentle extraction of the tooth.

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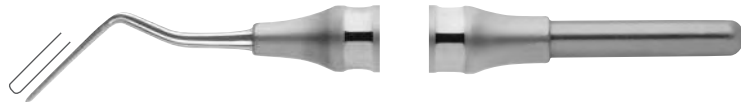
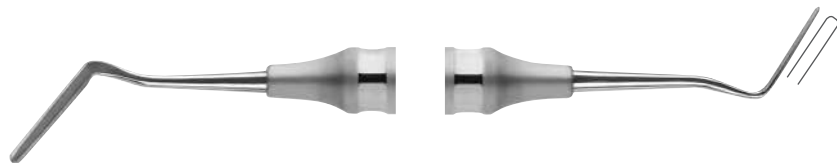
Periotome | doppelendig

Periotomes | double ended
Périotomes | doubles

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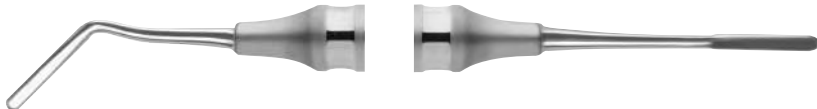
Syndesmotomes
Sindesmotomes

796-1
Chompret 16,5 cm
Hohlgriff
Hollow handle
Poignée creuse

796-1 M
Griff massiv
Handle solid
manche massif 16,0 cm

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INSTRUCTION FOR USE

Syndesmotomes are intended to be used for the transection of ligaments and for the removal of fractured roots or root fragments. Under no circumstances they may be used as root elevators. In case of violation, deve med GmbH assumes no liability.

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796-7 S
Griff massiv
Handle solid
manche massif 15,5 cm

796-8
Griff massiv
Handle solid
manche massif 15,5 cm

796-7
Griff massiv
Handle solid
manche massif 17,0 cm

796-7 A
Griff massiv
Handle solid
manche massif 17,0 cm

796-17
Griff massiv, gezahnt
Handle solid, serrated
Manche massif, denteles 17,0 cm

796-17 A
gezahnt
serrated
denteles 17,0 cm

Pivot-Point Instrumente

Pivot-Point Instruments
 Instruments Pivot-Point

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Due to the elliptically ground area on the instrument tip and the diamond-shaped plateau, gentle penetration into the interdental space or periodontal cleft is optimally guaranteed.

Through its ability to rotate 180° and its shape, four luxation movements are possible: 2x mesial and 2x distal. During luxation, the instrument operates with between 5-7 hypomochlions (rotation and supporting actions), which is vastly superior to any conventional elevator.

The grooves on the edge of the instrument provide especially efficient luxation. The instrument tips are not designed for axial leverage as this could lead to fracture.

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F-line PH

Pivot-Point Instrumente

Pivot-Point Instruments
Instruments Pivot-Point

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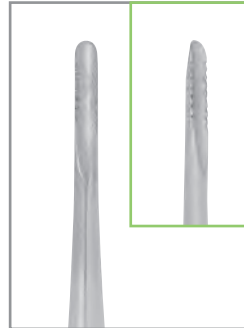
The Pivot-Point Instruments are not suitable for use as axial levers. Disregard of the usage instructions (Abb. 1-4) may result in breakage. We can accept no liability for this damage!

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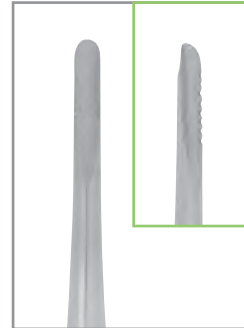
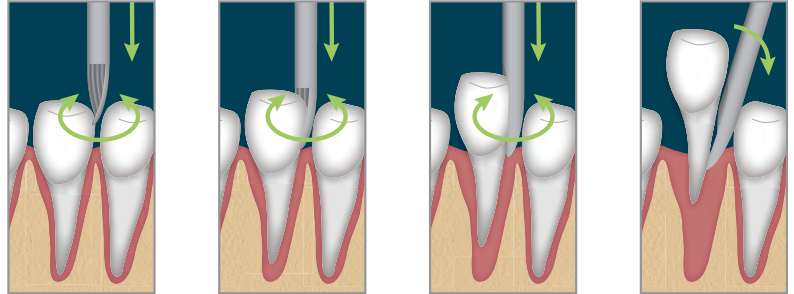
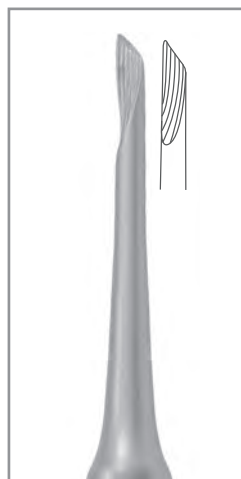
Bernard root elevators are particularly suitable for the excision of fractured roots or root fragments. They represent a combination of root tip elevators and periolutes. The inner side is concave and the lateral edges are for cutting. This allows for rotary movements, and when applying slight pressure, the ligaments can be separated and the remaining root fragments can be loosened.

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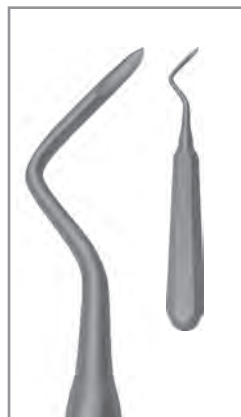
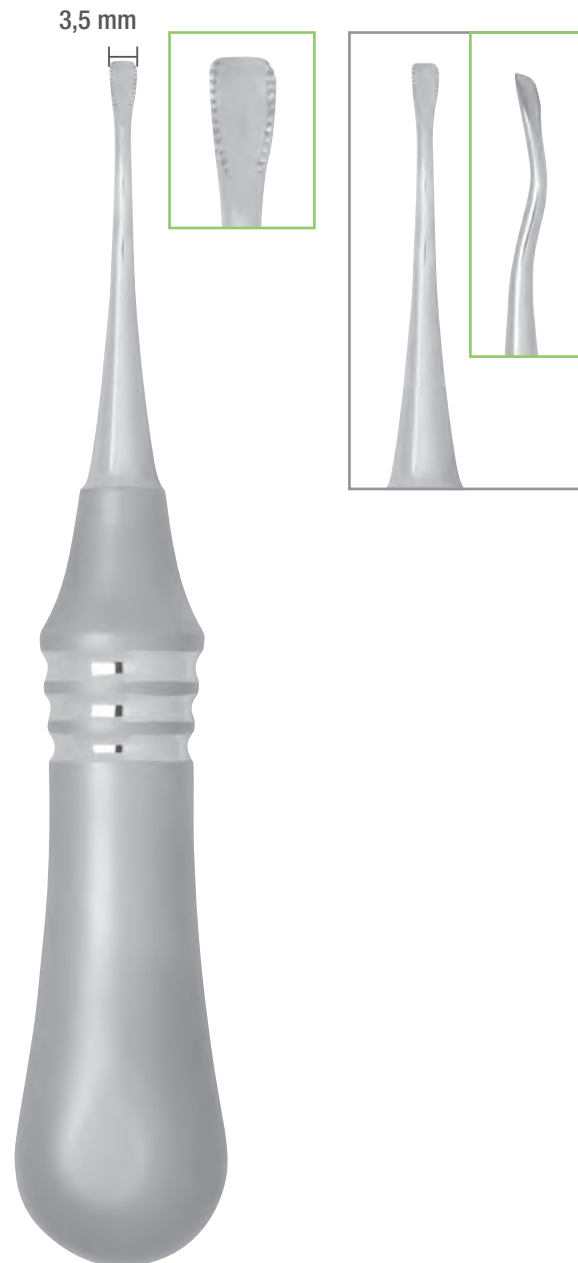
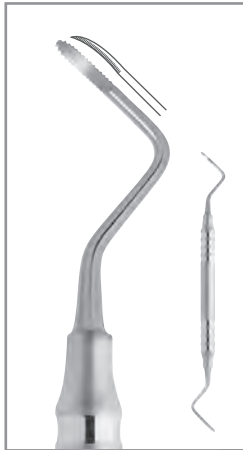
GES LINE | GENTLE EXTRACT SYSTEMGES LINE | GENTLE EXTRACT SYSTEM
GES LINE | GENTLE EXTRACT SYSTEM**gentle extraction****beaks run parallel to the tooth****smooth handle prevents sediments****constant power transfer****good grip at the tooth****minimizes annoying light reflections**

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A new series of extracting forceps has been created with the GES LINE. Due to the special adaption of the jaws onto the tooth, only a minimal pressure must be applied during use. The jaws run parallel to the neck of tooth and enable a gentle extraction of the teeth.

The newly developed handle provides constant power transmission for circular motion as well as for push and pull motions. The new **F-line** PLAIN SURFACE with its smooth handle prevents sedimentation and is therefore easy to clean.

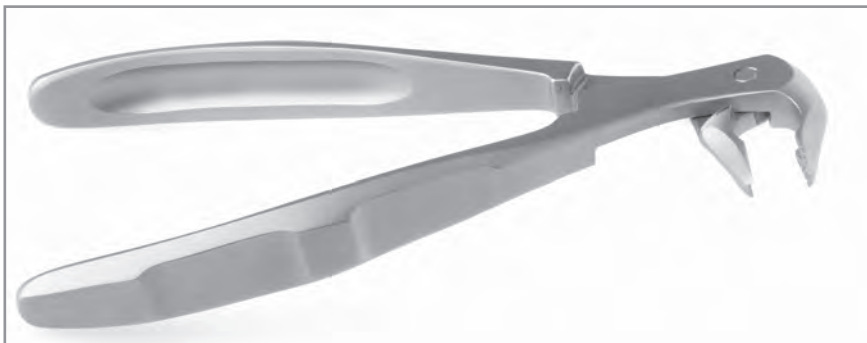
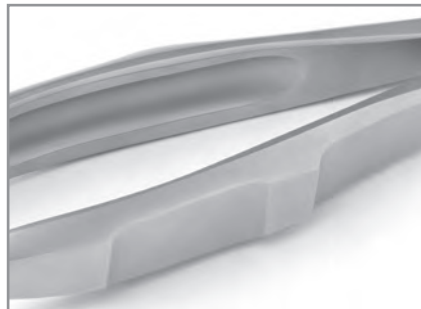
The sand-blasted surface minimizes annoying light reflection when working.

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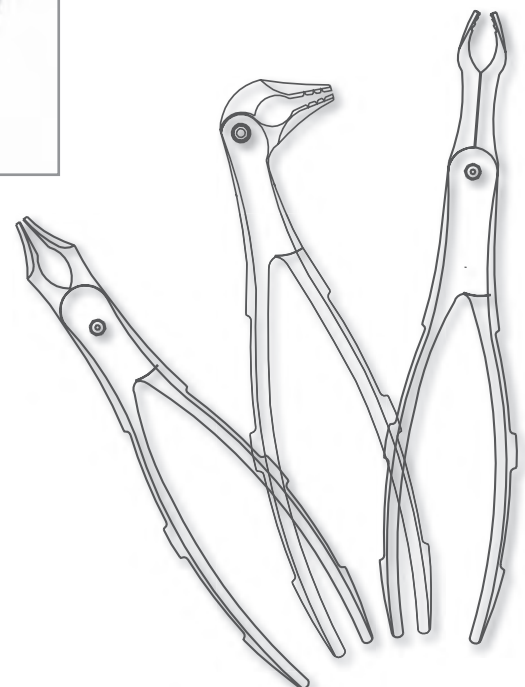
The jaws are perfectly suited to the tooth form.

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GES LINE | GENTLE EXTRACT SYSTEM

GES LINE | GENTLE EXTRACT SYSTEM
GES LINE | GENTLE EXTRACT SYSTEM

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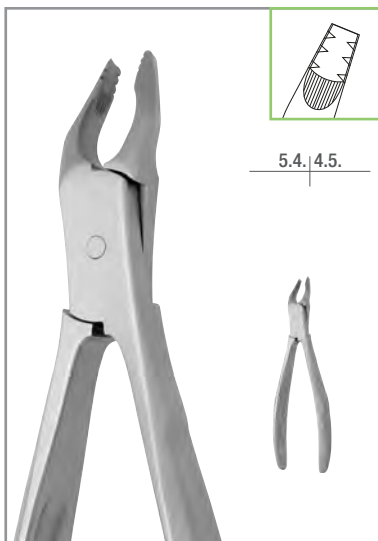
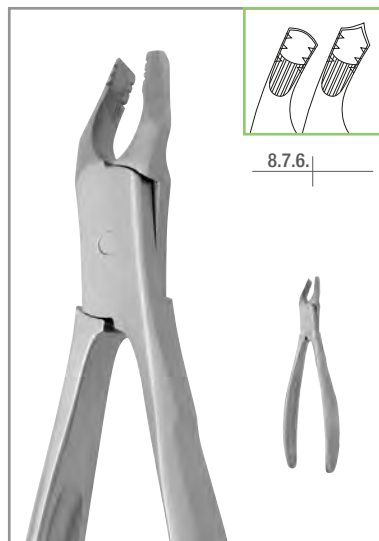
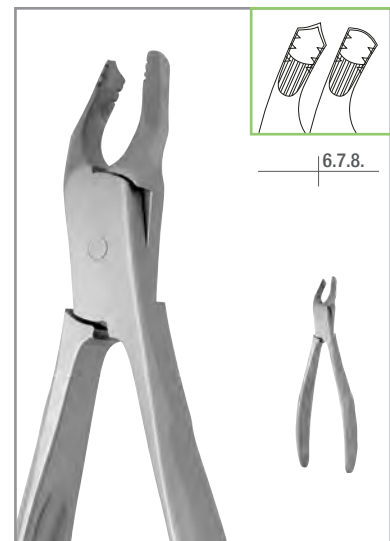
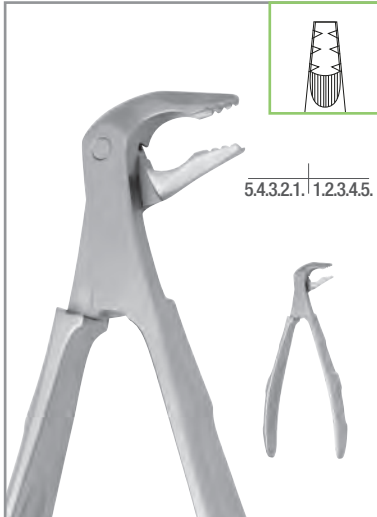
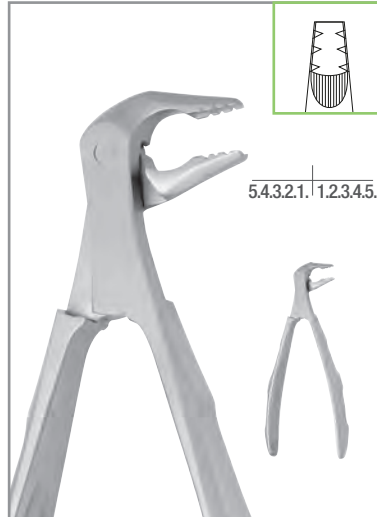
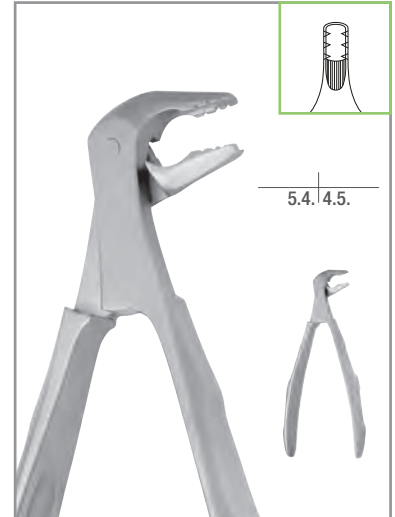
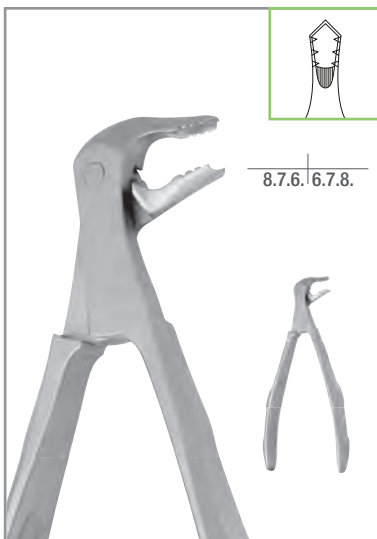
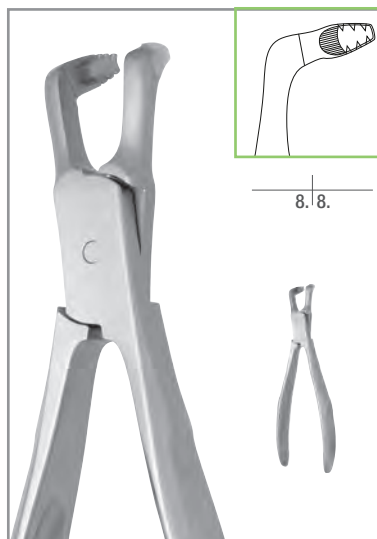
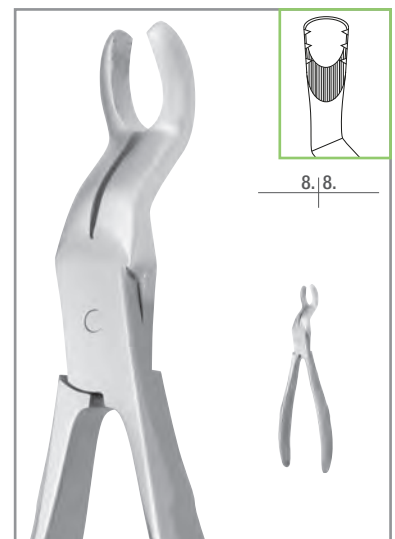
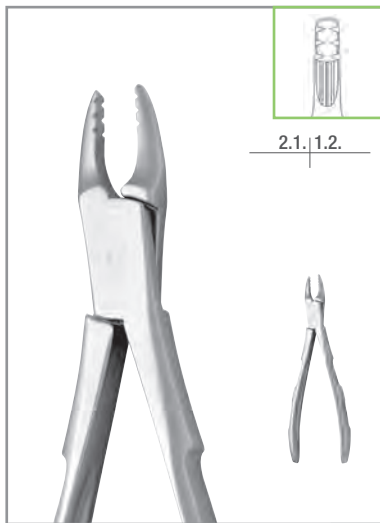
KIDS GES LINE | GENTLE EXTRACT SYSTEM

KIDS GES LINE | GENTLE EXTRACT SYSTEM

KIDS GES LINE | GENTLE EXTRACT SYSTEM

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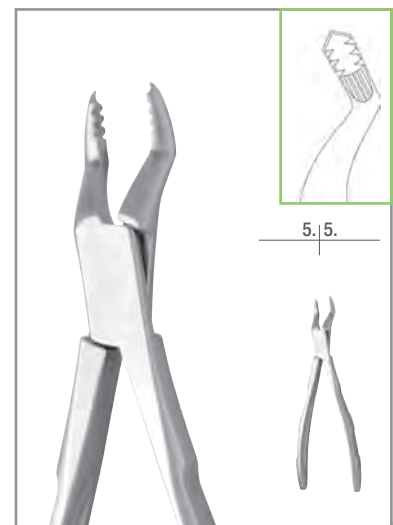
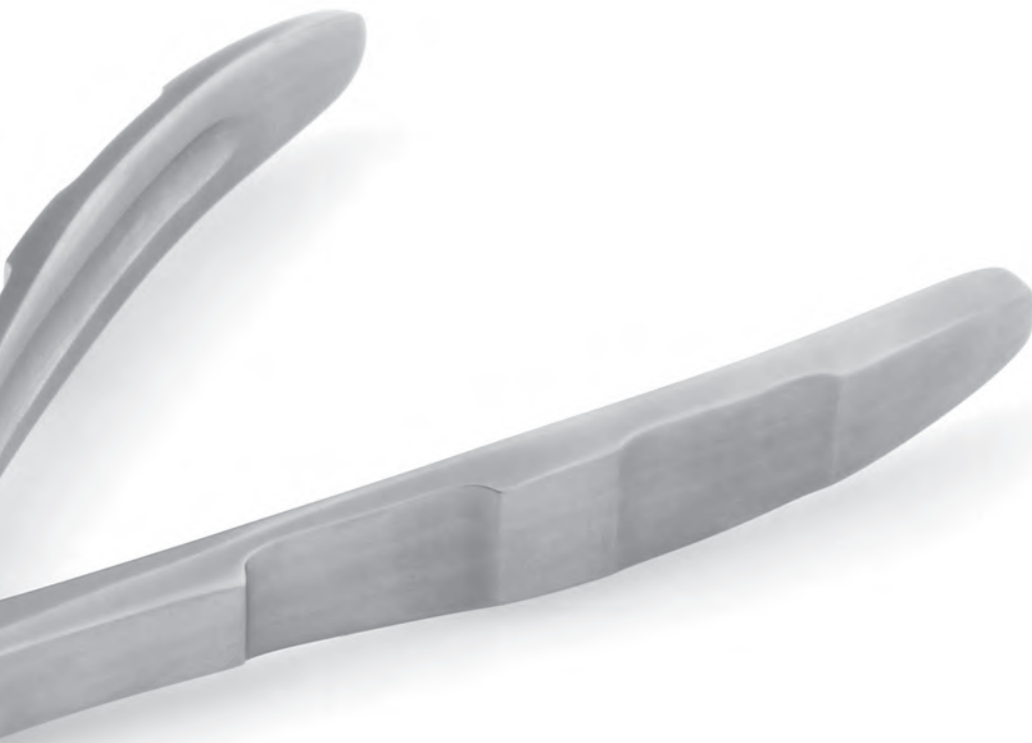
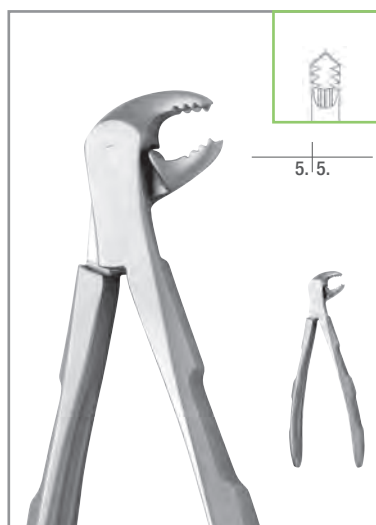
Minimal-traumatic axial tooth extraction with ZALEX®

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During and after the tooth extraction, there are frequently complications. They are correlated with the burdens in the tooth and the surrounding tissues during the luxation and contribute to tissue loss. ZALEX® is a extraction system that optimally uses the force acting in the longitudinal axis of the tooth, so that minimally luxation and rotation movements are necessary. Methodically and instrumentally ZALEX® follows on seamlessly after the actual tooth extraction, allowing easy acceptance into the practice process.

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Connection between holding forceps (item no. 310-73) on the right and lifting forceps (item no. 300-00) on the left for axial extraction of tooth 37.

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The holding forceps (item no. 310-73) is applied on tooth 37. The lifting forceps (item no. 300-00) lies with absorbed support (item no. 300-90) on teeth 36, 35, 34 and 33.

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The lifting part of the lifting forceps (item no. 300-00) is connected with the peak of the working part of the holding forceps (item no. 310-73).

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The working parts of the lifting forceps (item no. 300-00) are splayed; tooth 37 is axially extracted.

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Significant axial extraction progress.

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Result: minimal-traumatically extraction trauma after ZALEX-extraction

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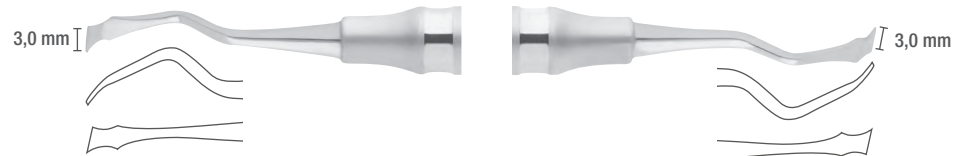
The chisel is recommended for atraumatic removal of bone material attached to the tooth and is particularly suited to the rear distal molars. (pull stroke)

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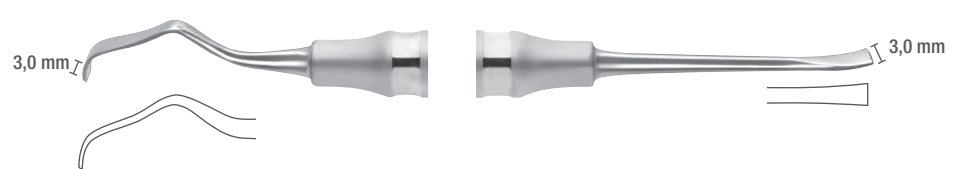
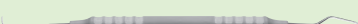
Implant Maintenance | Implantat-Prophylaxe

As with Gingivitis the same bacterial strains are caused by the peri-implantitis. Therefore, the same processes apply: the removal of dental plaque, surface cleaning and surface polishing.

Scientific studies have shown that cleaning the titanium abutments with the titanium instruments causes no change whatsoever to the surfaces. Therefore our implant prophylaxis instruments are manufactured with pure, soft titanium. The workpiece ends correspond with our conventional curettes.

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For the cleaning of implants, as well as the removal of calculus and plaque, without damaging the titanium abutments.

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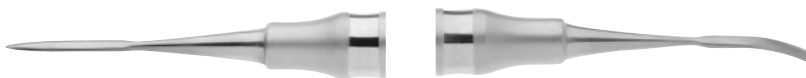
PPS I Raspatorium
PPS I Periosteal elevator
PPS I Rugines-décolleur

Starting at the Gingival mucosa, the attached mucous membrane can be surgically removed supra-periosteal with the slightly curved workpiece ends. The lightly rounded working part prevent perforation of the mucosal flap. In the mobile mucosa area, the minimally invasive surgical removal can continue with the straight, pointed workpiece ends.

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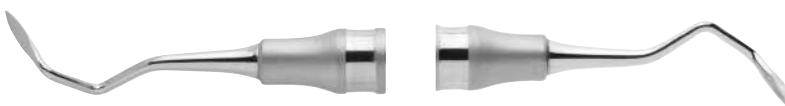
The workpiece ends are arrow-head shaped and angulated specifically to allow for lateral tunnel procedures in difficult to access approximal regions. The papillae are gently tunnelled by using the very flat workpieces.

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This instrument differs considerably in comparison to PPS 1200 F because of the curvature of the workpieces. With a bridge flap, this instrument can be optimally used to prepare tunnels from apical to coronal through an entry in the periodontal membrane.

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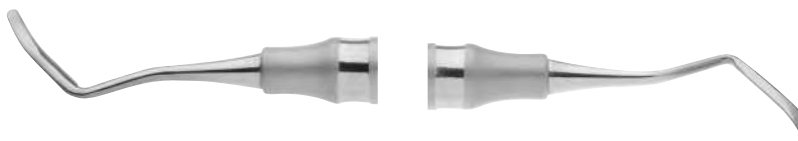
The mucosal flap preparer is particularly suitable in the posterior tooth area. Access and atraumatic mucosal flap preparation is possible by the lightly curved workpieces.

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The mucous membrane transplant can be inserted into the pre-prepared tunnel using this instrument.

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This instrument is an alternative to the PPS 1000 F and is used for the same application. The lineal, rounded work piece favours a more conservative working method.

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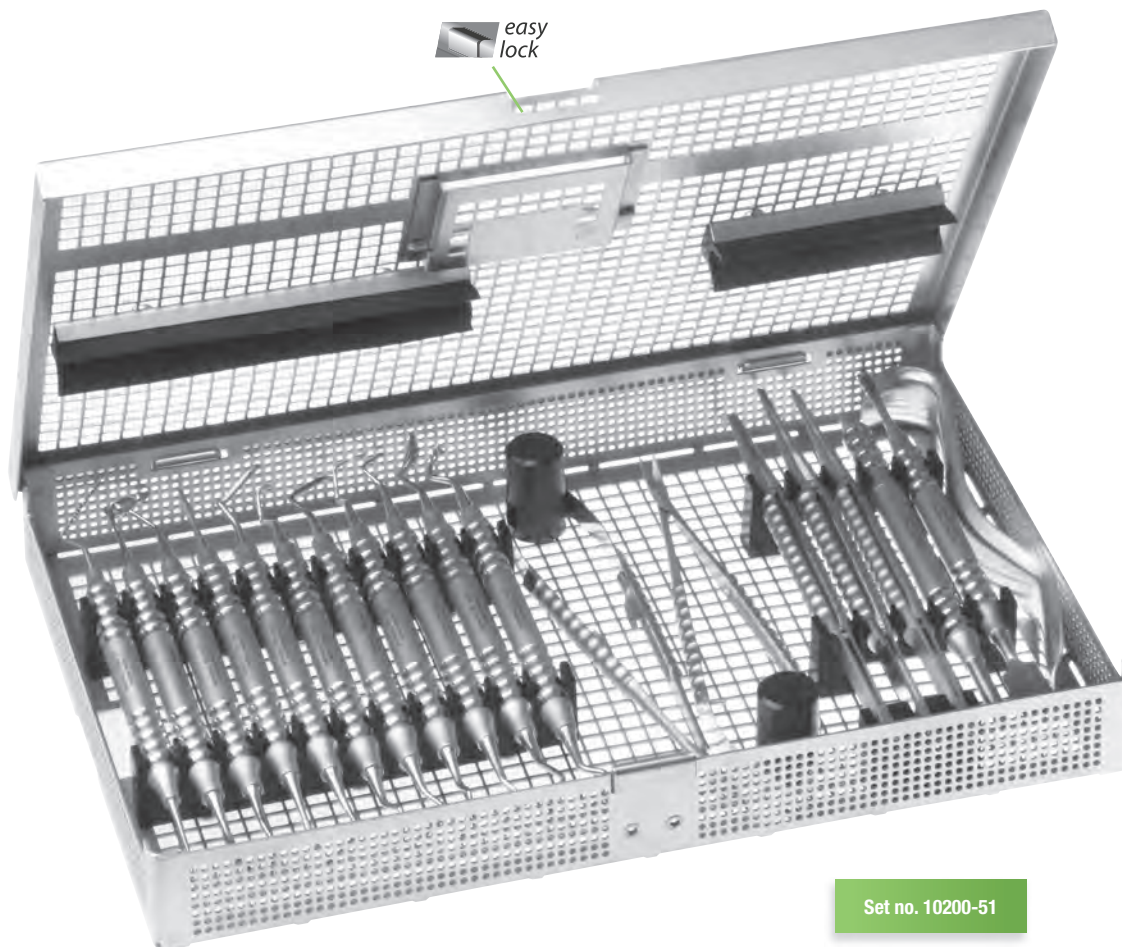
The familiar properties of an aesthetic smile include harmony, balance and continuity of form. Gingival degeneration in the aesthetic area is disturbance to these characteristics and must be corrected if an aesthetic smile shall be achieved. The plastic periodontal techniques are our tools to resolve such defects.

In order to improve the effective performance of the periodontal procedures in these difficult areas, particular micro surgical instruments and knives have been developed. These instruments have a special form which allow for better preparation around the root surface and are delicate enough to securely cut, even in restricted areas.

If standard instruments are used, it is often very difficult/impossible to effectively enhance the preparation or to lift the papilla without perforating the labial lobe. Lateral tunnel procedures are an approach which improve the aesthetics and their predictability, as they avoid loosening critical papillae and maintain high vascularization of the connective tissue graft.

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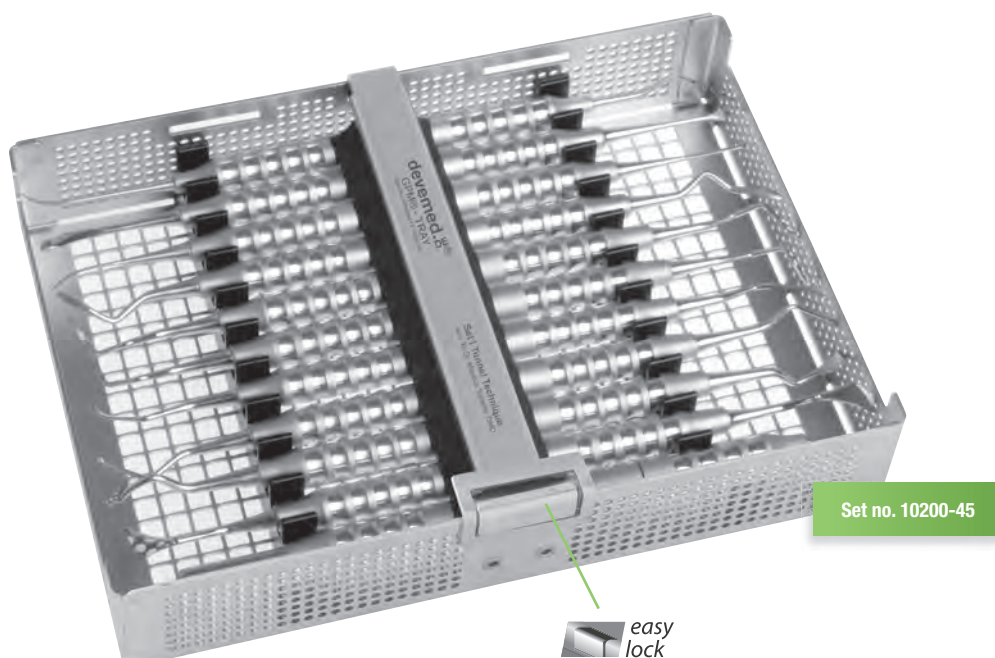
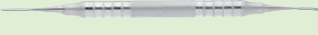
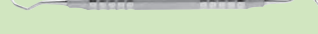
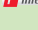
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
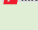
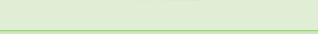
With regard to aesthetic appeal, low-risk rapid implant surgery and minimal tissue damage, the demands of patients are becoming more important. To satisfy these needs, a well-coordinated and high-quality range of instruments is required.

The basic surgery set provides a well thought efficient and tissue-protective choice of basic instruments. Perfect incision and suturing, as well as gentle flap preparation and pre-implant measuring can now be implemented.

We recommend this set to newcomers as well as to experienced implantologists, who would like to improve and optimize their surgical work.

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The 3D rotary head allows the blades to be turned in almost any direction, enabling you to position as required by the situation. Positioning problems caused by customary blade holders will be easily resolved.

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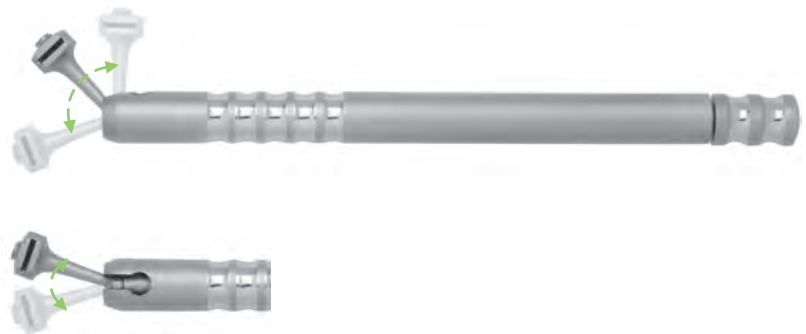
The opening with the screw cap allows a simple and safe insertion and removal of the standard scalpel blade (#3).

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The surface

To prevent distracting reflection, these scissors are offered in matt finishing.

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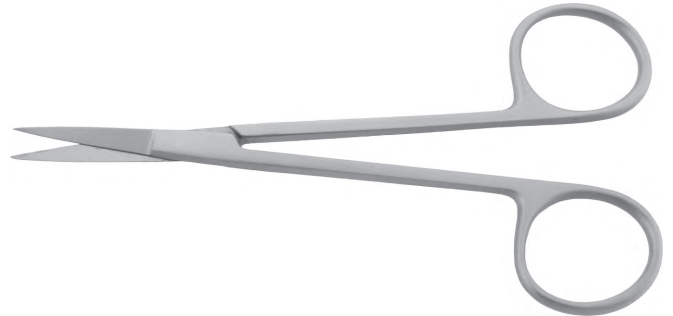
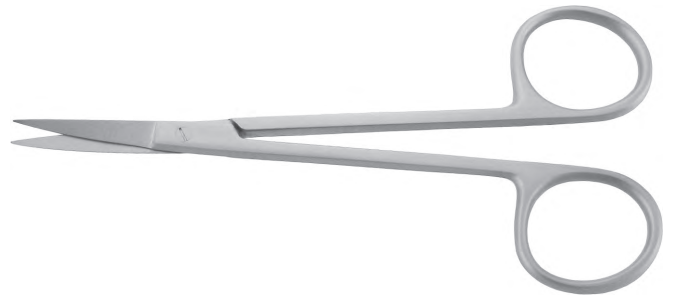
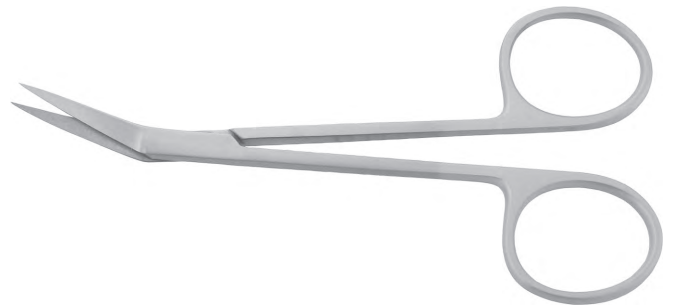
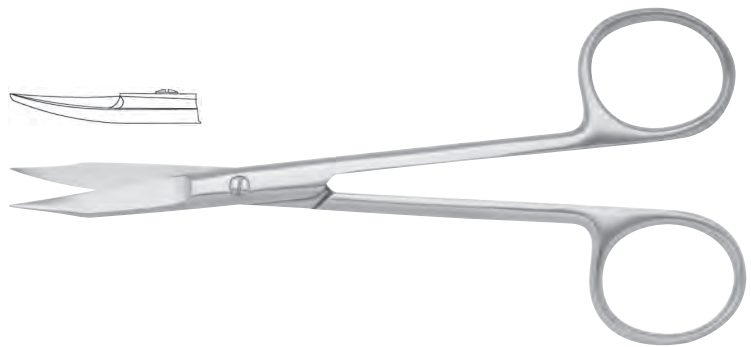
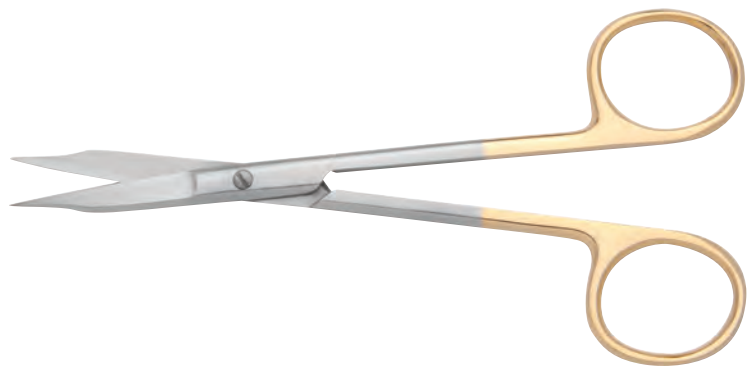
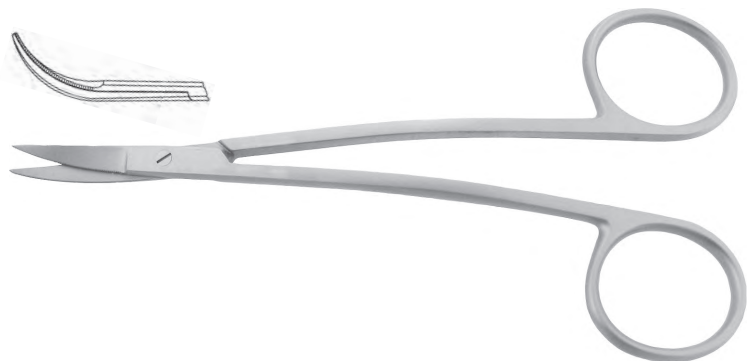
The curved gum scissors enable a diagonal incision in the distal area. The single sided micro serration makes lobe thinning possible.

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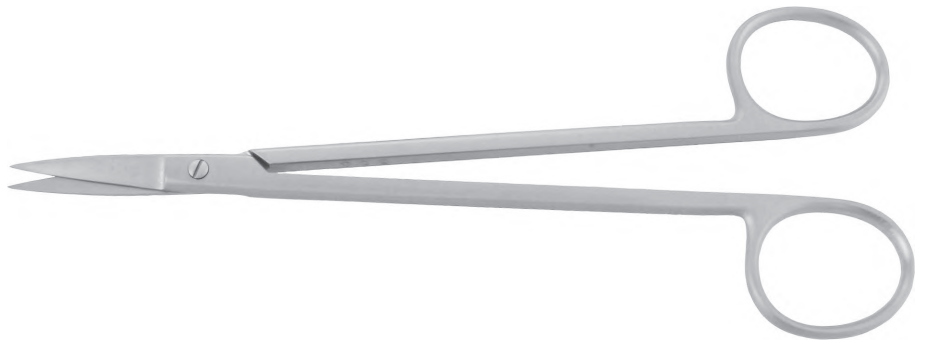
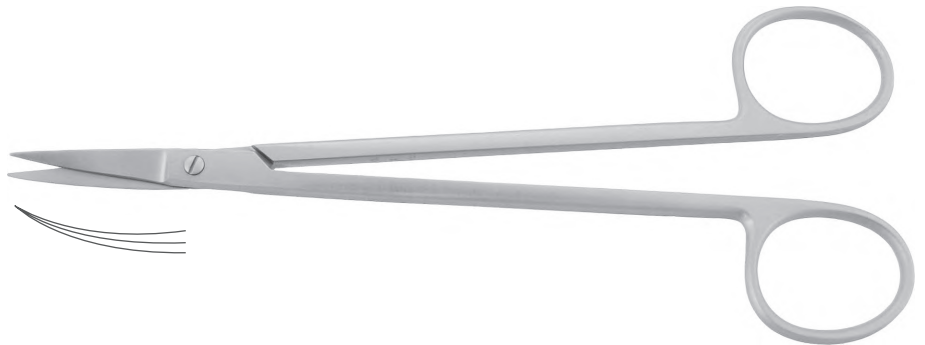
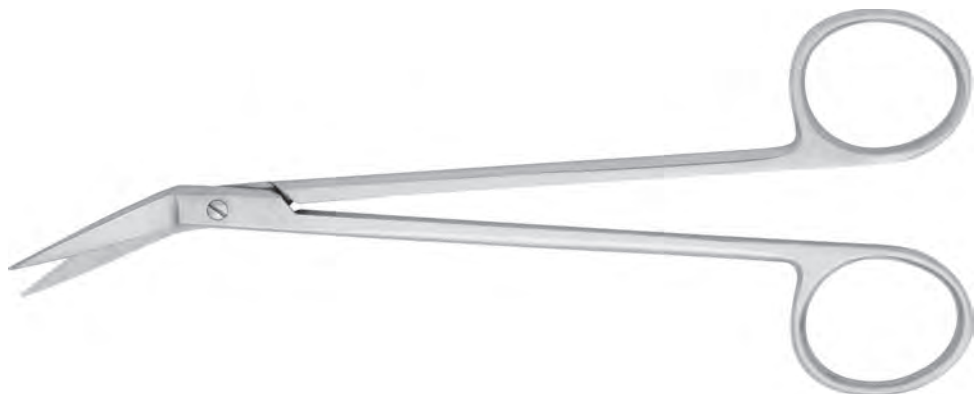
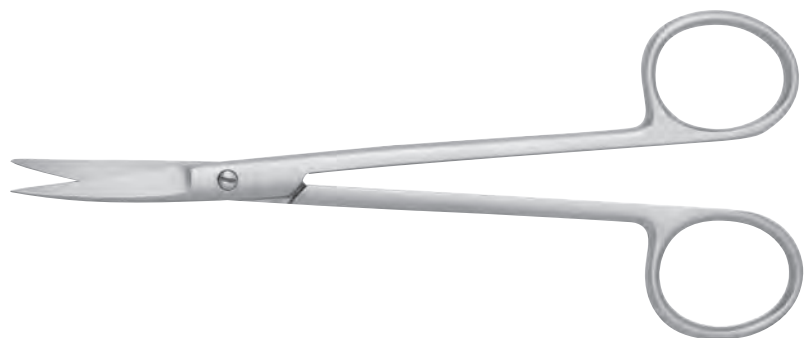
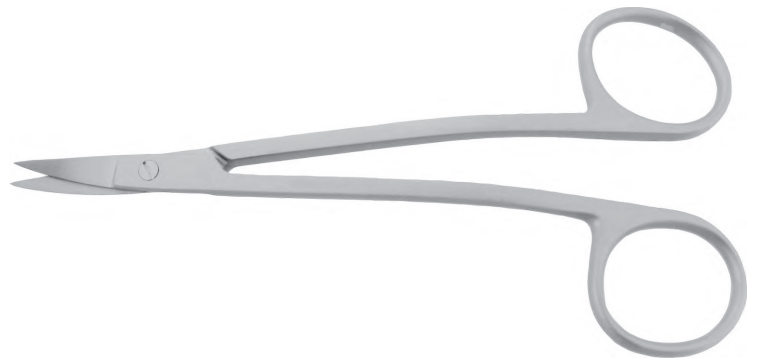
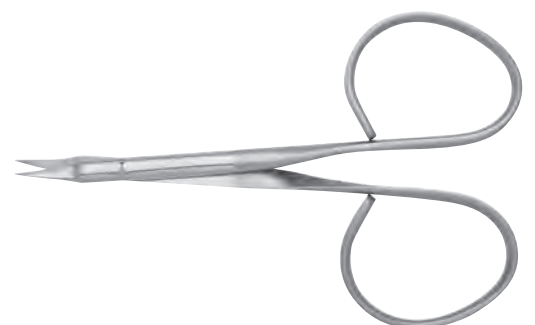
Scissors for minimalinvasive surgery

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These scissors have pointed working ends with a Super-Cut finish. They are curved and have been developed especially for minimal invasive surgery. As with the needle holders, the scissors also offer the possibility of a rotary movement. These scissors are ideally suited when working with magnifying glasses or a microscope.

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**Super-Cut**

The razor-sharp cutting edge cuts effortlessly and softly through thicker material. The serrated cutting edge on the second blade prevents slipping from tissue and sutures. Super-Cut is not practical for wires, cartilage and other hard materials. One handle ring is gold-plated to help with easier identification. Scissors in our F-line Design are marked by -Symbol.

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For the S-tec (S = Standard) series, standardized shapes and dimensions of the workpiece ends have been combined with our **F-line ROUND SOLID** handle.

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Our U-tec-series with the **F-line ROUND SOLID** handle has been developed for dentists who are already familiar in the field of minimal invasive surgery. The ultra delicate workpiece ends are proportionally based on fabric structure and material and allow a new dimension of selective working with magnifying glasses or microscopes.

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Our **F-line ROUND SOLID** handle, which allows a selected rotation, is an important aid by using a loupe or a microscope.

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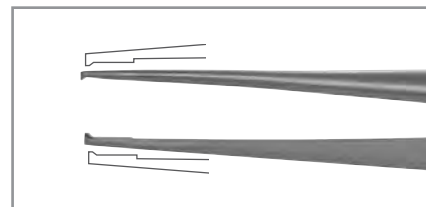
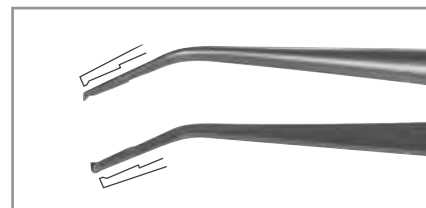
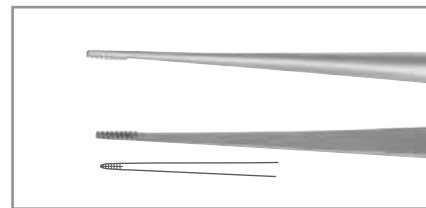
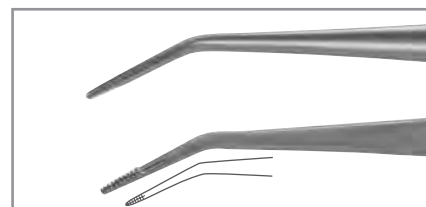
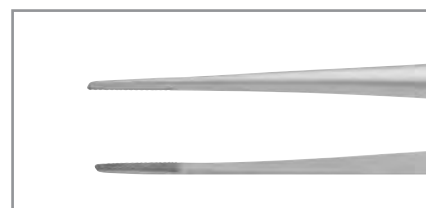
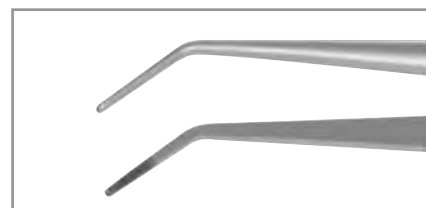
The especially curved tweezers enables optimal functioning in the retro-molar area. The extraordinary bending provides an unprecedented view in functioning the working area.

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Suitable for gripping any kind of round or square objects. For example, root canal instruments, crowns or rotary instruments and for the safe removal of covers (e.g. from syringes, canulas).

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These suture tweezer allows an optimal insertion of regeneration membrane as the gingiva can be sutured simultaneously.

While the lobe is being held, the needle can be exactly guided through the aperture. This precision work largely prevents any tearing.

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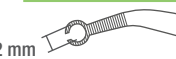
This forceps serves perfect for fixation and suturing. The shaping of the double tweezers allows a safe reposition of the soft tissue graft. The circular tips avoid tearing of the tissue.

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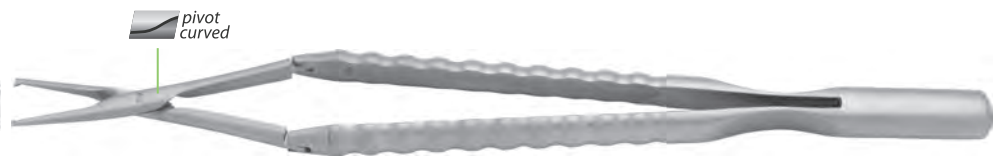
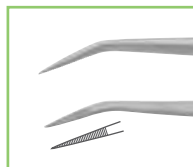
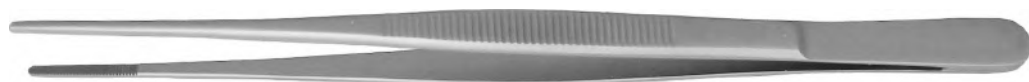
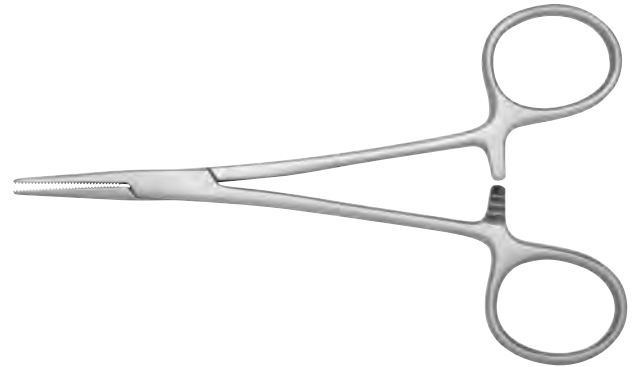
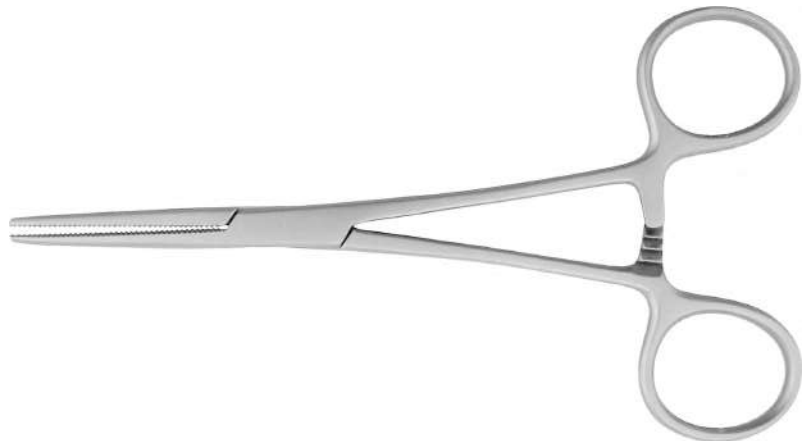
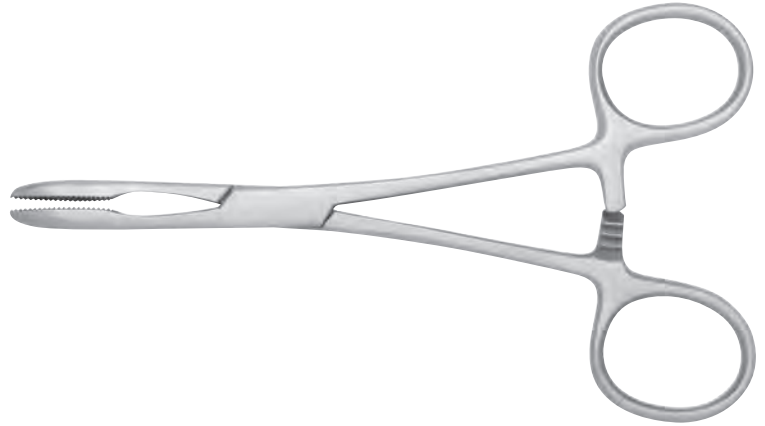
Retractors, cheek and lip retractors

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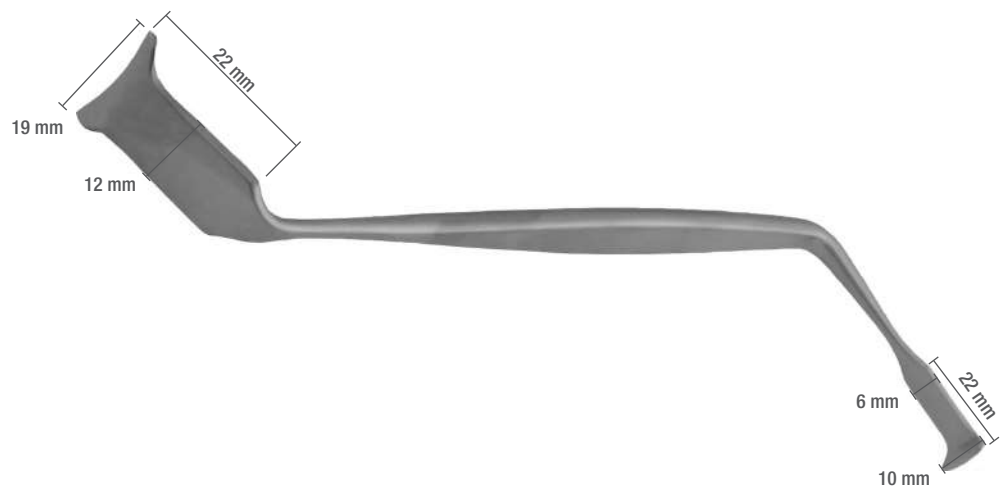
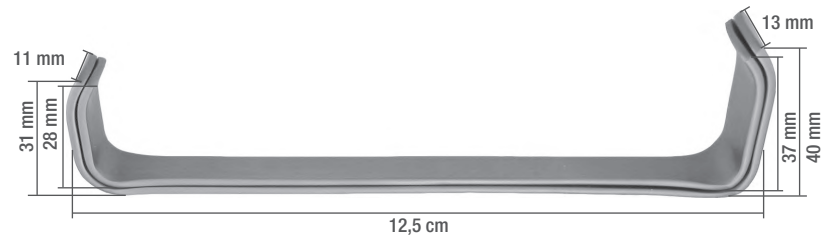
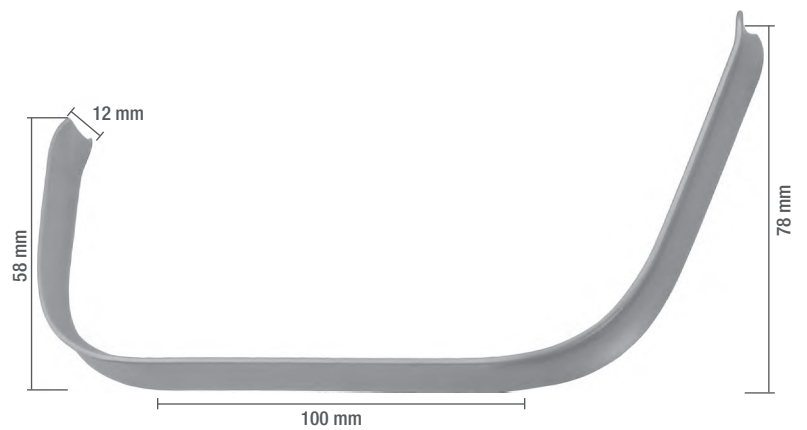
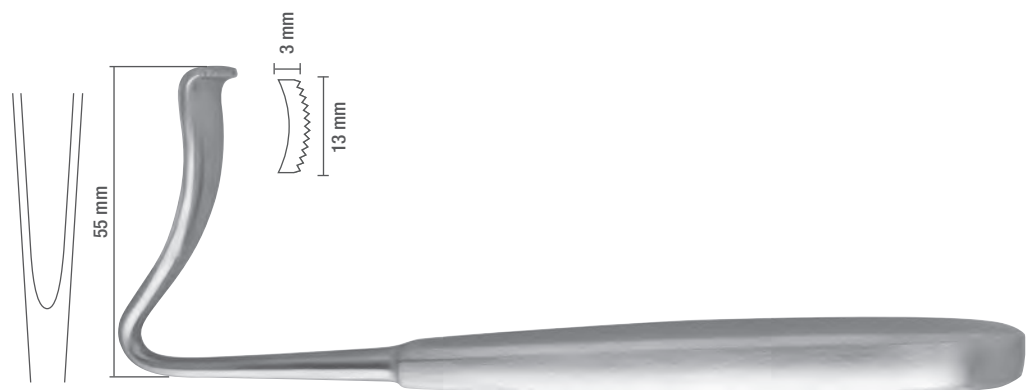
This light weight cheek retractor was developed for areas which are difficult to reach. It is often used in open curettage.

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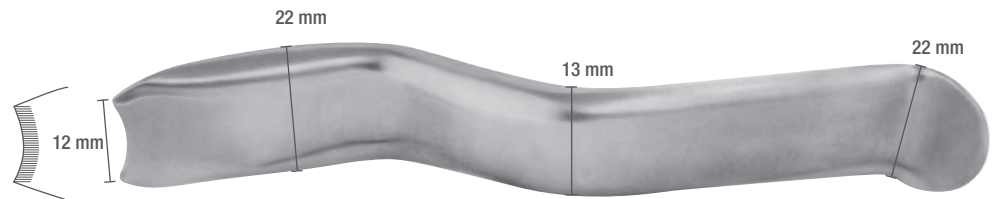
serrated

dentelés

new**998-21**University of Cawood-Minnesota
modified acc. to Dr. Ziv Mazor

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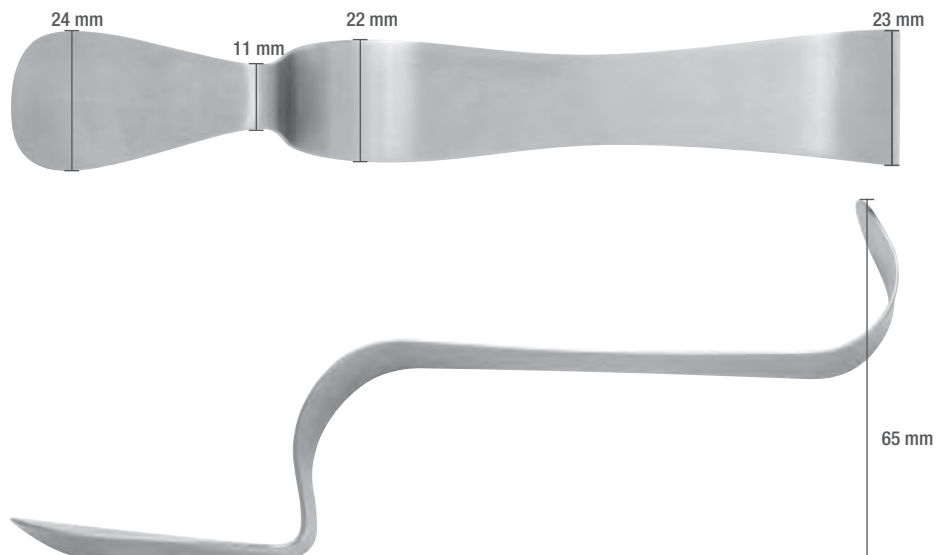
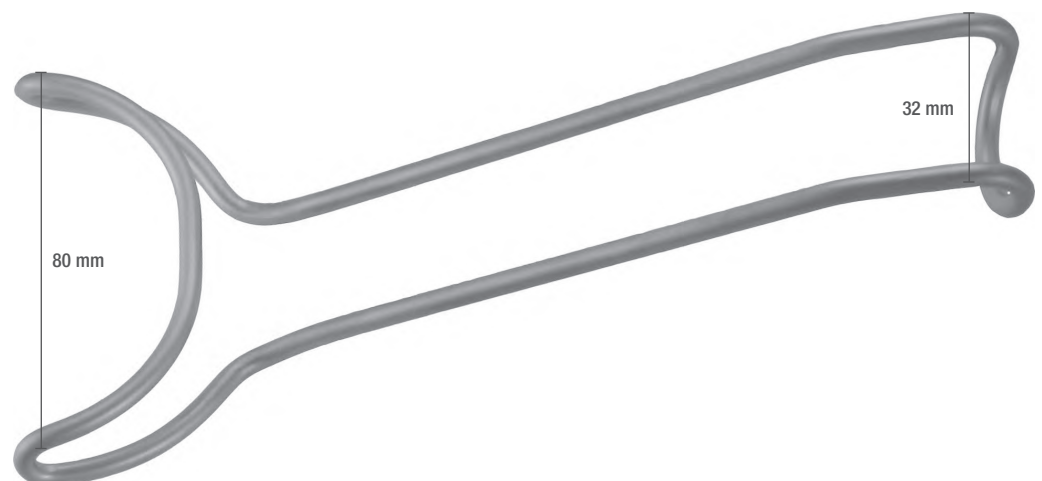
This lip retractor enables one to hold the whole lip in upper and lower jaw areas with only one hand. A clear view of the operation field is ensured. Therefore, this lip retractor is perfectly adapted for photographic documentation.

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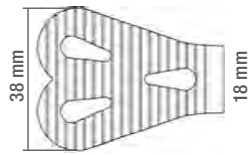
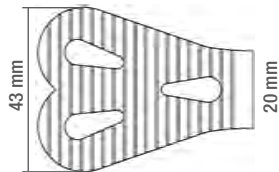
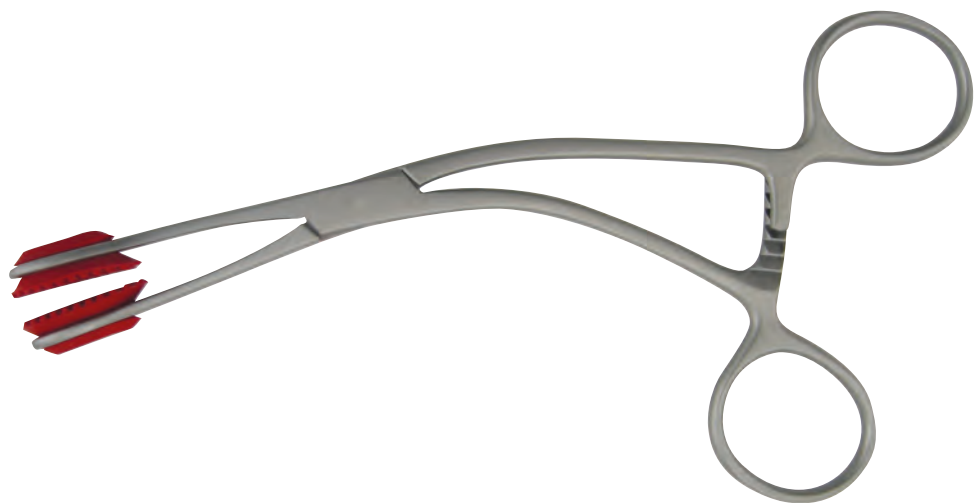
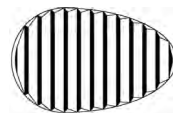
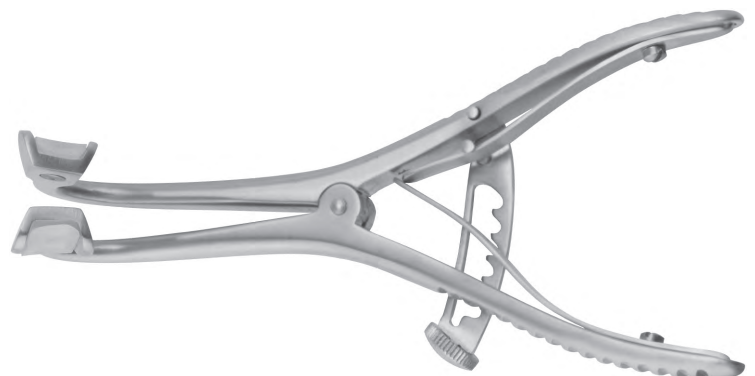
Three-In-One | Cheek and tongue retractor with saliva ejector

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**Three-In-One | Cheek and tongue retractor with saliva ejector**

Sterilizable stainless steel tubes in three various versions (one double sided for the left, one double sided for the right and one straight) to protect and retract the cheeks and tongue while aspirating.

Advantages: Whilst protecting the cheeks and tongue, it also provides space and careful aspiration and allows one hand to be free, particularly during endodontic operations. This saliva ejector will connect to any surgical aspirating device.

Care: It may be sterilized, as it is made from stainless steel. This item should be cleaned after each use with an antiseptic solution.

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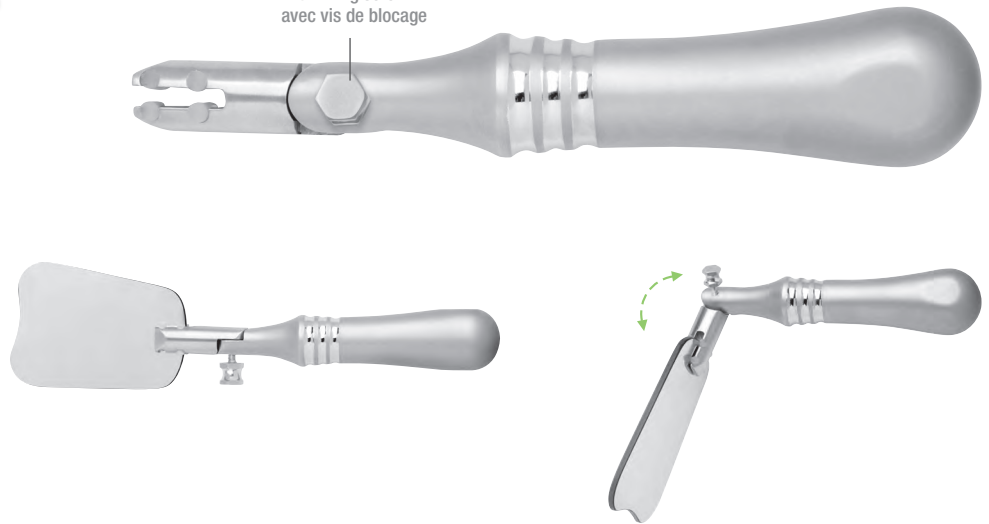
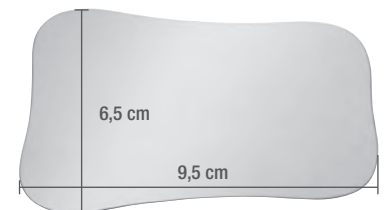
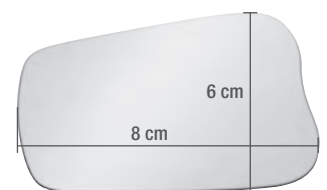
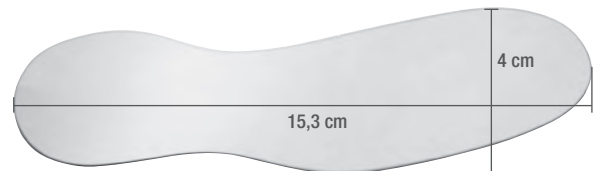
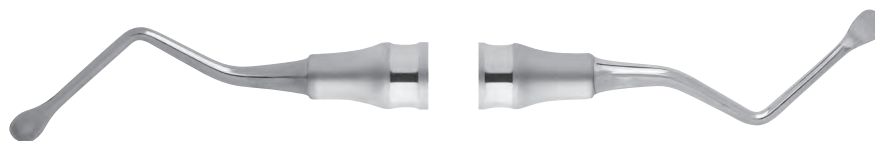
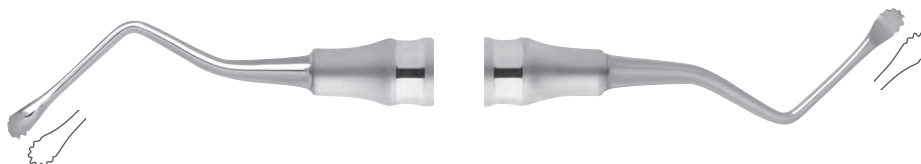
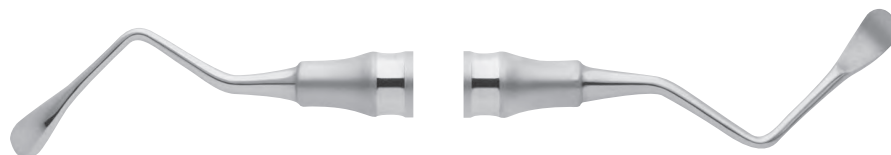
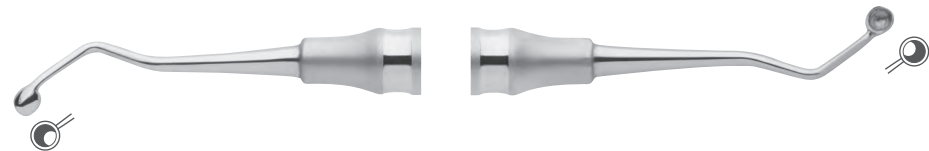
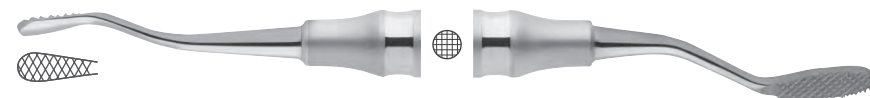
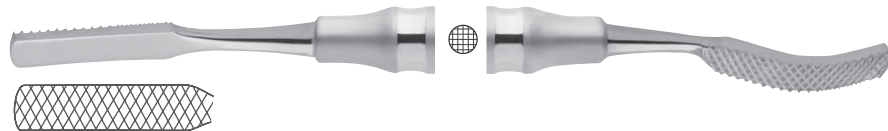
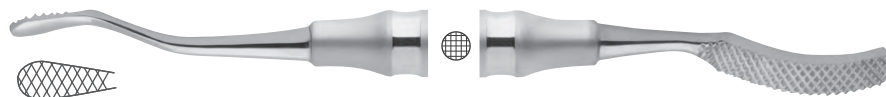
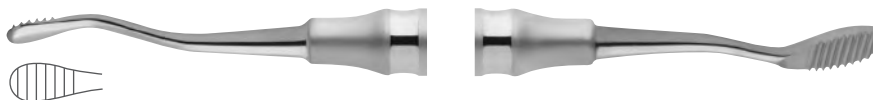
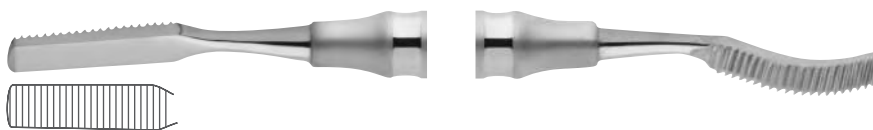
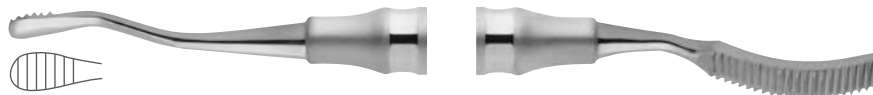
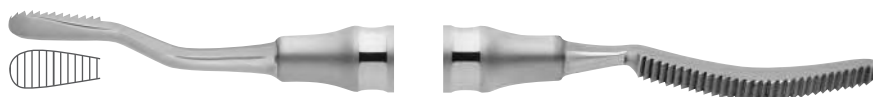
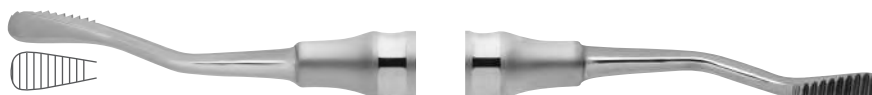
This periosteal elevator allows gentle detachment of the mucoperiosteal flap as well as the flap retraction.

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For the preparation of tissue in the corners of the approximal cavity, the bevelled instrument part (lobe knife) can be used. On the other side is a delicate periosteal elevator, with which can go under the periosteum. The Glickmann periosteal elevator is used mainly for preparing for PA treatments.

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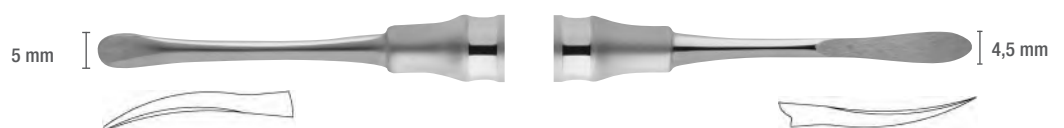
Extended field of view through the opening of the working part.

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The Perforation is for fixation of a membrane by using a pin.

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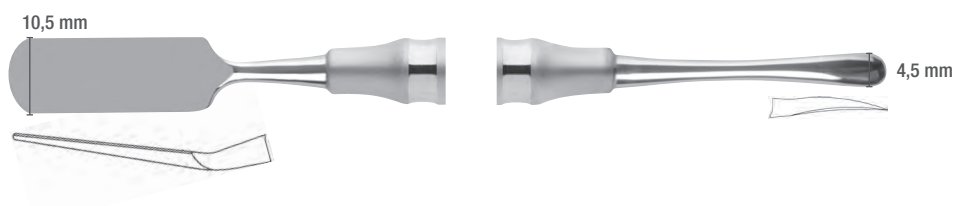
The lance-shaped workpiece-end enables the detachment of the papilla as well as detachment of the marginal gingiva crest. The mucoperiosteal flap can then be gently detached with shovel-shaped workpiece-end on the opposite side.

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The periosteal elevator was designed specifically for use in periodontal surgery. It is crucial for the success of the therapy, especially in relation to regenerative measures, that tight wound closure is achieved after the surgery. A prerequisite for this is the maximum possible preservation of soft tissue. This problem is presented in the interdental area. This very narrow and sharpened periosteal elevator simplifies this task.

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SOCKET PRESERVATION

SOCKET PRESERVATION
SOCKET PRESERVATION

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Gentle extraction of the teeth has taken place. The lobe had already been prepared in advance.

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Here, the lobe was sutured, so that the soft tissue can heal.

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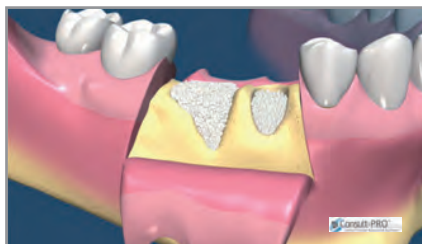
The thin bone in lower jaw area is often destroyed through an inflamed resorption (parodontitis, cyst) or the after-effects of earlier surgery/accidents. There are various possibilities (socket preservation) to prevent the subsidence of the gingiva into the injured area.

Surgical «Socket preservation»

After the extraction of the tooth the alveole can be filled with bone grafting material. This helps to prevent the subsidence of the gingiva. An implantation is mostly guaranteed once the bone and bone grafting material have fused.

Immediate implantation

If there is no inflammation, an immediate implantation is possible. Here, a combination with augmentative measures is conceivable because in this way a further absorption of the remaining residual bone can be prevented.

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The alveole can now be filled with bone grafting material.

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The corresponding base is now in place. Adequate soft tissue support and bone height are guaranteed.

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Socket preservation chirurgicale

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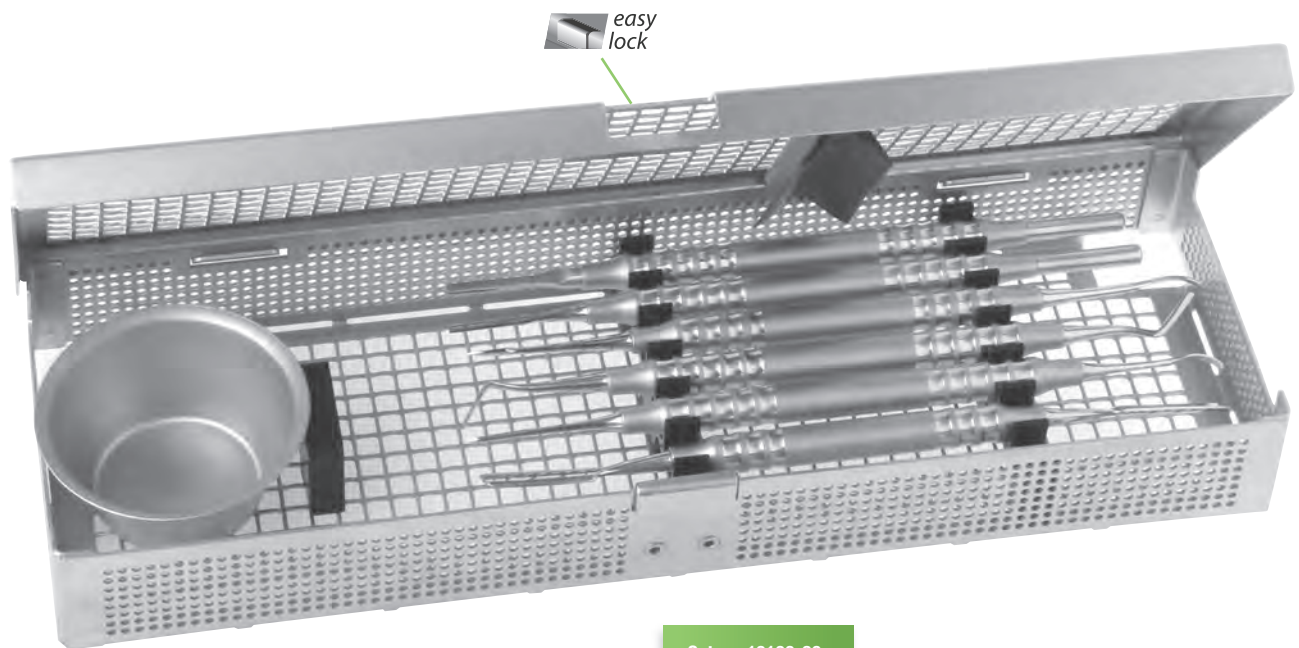
To ensure better regeneration of the bone, a collagen membrane or fleece is placed accordingly. This should provide protection from the ingrowth of soft tissue.

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SOCKET PRESERVATION SET consisting of:

SOCKET PRESERVATION SET consiste de:

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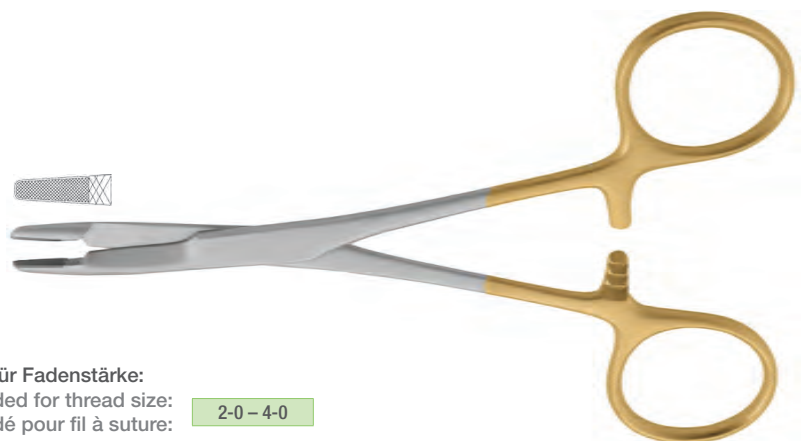
Suture groove in box lock.

To prevent threads getting caught during knotting, a special suture groove has been developed.

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TC (Tungsten-Carbide) inserts

To ensure absolute close fitting of the needle, the jaws of the needle holder are equipped with precisely machined tungsten carbide tips (TC plates).

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Spring

A double-spring has been inserted to guarantee the simple functionality of the needle holder. Generally, these do not require maintenance as no abrasion occurs during motion.

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Surface

The surface of the needle holder is matted to prevent light reflection

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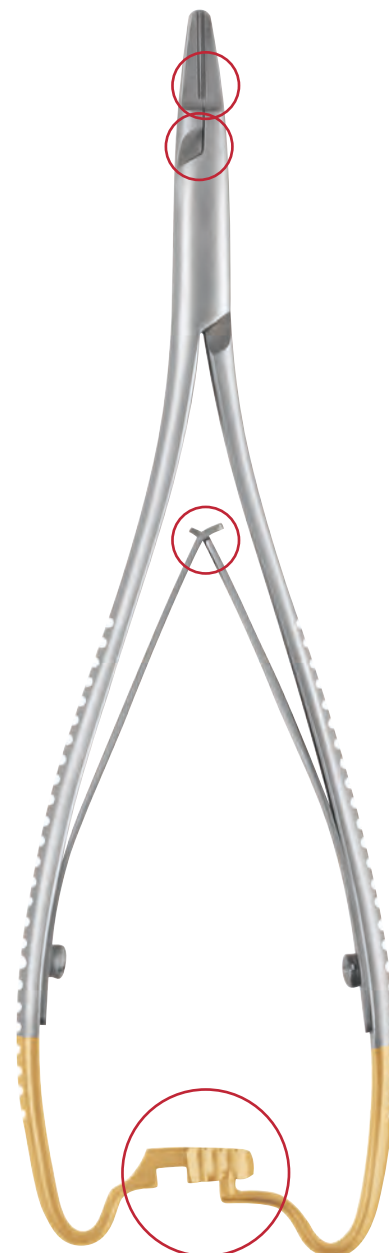
Identification

For easier identification both ends of the handles are gold-plated.

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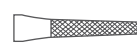
Rotary catch

The needle holders are designed with internal rotary catches to eliminate the risk of surgical gloves getting caught.

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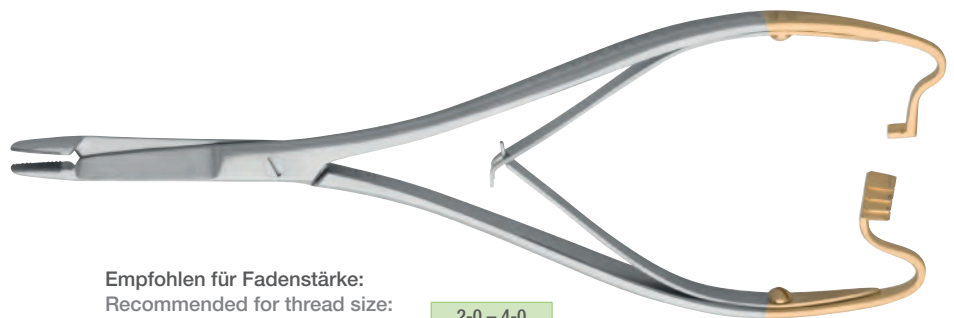
Minimal invasive needle holders with lock

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These fine and lightly turnable needle holders have been developed for atraumatic and minimally invasive sewing. Such sutures are used with free mucous membrane transplantation and apical and lateral shifting lobes. Interproximal places can be reached with the slight form of the Barraquer and Castroviejo needle holders. These needle holders are suited for the operation with loupe or microscope.

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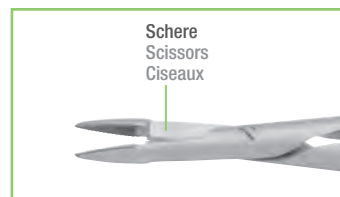
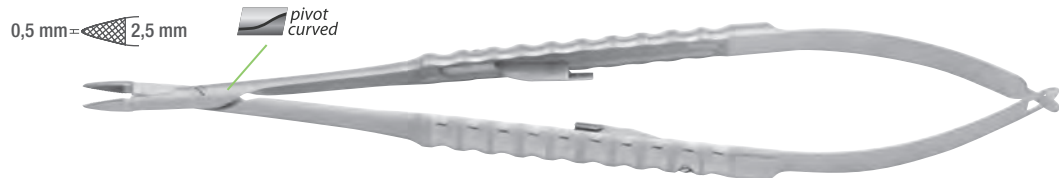
Micro needle holder with scissors and lock

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**2-in-1 instrument****saving of costs and space****no change of instrument****TC**

| | |
|------------------|-----------------------|
| 1089-10 F | F-line |
| M TEC | 0,5 / 2,5 mm 17 cm |

gerade, mit Kreuzriefung
straight, with cross serration
droit, striure en forme de croix

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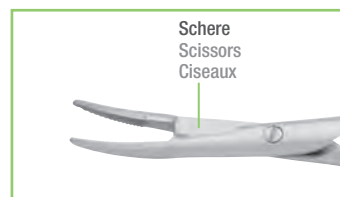
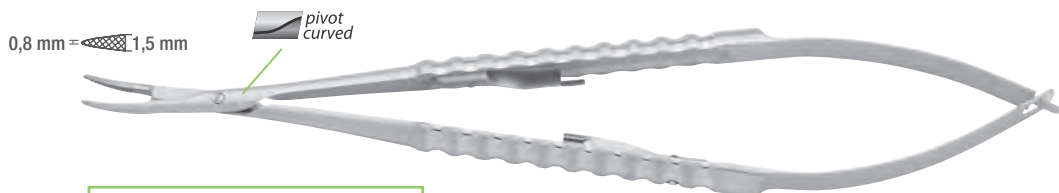
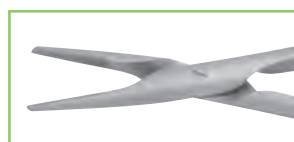
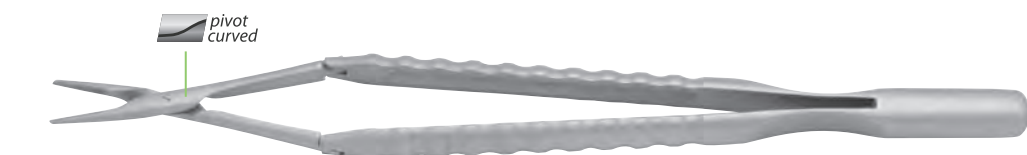
For removal of bone protrusions and excess soft tissue

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Soft tissue management has become more and more significant in today's dental implant procedures. Therefore work around the implant should be as atraumatic as possible. For this reason, we have developed special instruments for minimally invasive intervention in the spectrums of periodontology and implantology. Due to aesthetics, these instruments are indispensable.

As basic equipment for soft tissue management we have compiled the Flap-Design Set. This kit is clearly arranged and has a small number of high quality instruments.

In the future, these instruments will be essential for applications involving the transplantation of connective tissue, precise split lobes, shaft lobes, roller lobes, apical and coronal adjustment lobes and the double papilla lobe.

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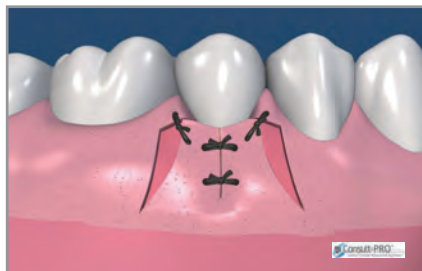
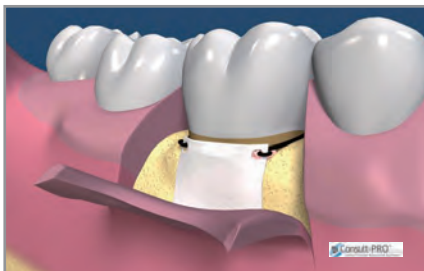
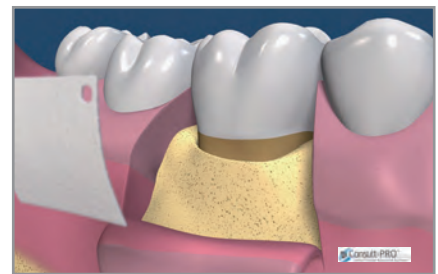
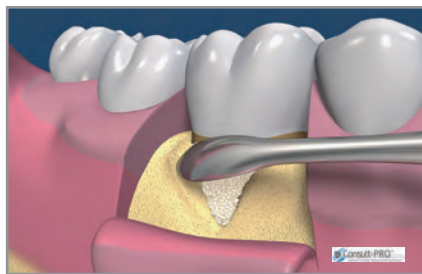
Example of use:

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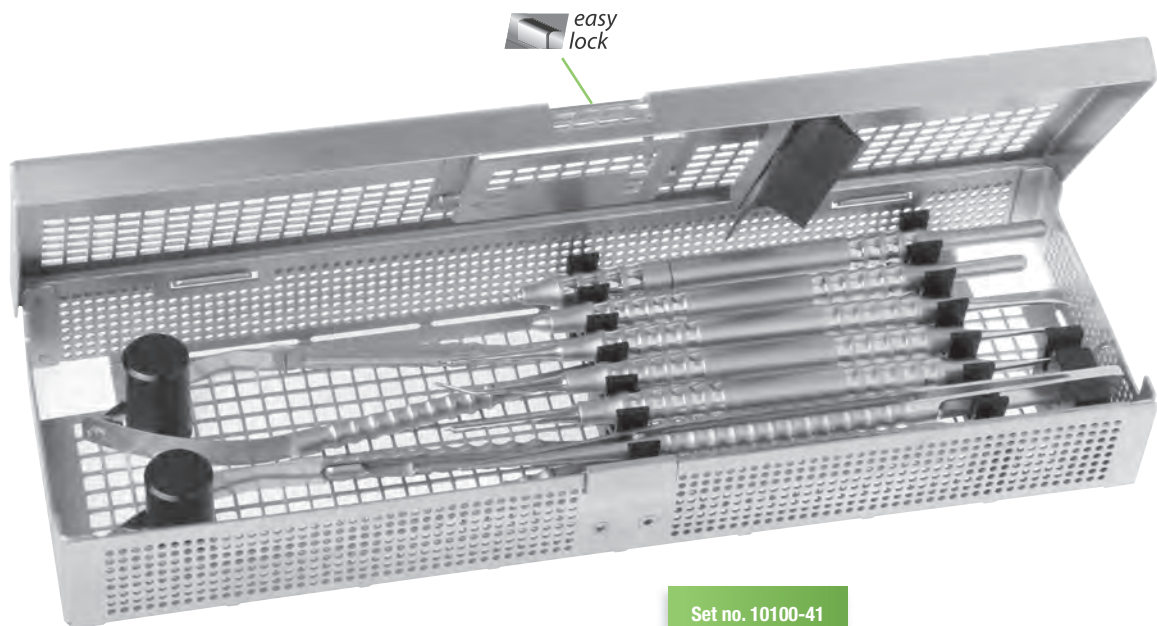
Images provided by
www.consult-pro.com
Consult-PRO
Dental Patient Education Software

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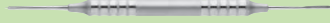
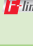
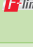
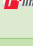
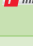
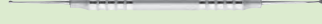
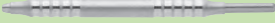
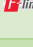
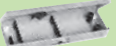
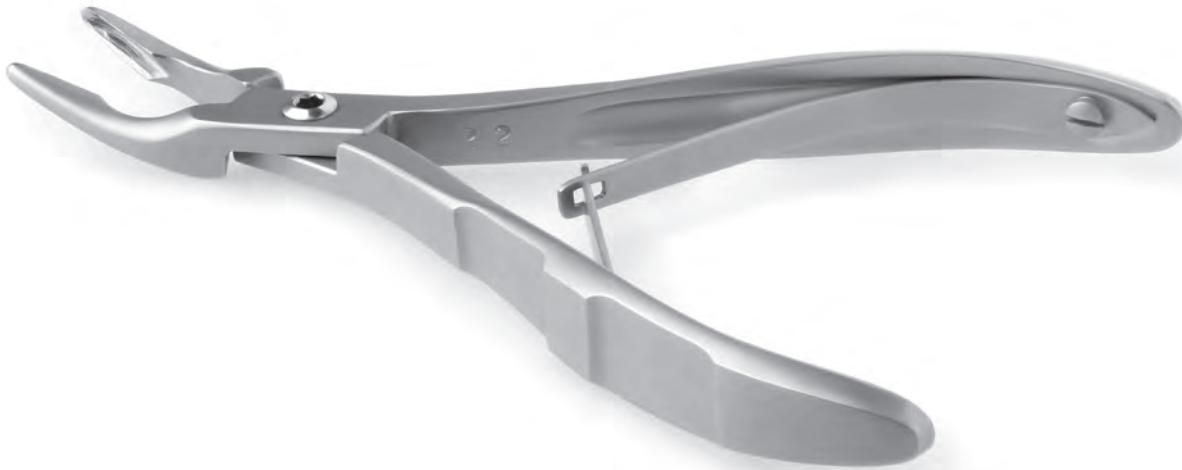
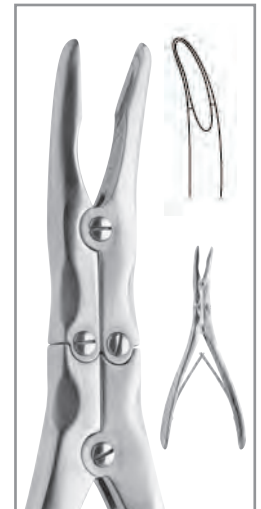
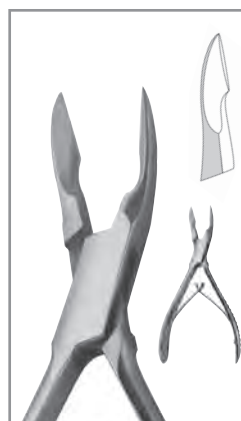
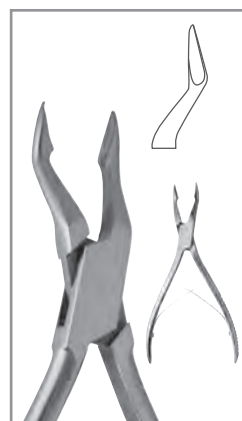
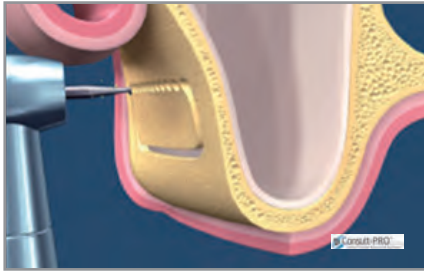
Utilization of the Sinus-Lift instruments

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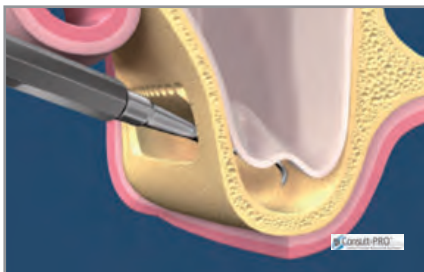
A paramarginal wide palatal incision is made in the mucous membrane. Once the periostum lobe has been prepared, an area approximately 7 x 10 mm is marked with a diamond-coated ball-drill and separated from the bone.

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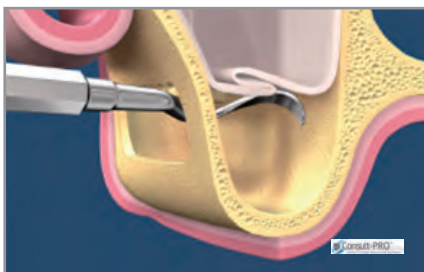
Specifically designed Sinus-Lift elevators (or curettes) are used to carefully remove the thin mucous membrane. The dissection of the mucosa takes place up to and including the upper lid margin.

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The bone implant site exposed by the dissection is now fully prepared for the proposed implant system, and its depth can be measured.

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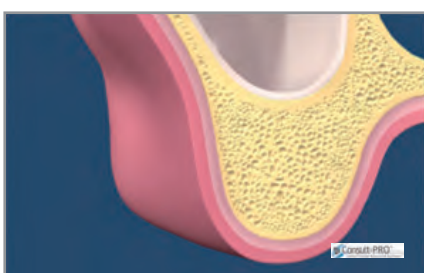
After the material for augmentation has been inserted, the Sinus-Lift instrument can be removed. (In this example, bone grafting material has been used for the augmentation).

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After the instrument has been removed, the mucous membrane lobe can be fixed and sutured.

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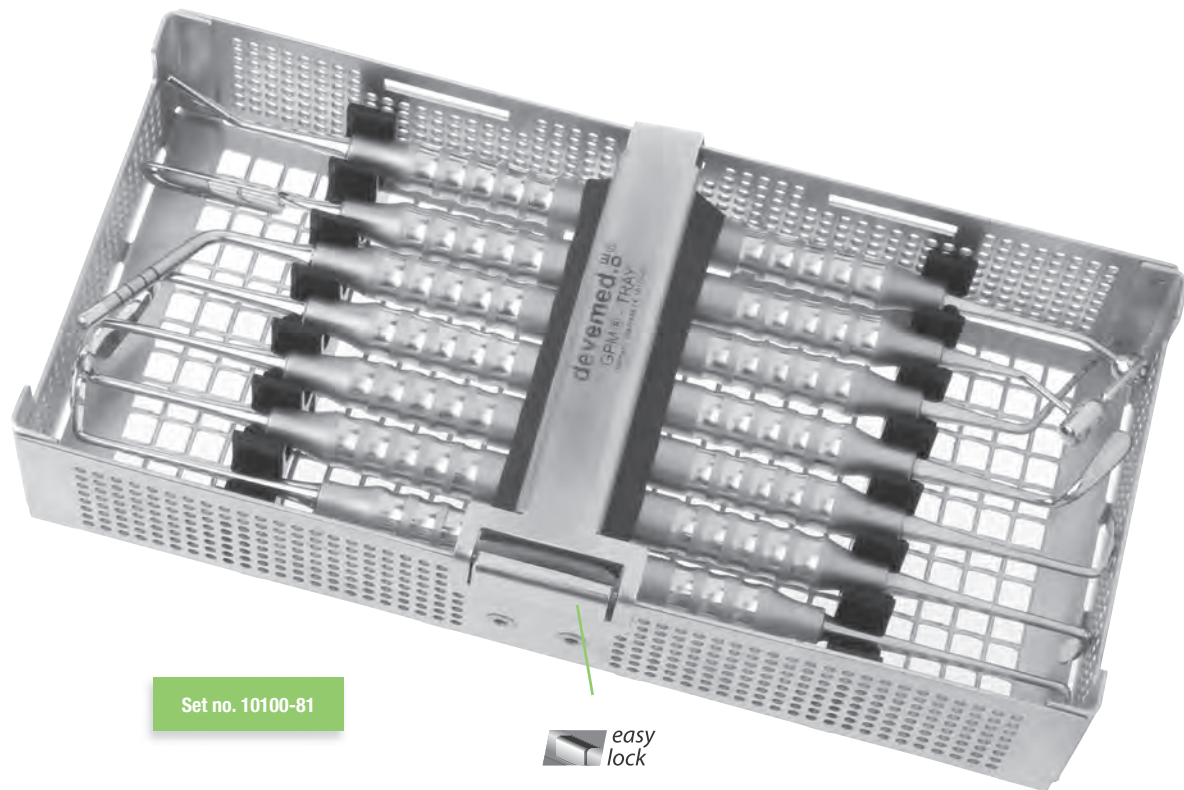
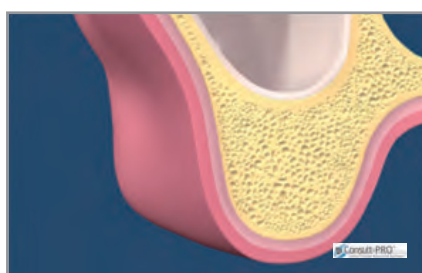
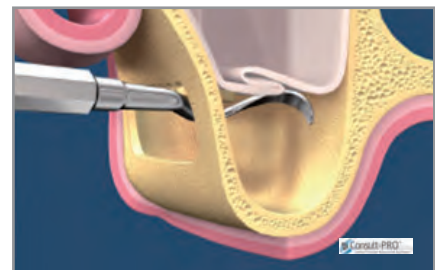
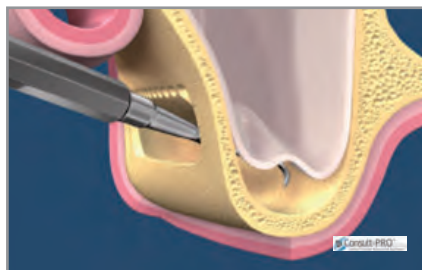
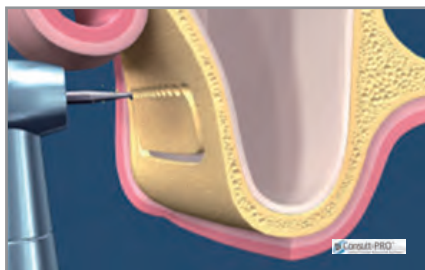
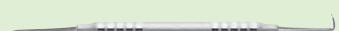
When there is insufficient natural bone to anchor an implant in the molar region of the maxilla, a Sinus-Lift operation is generally carried out. The traditional Sinus-Lift operation bears certain risks. These can be reduced by minimally invasive sinus cavity fillings to a minimum.

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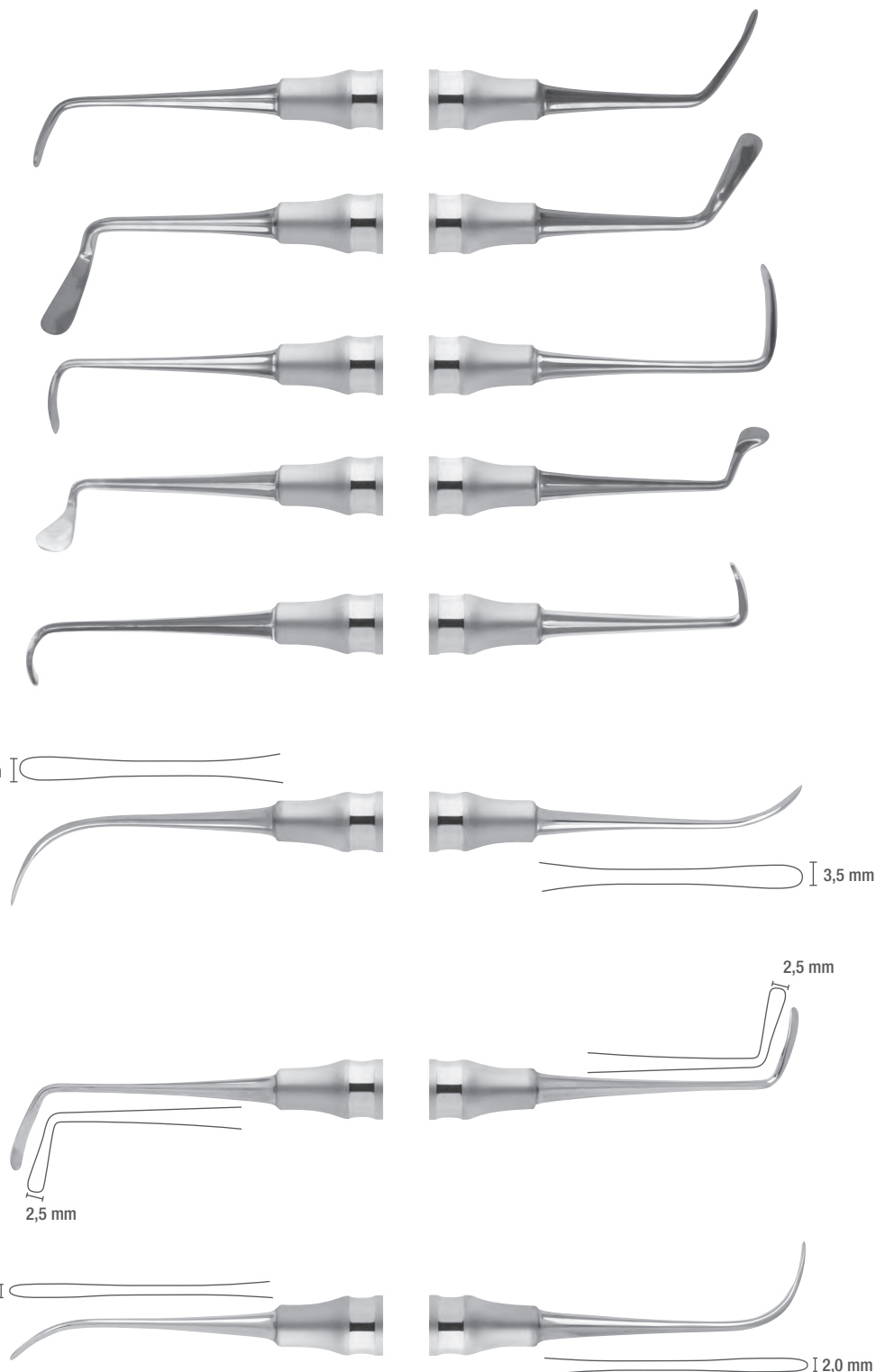
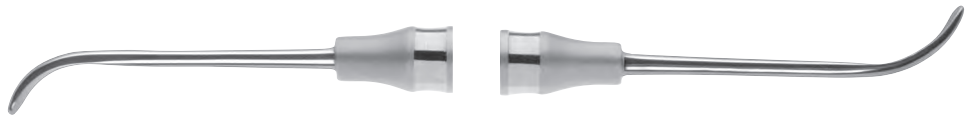
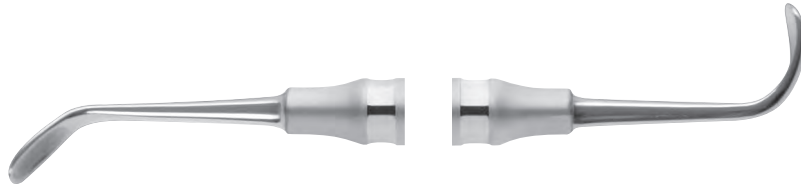
Flexible Sinus-Lift instruments support you in your work. The flexible work-ends are pre-bent to suit normal circumstances. They can be carefully re-bent and customised to the local anatomy, greatly facilitating the work.

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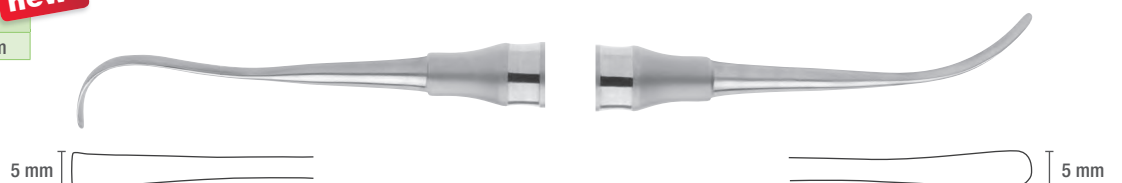
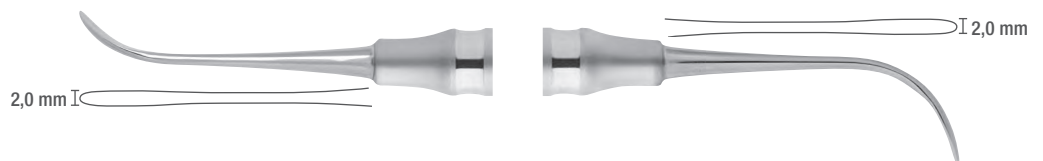
Injuries to the diaphragm are prevented and operating sensitivity increases, due to the flexibility of the Sinus-Lift curettes.

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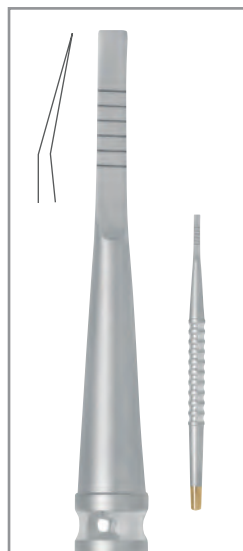
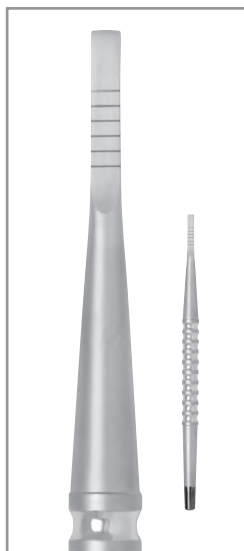
The sharp workpiece ends can be used to optimally open and stretch the alveolar ridge, after the use of a diamond cutting wheel. The 2 mm graduations ensure precise monitoring of the penetration depth.

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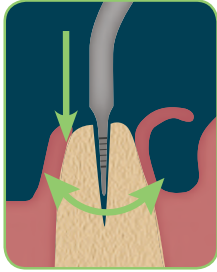
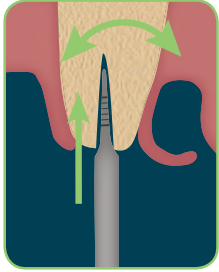
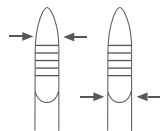
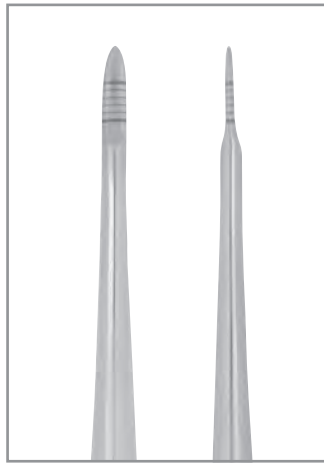
 The alveolar ridge is prepared for bone spreading with a rotating disk or with a fine chisel. Use the elliptical spreaders to sagittally widen the alveolar ridge, in order to maintain the implant site.

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gerade, graduert; 5-6-7-8-9-10
straight, graduated; 5-6-7-8-9-10
droit, gradué 5-6-7-8-9-10

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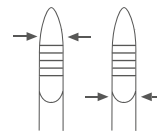
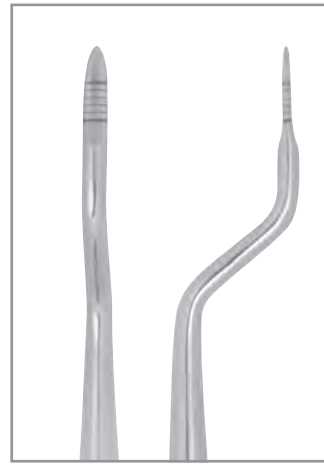
The straight bone spreaders are particularly suitable for the front teeth area.

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The bayonet bone spreaders are particularly suitable for the area of molars and premolars.

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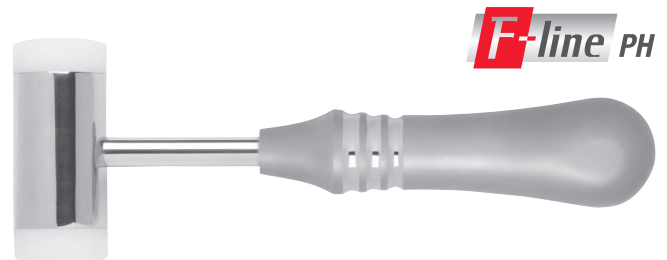
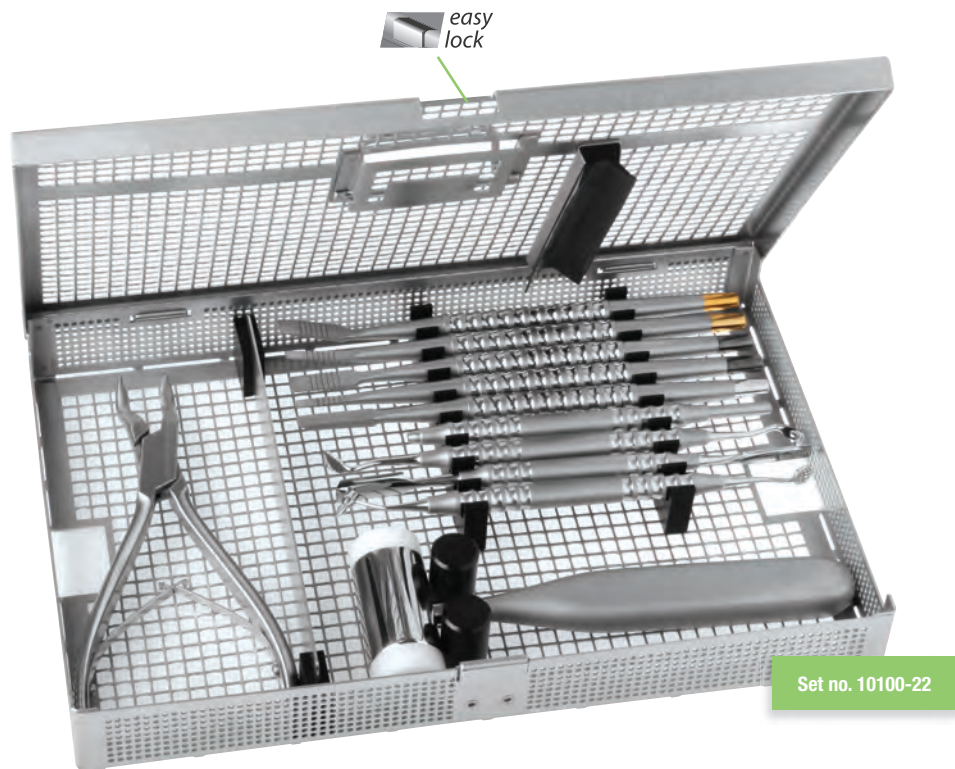
für 860-4
for 860-4
pour 860-4

RIDGE SPLITTING SET

RIDGE SPLITTING SET
RIDGE SPLITTING SET

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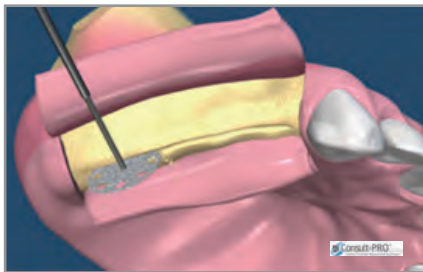
The use of human, local bone considerably enhances the rate of success as opposed to when using bone blocks or bone pieces from other areas of the body. Therefore, Ridge-Splitting, a very gentle method, is used to widen the autologous bone of the alveolar ridge. Our recommended instruments range is suitable for single tooth implants as well as for wider ridge patterns.

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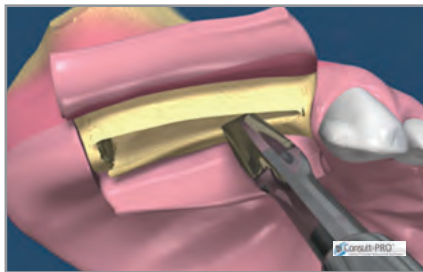
A diamond-coated disk is used to mark the fracture line on the upper jaw.

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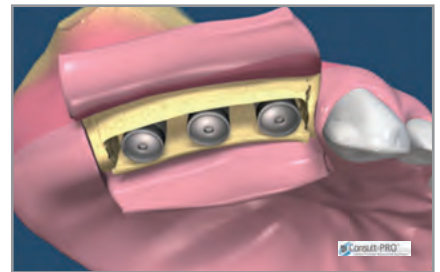
A flat chisel is used for the widening/luxation of the alveolar ridge in the actual ridge-splitting process.

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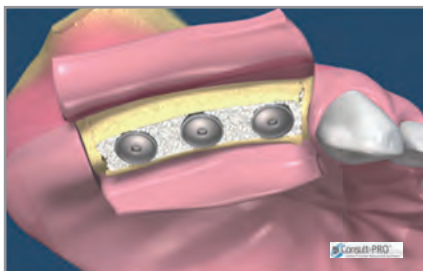
After the implants have been inserted, the fill-sites can be seen resulting from the ridge-splitting.

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In this example, these filling areas have been generously filled with bone grafting material.

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After about seven months of regeneration, the widening of the ridge is clearly visible.

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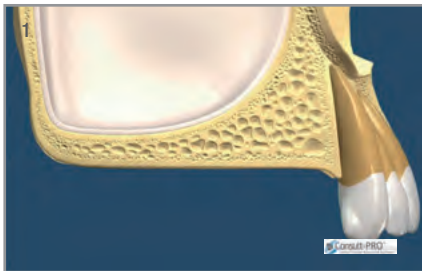
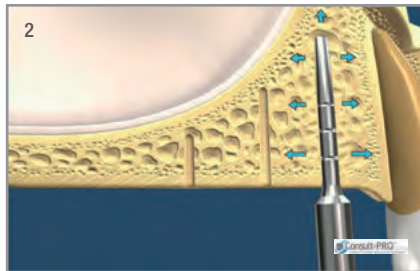
An extension of the maxillary sinus and resorption of the ridge can be expected after the extraction. If this occurs, bone condensation should be applied.

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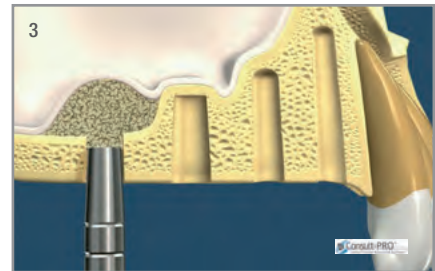
When executing a closed sinus elevation in conjunction with a lateral condensation (Bone Condensing), a pilot hole with 2 mm diameter to the cortical floor of the maxillary sinus is made. When inserting the first osteotome (Art. No. 2821-27 is recommended) into the predrilled cavity, attempts should be made with light pressure or light hammer blows to lift the Schneiderian membrane.

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If you have reached the desired diameter, the augmentation material should be inserted and condensed with the respective bone pusher.

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When you have reached the appropriate height for the implants (plus 2-3 mm), they can be inserted. Please ask your implant supplier for correct implants for this surgical procedure. As of now on a healing period of between four to eight months begins.

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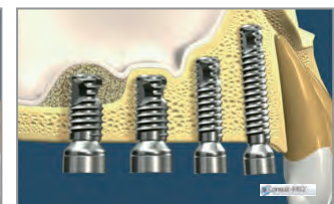
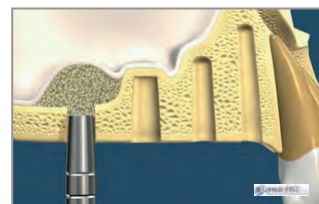
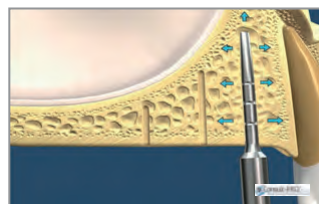
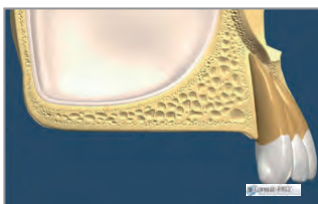
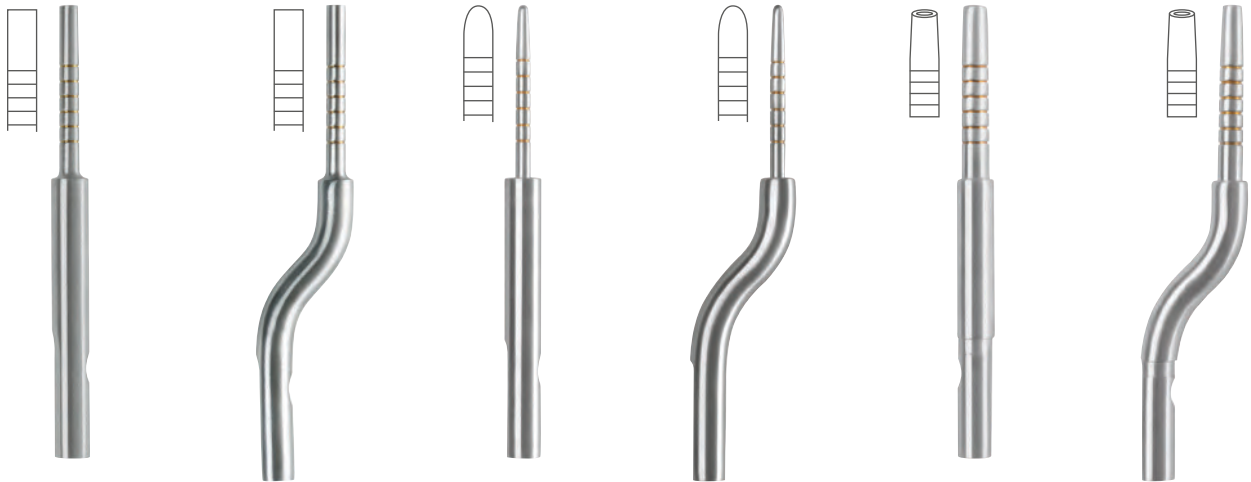
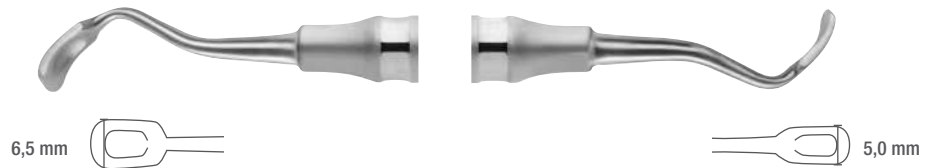
With this bone scraper, you can significant limit the postoperative discomfort of your patients, because it guarantees the uncomplicated and fast removal of bone material. The sharp working ends allow the bone material to be cut into fine slices without great efforts. For optimum sharpness after each use, we recommend that the tips are regrind with an Arkansas stone.

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This applicator ensures the optimal blending and transportation of bone grafting material. The narrow workpiece is available for smaller quantities, and the wider workpiece for larger quantities.

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To be able to insert the implant into the maxilla, enough jaw bone must be available. Should this not be sufficient bone, an additional bone augmentation must be carried out in addition.

This cleverly compiled Set allows you to perform each of the 5 augmentation techniques shown on the next page.

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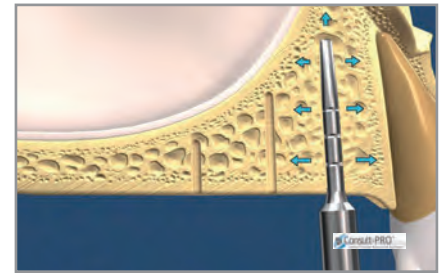
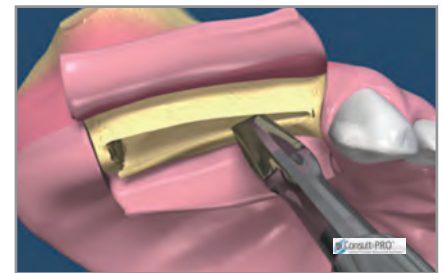
BONE-GRAFTING SET – ALL IN ONE consisting of:

AUGMENTATION SET – ALL IN ONE consiste de:

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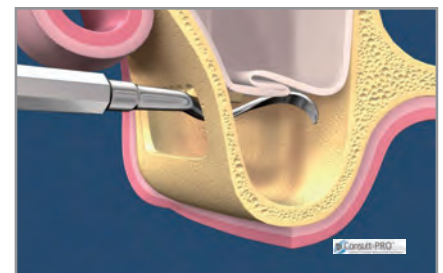
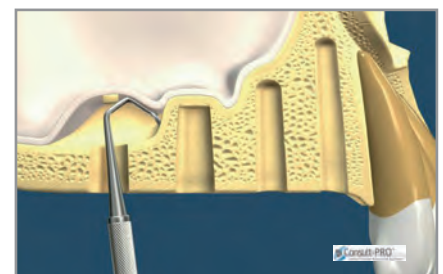
These holding forceps enables you to work with all major implant system screwdrivers. The abutments can be easily replaced when working with the holding forceps.

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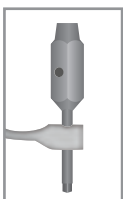
These bone ring holding forceps have a very fine thread, which enables optimal adjustment when setting the bone block. Furthermore this forceps allow for the optimal stabilization and precise machining of the bone. It also serves to minimize fracturing.

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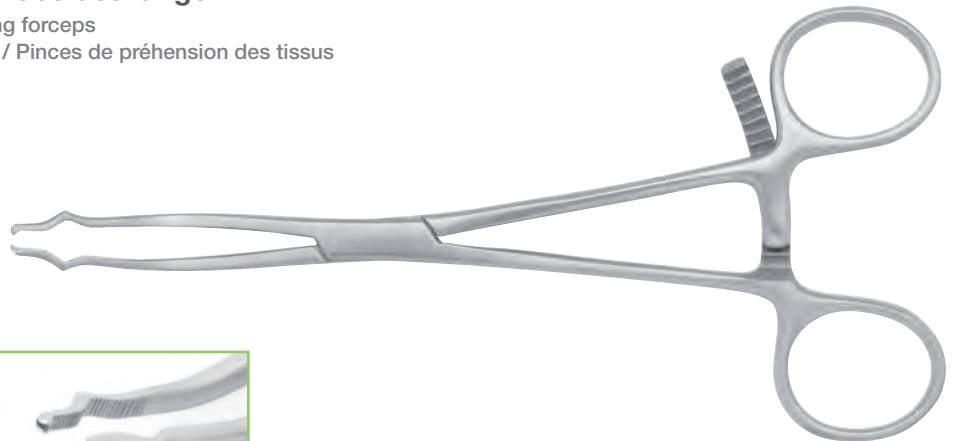
Bone holding forceps / Tissue grasping forceps

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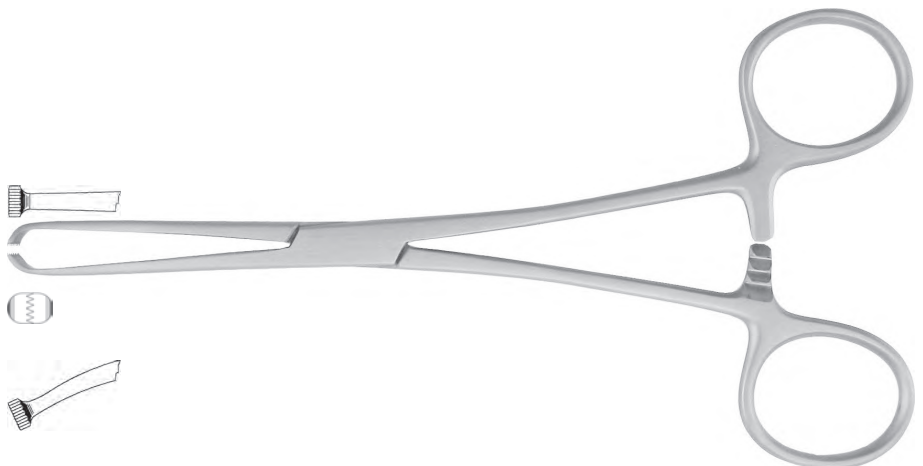
This depth gauge shown here allows you to accurately measure the depth of the implant cavity.

The neighbouring anatomical structures and the of the cavity can be optimally checked. The horizontal and vertical bone volume can also be precisely measured intraoperatively.

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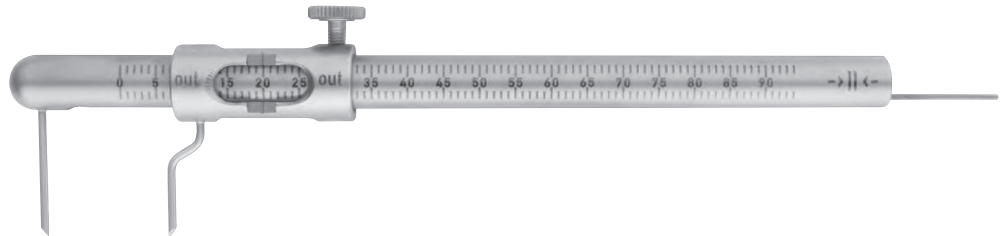
Markierung: 1-2-3-4-5-6-7-8-9-10
Graduation: 1-2-3-4-5-6-7-8-9-10
Graduation: 1-2-3-4-5-6-7-8-9-10

Messinstrumente

Measuring instruments
Instruments de mesure

| | |
|----------------|---------------------|
| 2406-50 | 12,5 cm 0-100 mm |
|----------------|---------------------|

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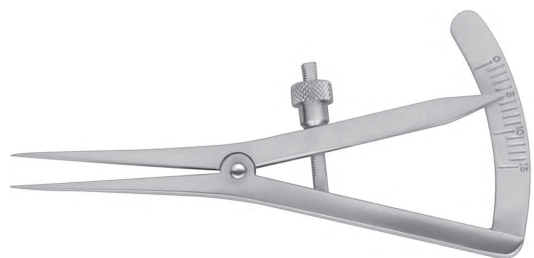
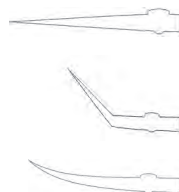
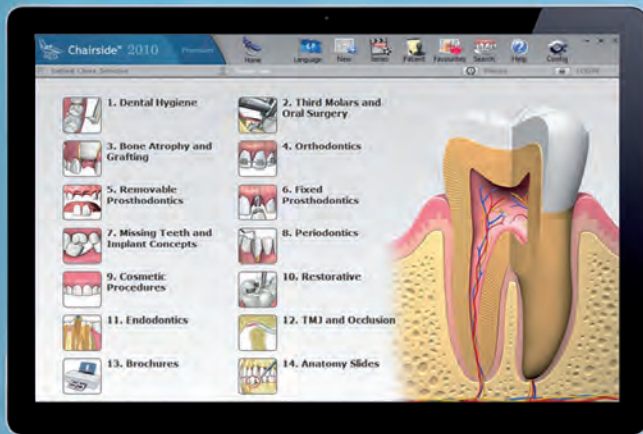
Chairside™

The World Leader in Dental Patient Education

Spend less time Explaining Dentistry and more time Doing Dentistry

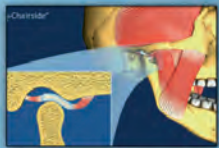
With over 800 3-D animated presentations that clarify:

The Need for Treatment ✓

Complex Concepts ✓

Treatment Recommendations ✓

Education Made Easy!

Consult-PRO is the leading manufacturer of dental patient education software. Educate your patients, motivate them to proceed with treatment and create an efficient and patient-friendly office environment, with Consult-PRO's Chairside!

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| | |
|--|-----|
| 5702 | # 2 |
| Form C½R | |
| ungelocht unperforated non perforé | |

| | |
|----------------------------------|-----|
| 5722 | # 2 |
| Form C½R | |
| gelocht perforated perforé | |

| | |
|--|-----|
| 5712 | # 2 |
| Form C½L | |
| ungelocht unperforated non perforé | |

| | |
|----------------------------------|-----|
| 5732 | # 2 |
| Form C½L | |
| gelocht perforated perforé | |

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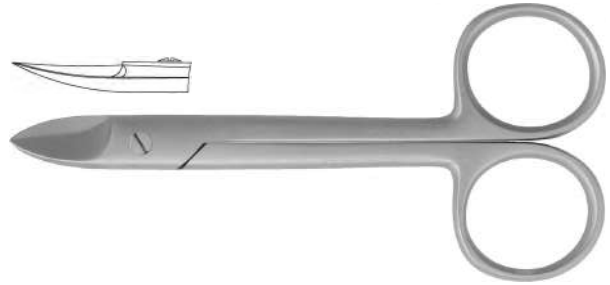
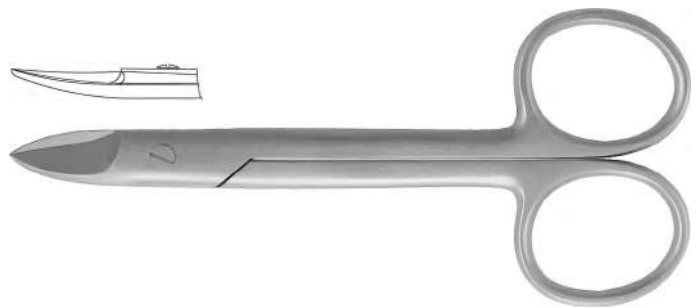
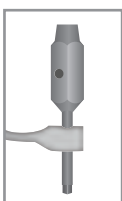
These holding forceps enables to work with all major implant system screwdrivers. The abutments can be easily replaced when working with the holding forceps.

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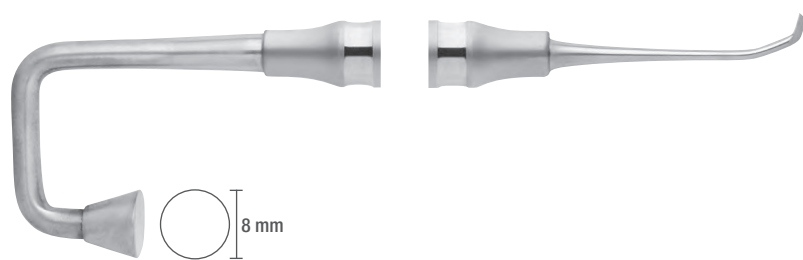
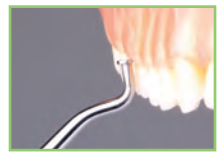
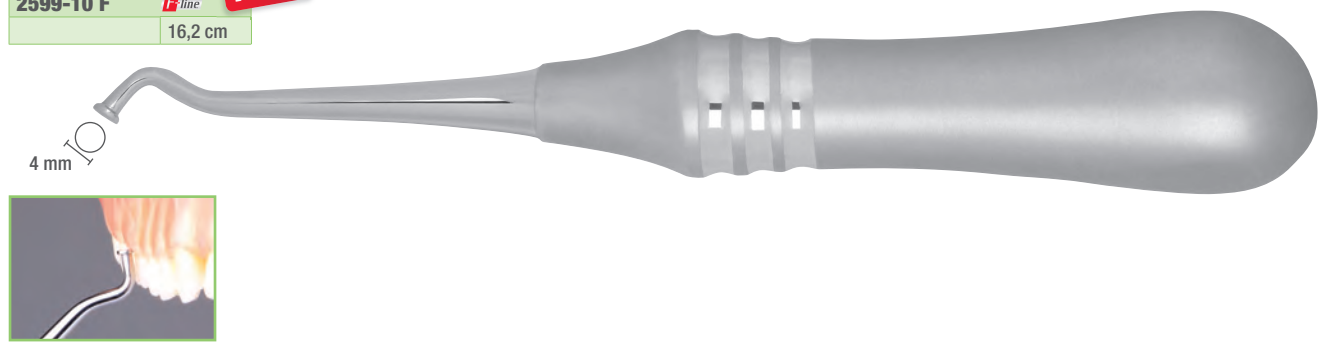
 Insert the pointed end of the prosthesis lifter between two telescopic crowns or between the telescopic crown and the prosthesis.

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By even and gentle pulling (at the front and rear ends of the telescopic crown or prosthesis), the compound will be dissolved. This should be done uniformly, in small steps.

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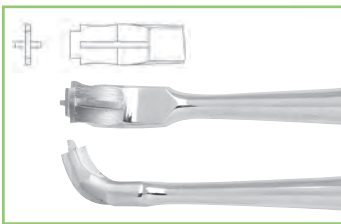
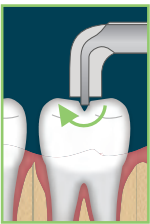
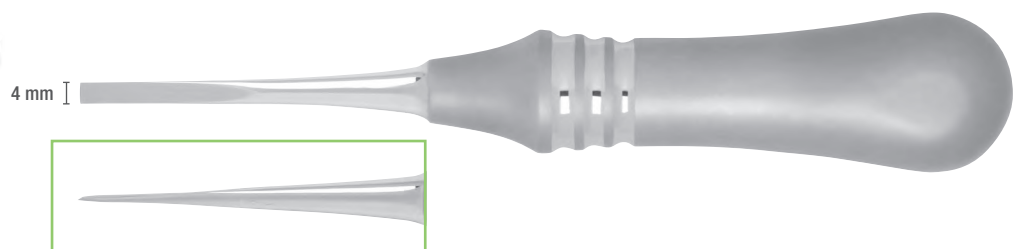
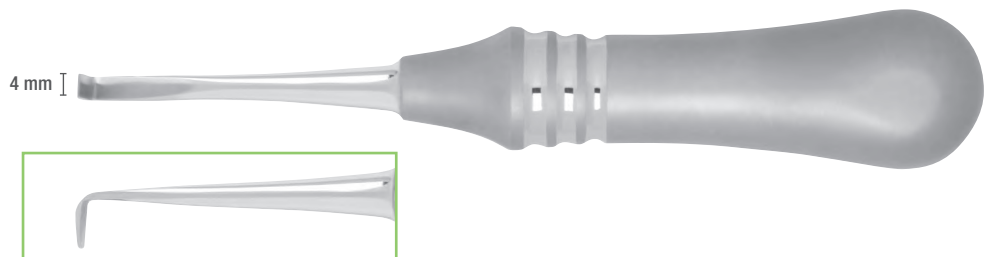
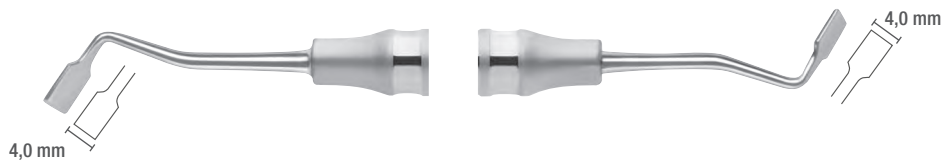
This elegant spreader serves the purpose of crown spreading despite the appearance. Its fine working tips allow a better access.

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These forceps enable the removal of the temporary synthetic inserts in the upper and lower jaw. Application is possible in frontal area as well as in the rear. The TC inserts enable a firm grip, so that the provisional insert can be removed without causing any damage.

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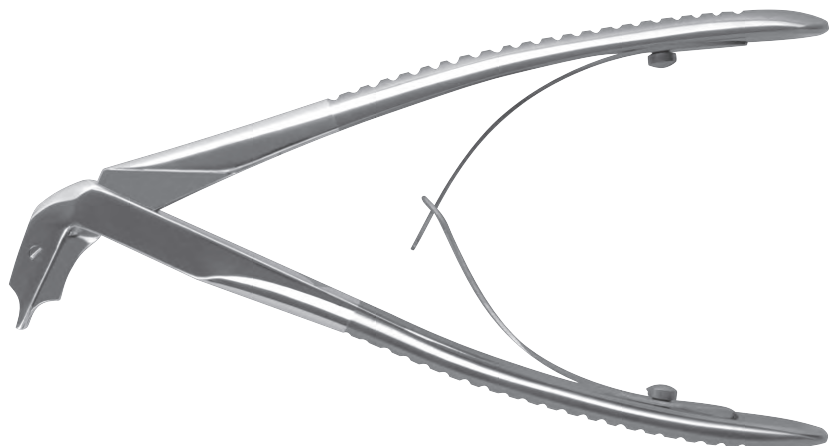
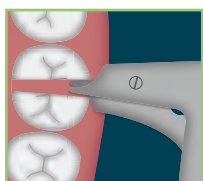
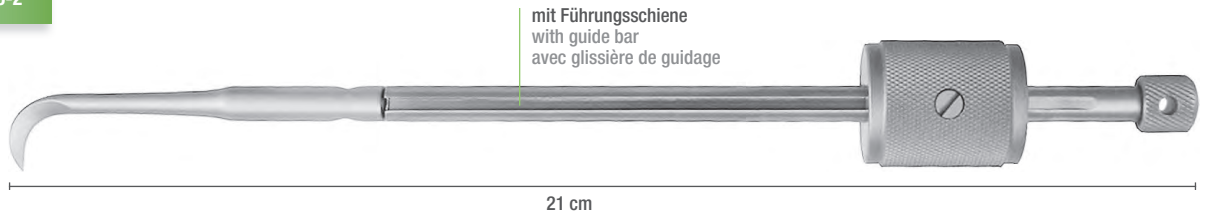
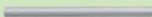
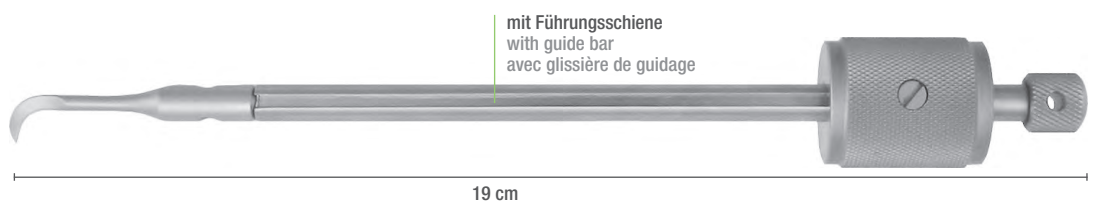
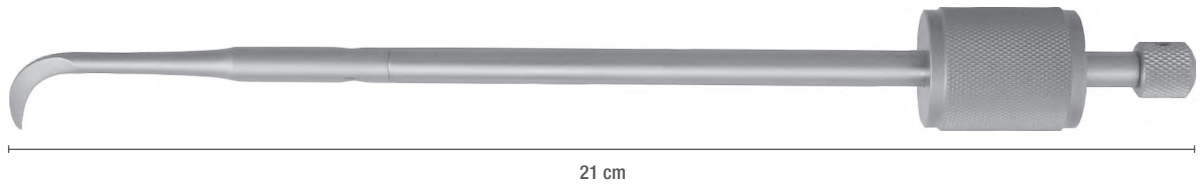
Teleskopkronenzange und Zubehör

Telescopic crown pliers and accessories

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Spitze: beweglich
Tip: movable
Pointe: mobile

2746-6

Inbusschlüssel
Allen wrench
Clé six pans

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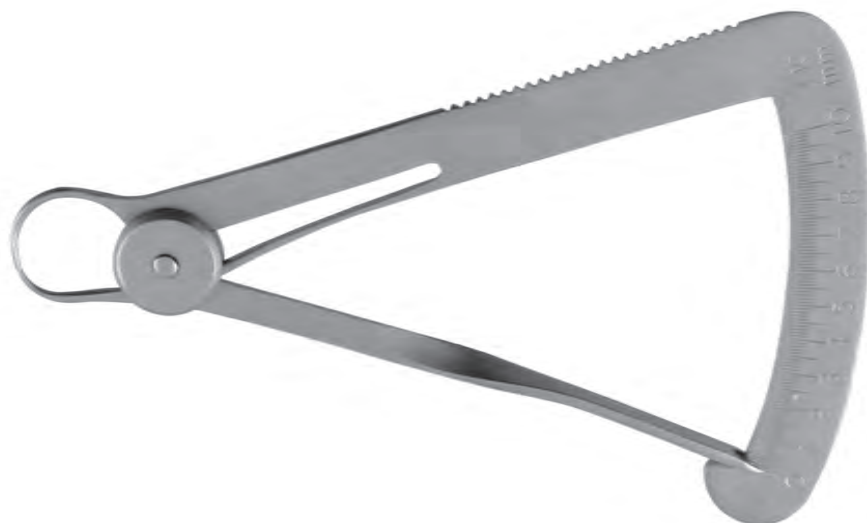
Dr. Krause-Hohenstein Crown Pliers
 - Ergonomic design with cranked jaws for assured grip
 - jaw configuration provides circular aperture
 - grips crowns and bridges securely, regardless of material
 - does not damage the tooth neck area.

The crown forceps can be used for a variety of applications, e.g.:

- to insert and remove temporary dressings
- to fit tight-fitting crowns
- to grip and remove superficial root pieces, especially when treating
- for extraction of all deciduous teeth
- also an excellent tool in the hands of an assistant

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Due to the particular shape of these pliers, you are able to remove crowns and bridges without any problem problems, when working in the upper and lower jaw areas even during the intraoral trial.

The advantage of these pliers is clearly evident. Previously two separate pliers were required, one for the upper jaw and one for the lower jaw. The exchangeable rubber inserts prevent damage to the crown surface and can be sterilized up to 180° C.

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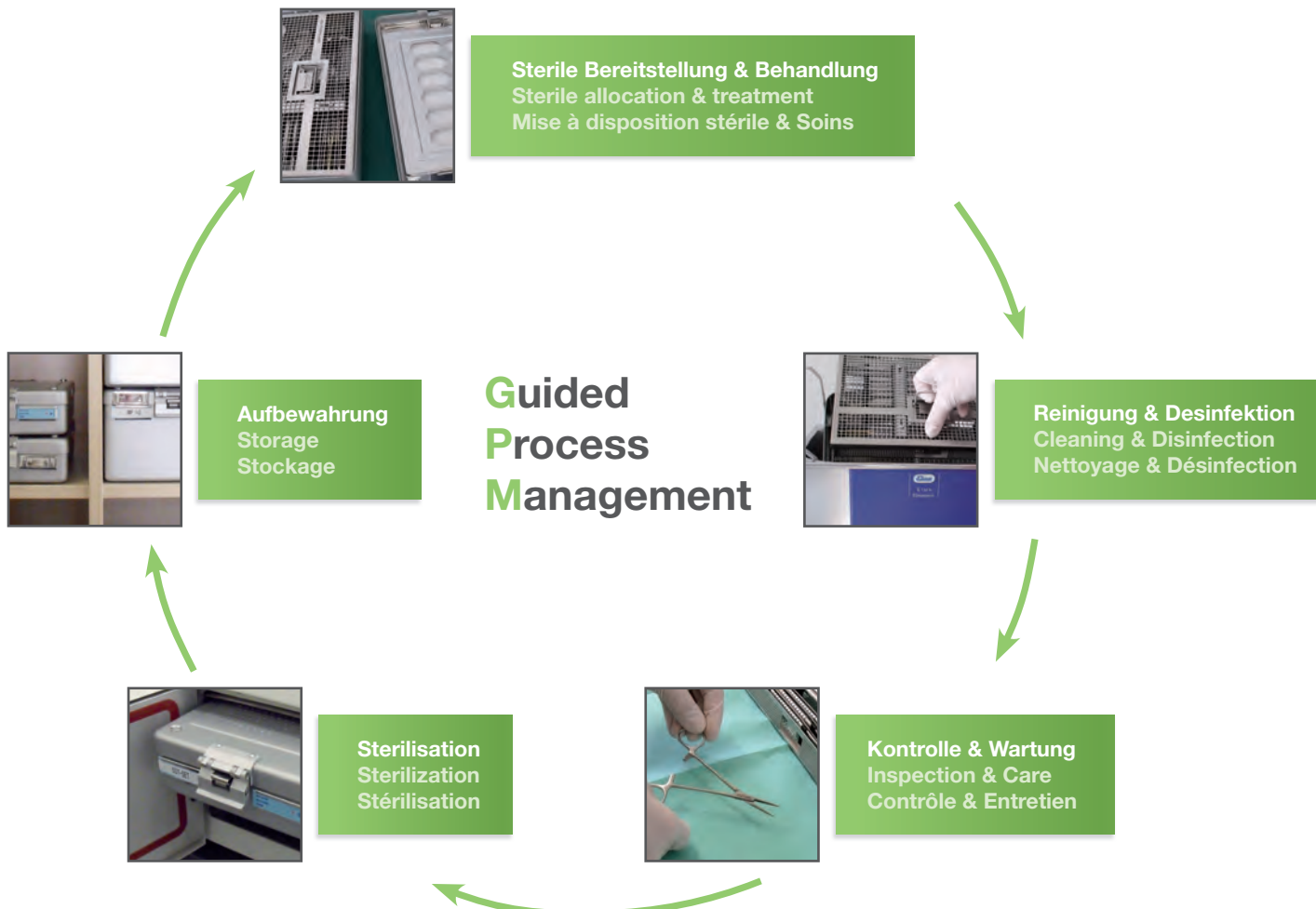
I

PRAXIS ORGANISATION | GPM

PRACTICE ORGANIZATION | GPM
ORGANISATION DU CABINET | GPM

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GPM and its advantages

GPM allows the best possible security for you, your staff and your patients against injuries and infection.

GPM system reduces your requirement for sterile packaging and thus contributes to longterm cost reductions. Therefore the return on your investment in the GPM system is assured.

The GPM system enables easy documentation and enhances the traceability of your sterilization process.

Your high-quality instruments will be optimally protected.

With the use of a GPM system your internal processes are optimized and thereby the best procedural organization is guaranteed. Thereby your practice complies the current hygiene guidelines.

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GPM Trays are made of stainless steel.

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The silicone band in the cover of the GPM Tray is excellent for placing the instruments and optimally secures their storage location.

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Effective cleaning is guaranteed due to the large perforations in the whole tray.

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The integrated handle in the cover of the GPM Tray facilitates safe and easy transport.

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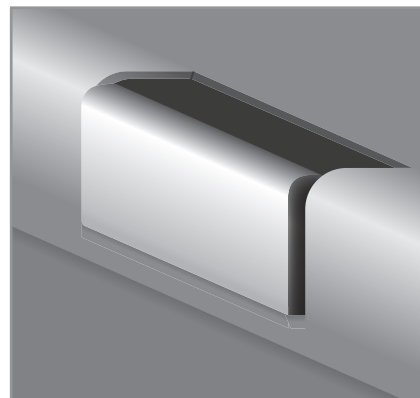
The newly developed pressure-cap *easy lock* ensures that the GPM Tray can be opened and closed comfortably and without complications.

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All instruments are adapted to fit in our system perfectly because of their maximum length of 17,5 mm. In the largest GPM Tray (275 x 180 mm), it is possible to position the racks horizontally and vertically, so bulky instruments like retractors or mallets can also be integrated in the GPM Tray without problems.

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During the development of the GPM Tray, particular importance was placed on the greatest possible use of the space available. Four separate and coordinated GPM Tray sizes allow for optimal useage and space-saving storage.

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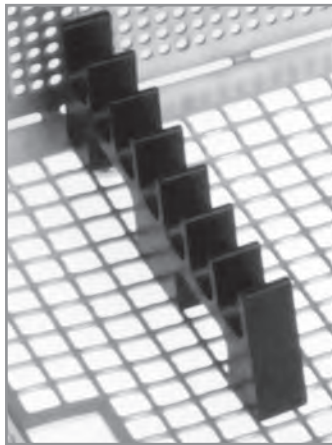
GPM System | Trays

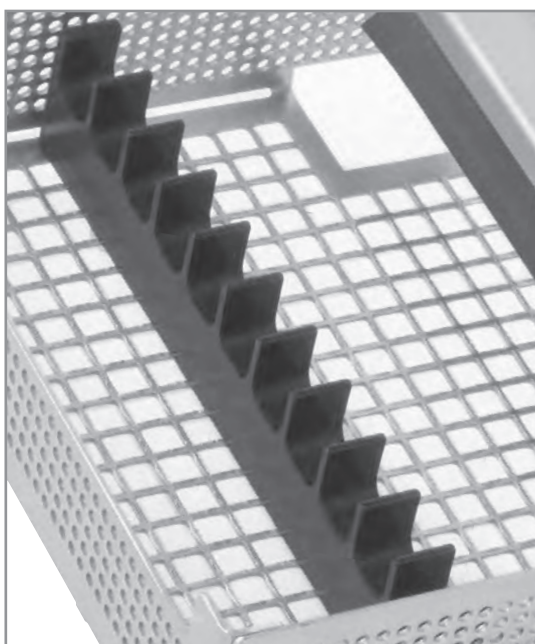
GPM System | Trays
 GPM System | Cassettes

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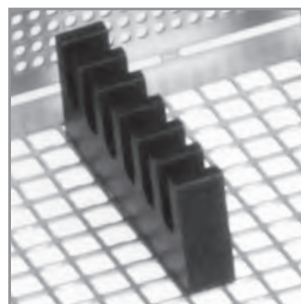
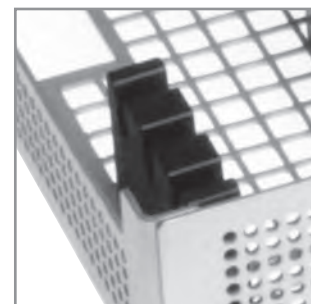
GPM Container:

- suitable for all standard manufactured autoclaves
- compatible with the devedem GPM Wash Trays
- manufactured from aluminium
- the silicone cover seal guarantees that the container will be securely locked
- the new locking system ensures hygienically perfect opening and closing of the container
- stackable
- 6 months storage time, in accordance to DIN 58953
- with perforated base and top cover
- only suitable for steam sterilization by the gravitational or fractional vacuum method
- safe transportation of sterile materials
- safety seals prevent accidental opening of the container
- the container is highly cost-effective due to its long service life
- no additional costs for disposable aseptic packaging
- no negative environmental impact through the use of disposable aseptic packaging

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The dimensions of our GPM containers and GPM Trays are completely coordinated.

Below you can find an overview of how our GPM-trays can be used with our standard GPM containers 8200-20, 8200-25 and 8200-30

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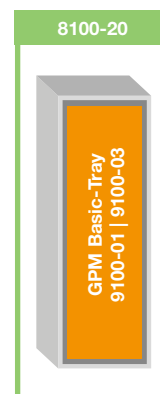
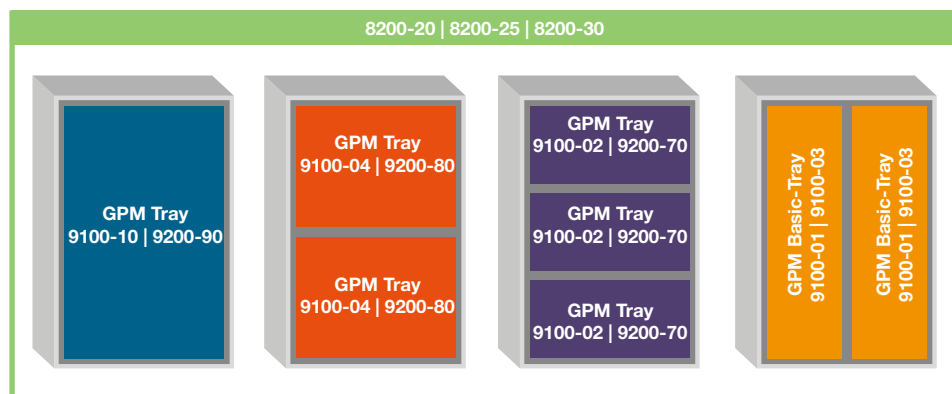
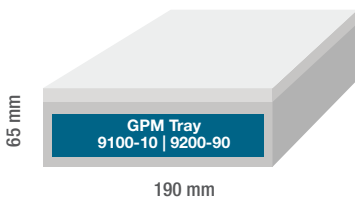
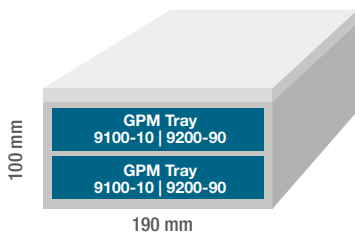
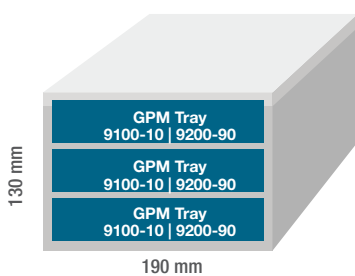
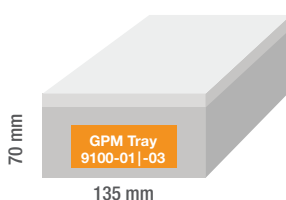
GPM Aluminium Container | Zubehör

GPM Aluminium container | Accessories

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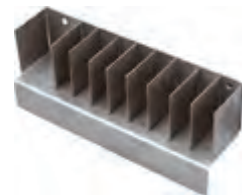
Special Oil Spray for the maintenance of surgical instruments

The Special Oil Spray is silicone-free, so staining and encrustation is avoided. It is used for the maintenance of hinged surgical instruments, such as scissors and forceps. Due to the ingredient paraffinum liquidum the product is toxicologically harmless and offers the highest degree of purity. Special Oil Spray preserves the value of instruments by providing an optimal lubrication and effectively inhibiting corrosion.

FEATURES

- for care of steel goods and surgical instruments
- avoids encrustation on instrument surfaces
- the special spray nozzle with its 40 mm cannula helps to reach hard-to-access areas. It reduces the risk to inhale spray mist and guarantees to apply a ultrathin film, which doesn't affect the sterilization.
- according to food and drug law toxicologically safe
- not suitable for a sterilization temperature exceeding 150° C

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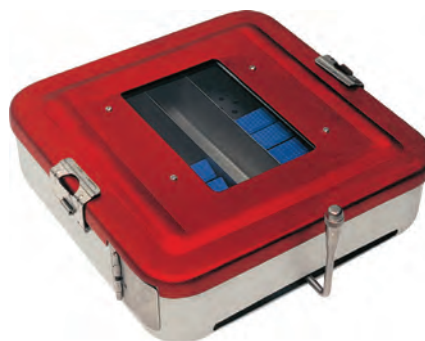
INDEX
INDEX

E

ECI | Composite instruments for aesthetic teeth restoration

ECI | Flat plugger

ECI | Occlusion former

ECI | Spatula

Ehricke, Impression tray

Electrodes for cutting and coagulation

Elevators, Retractor

Emos, Retractor

Endodontic-tweezers

Endo probe

Excavators

Extracting forceps

Extracting forceps, American-Extract | Lower Jaw

Extracting forceps, Extract 1100 | Upper Jaw

Extracting forceps, Extract 1200 | Lower Jaw

Extracting forceps, Extract 1200 | Upper Jaw

Extracting forceps, Extract 400 | Upper Jaw

Extracting forceps, Extract 400 | Lower Jaw

Extracting forceps, Extract 500 | Upper Jaw

Extracting forceps, Extract 500 & 650 | Lower Jaw

Extracting forceps, Extract 500 & 650 | Upper Jaw

Depth Gauge

Dev. Aspir 1 | Suction

Dev. Aspir 2 | Suction

Dev-lux-Set

Dev-lux-Instruments

Dev-lux blade

Dev-lux tweezers

Scalpel end cutter

Distal end cutter

Dissecting and cotton swab forceps

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B

| | |
|---|-------------|
| Baade, Copper Ring Removal Forceps | H.14 |
| Backhaus, Towel Clamp | G.27 |
| Ball-Shaped Dykal Filling Instruments | B.03 |
| Ball-Shaped Filling Instruments | B.04 |
| Band Pinching and Ligature Locking Pliers | H.17 |
| Bar Bending Pliers | H.18 |
| Bar for Crown Remover | H.09 - H.10 |
| Barnhart Standard, Universal Curette | C.08 |
| Barraquer, Needle Holder | G.53 |
| Barry, Root Elevator | E.20 |
| Barth, Holding Forceps | G.88 - H.05 |
| BASIC IMPLANTOLOGY SURGICAL SET | G.02 - G.03 |
| Beale, Modelling Instrument | B.11 |
| Beck, Root Fragment Elevator | E.23 |
| Beebee, Crown Scissors | H.04 |
| Bein, Root Elevator | E.18 |
| Bending Pliers for Plates | H.17 |
| Bernard, Desmotom | E.04 |
| Bernard, Root Elevator | E.16 - E.17 |
| Beyer, Bone Rongeur | G.59 |
| Biehler, Compactor/Applicator | G.80 |
| Biehler, Membrane Lift/Granulate Plugger | G.79 |
| Bishop, Cheek and Tongue Retractor | G.32 |
| Black, Filling Instrument | B.06 |
| Black, Gingival Margin Trimmers | B.12 |
| Blades, Scalpel | G.06 - G.07 |
| Bone Aspirator | G.83 |
| Bone Carrier | B.12 |
| Bone Chisels | G.67 - G.69 |
| BONE CONDENSING SET | G.74 - G.75 |
| Bone Curettes | G.38 |
| Bone Cutting Forceps | G.59 |
| Bone Extraction and Application | G.78 - G.82 |
| Bone Files | G.39 - G.40 |
| Bone Holding Forceps | G.89 |
| Bone Mill | G.84 |
| Bone Pusher Attachments | G.77 |
| Bone Pusher | G.76 |
| Bone Ring Holding Forceps | G.89 |
| Bone Rongeurs | G.58 - G.59 |
| Bone Scraper | G.78 |
| Bone Spreaders | G.68, G.70 |
| BONE SPREADING SET | G.69 |
| Bowdler Henry, Retractor | G.31 |
| Bruenings, Tongue Spatula | G.34 |
| Brush Dispenser | I.17 |
| Burnishers | B.10 |
| Buser, Bone Scraper | G.78 |
| Buser, Membrane Positioning Instrument | G.82 |
| Buser, Periosteal Elevators | G.44 |

C

| | |
|---|-------------------|
| Caliper | G.91, H.11 |
| Cannula, Suction | G.08 - G.09 |
| Cannula for Water Syringe | G.10 |
| Cartridge Syringes | G.11 |
| Castroviejo, Caliper | G.91 |
| Castroviejo, Needle Holder, minimal invasive | G.53 |
| Castroviejo Genon, Needle Holder minimal invasive | G.53 |
| Cattoni, Scaler | C.06 |
| Cement Spatulas | B.09 |
| Cheek and Tongue Retractor | G.36 |
| Cheek Retractor | G.29 - G.33 |
| Cheek Retractor Mouth Mirrors | A.08 |
| Chisels | B.12 |
| Chisels, Bone | G.67 - G.68 |
| Chisels, Periodontal | F.04 |
| Chompret, Syndesmotomes | E.11 |
| Circular Knives | G.88 |
| Clasp Bending Pliers | H.18 |
| Cleaning Brush for Suction Cannulas | G.08 |
| Cleveland, Bone Cutting Forceps | G.59 |
| College, Anatomic Forceps | G.24 |
| College, Tweezers | A.04 - A.05 |
| College Modified, Tweezers | A.05 |
| Columbia Standard, Universal Curette | C.08 |
| Compactor Applicator | G.78 - G.81 |
| Compactor Planishers Filling Instruments | B.07 |
| Composite Instruments for Aesthetic Teeth Restoration ECI | B.08 |
| Container Systems | I.14 - I.16 |
| Cooley, Atraumatic Micro Forceps | G.20 |
| Copper Ring Removing Pliers | H.14 |
| Cp11, Periodontal Probe | A.03 |
| Cp11.5b (Who), Periodontal Probe | A.03 |
| Cp12, Periodontal Probe | A.03 |
| Cp15 North Carolina, Periodontal Probe | A.03 |
| Cp2, Periodontal Probe | A.03 |
| Cq-2N Nabers, Furcation Probe | A.03 |
| Crane-Kaplan, Periodontal Pocket Marking Tweezers | F.02 |
| Crane-Kaplan, Scaler | C.04 |
| Crile-Wood, Needle Holder | G.48 |
| Cross-Action Micro Dissecting Forceps | G.23 |
| Cross-Action Micro Needle Holder | G.54 |
| Cross Cut Files | G.39 |
| Crown Pliers | H.12 - H.13 |
| Crown Remover | H.06, H.09 - H.10 |
| Crown Scissors | H.04 |
| Crown Spreader | H.07 - H.08 |
| Crown Spreading Forceps | H.08 |
| Cryer, Extracting Forceps | G.51, G.53 |
| Curettes, Bone | G.38 |
| Curettes, Furcation | C.15 |
| Curettes, Gracey | C.10 - C.13 |
| Curettes, Periodontal | C.07 - C.15 |
| Curettes, Periodontal Surgical | F.04 |
| Curettes, Sinus-Lift | G.66 |
| Curettes, Titanium | F.07 |
| Curettes, Universal | C.07 - C.09 |
| Cylindrical Filling Instruments | B.09 - B.07 |

D

| | |
|------------------------------|------|
| Darby-Perry, Scaler | C.06 |
| Dean, Operating Scissors | G.15 |
| De Bakey, Needle Holder | G.49 |
| De Bakey, Vascular Forceps | G.23 |
| Denhart, Mouth Gags | G.35 |
| Deppeler, Scaler | C.04 |
| Depth Gauge | G.90 |
| Derf, Needle Holder | G.48 |
| Desmotom | E.04 |
| Dev-Aspir 1 Suction System | G.09 |
| Dev-Aspir 2 Suction System | G.09 |
| Dev-Lux Instruments | E.02 |
| devemed # F, Crown Remover | G.58 |
| devemed # S, Crown Remover | G.58 |
| devemed, Bone Rongeur | G.58 |

Index

| | |
|--|-------------|
| devedmed , Crown Remover..... | H.06 |
| devedmed , Retractor..... | G.30 - G.31 |
| De Wijs , Periosteal Elevator..... | G.43 |
| De Wijs Mod. , Periosteal Elevator..... | G.44 |
| Distal End Cutter..... | H.17 |
| Dressing and Cotton Swab Forceps..... | G.27 |
| Dykal-Instrument..... | B.03 |

E

| | |
|--|-------------|
| ECI Composite Instruments for Aesthetic Teeth Restoration..... | B.08 |
| ECI Flat Plugger..... | B.08 |
| ECI Occlusion Former..... | B.08 |
| ECI Spatula..... | B.08 |
| Ehricke , Impression Trays..... | H.02 - H.03 |
| Elevators, Sinus-Lift..... | G.64 - G.65 |
| Emos , Retractor..... | G.30 |
| Endodontic Probes..... | D.04 |
| Endodontic Set..... | D.05 |
| Endodontic Tweezers..... | D.03 |
| Esthetic , Crown Spreader..... | H.08 |
| Exkavators..... | B.02, D.02 |
| Extract 1100, Extracting Forceps..... | E.40 - E.43 |
| Extract 1200, Extracting Forceps..... | E.44 - E.45 |
| Extract 400, Extracting Forceps..... | E.32 - E.35 |
| Extract 500 & 650, Extracting Forceps..... | E.36 - E.39 |
| Extracting Forceps, American-Extract..... | E.49 - E.53 |
| Extracting Forceps, Extract 1100..... | E.40 - E.43 |
| Extracting Forceps, Extract 1200..... | E.44 - E.45 |
| Extracting Forceps, Extract 400..... | E.32 - E.35 |
| Extracting Forceps, Extract 500 & 650..... | E.36 - E.39 |
| Extracting Forceps, GES LINE..... | E.24 - E.27 |
| Extracting Forceps, GES LINE KIDS..... | E.28 - E.29 |
| Extracting Forceps, Kids-Extract..... | E.54 - E.57 |
| Extracting Forceps, Special Extract..... | E.46 - E.49 |
| Extracting Forceps..... | E.24 - E.57 |

F

| | |
|---|-------------------|
| Farabeuf , Retractor..... | G.31 |
| Filling Instruments Ball-Shaped..... | B.04 |
| Filling Instruments Ball-Shaped Dykal..... | B.03 |
| Filling Instruments Compactor Planishers..... | B.07 |
| Filling Instruments Cylindrical..... | B.06 - B.07 |
| Filling Instruments Pear-Shaped..... | B.09 |
| Filling Instruments Spatulated..... | B.05 |
| Filter, Paper..... | I.16 |
| Filter, Permanent..... | I.16 |
| FLAP DESIGN SET..... | G.56 - G.57 |
| Flat Nose Pliers..... | H.18 |
| Flat Plugger Applicator..... | G.79 |
| Forceps, Anatomic..... | G.24 |
| Forceps, Articulating Paper..... | B.13, H.05 |
| Forceps, Atraumatic Micro..... | G.21 |
| Forceps, Bone Cutting..... | G.59 |
| Forceps, Bone..... | G.55 |
| Forceps, Bone Holding..... | G.89 |
| Forceps, Bone Ring Holding..... | G.89 |
| Forceps, Cotton Swab..... | G.27 |
| Forceps, Crown Spreading..... | H.08 |
| Forceps, Dressing..... | G.27 |
| Forceps, Extracting..... | E.24 - E.57 |
| Forceps, Holding, for Screw-Drivers..... | G.88, H.05 |
| Forceps, Holding, Zalex..... | E.31 |
| Forceps, Inlay..... | H.13 |
| Forceps, Lifting, Zalex..... | E.31 |
| Forceps, Micro..... | G.19 - G.20 |
| Forceps, Removing for Temporary Inserts..... | H.08 |
| Forceps, Rubber Dam..... | B.13 |
| Forceps, Soft Tissue..... | G.55 |
| Forceps, Surgical..... | G.20 - G.23, G.25 |
| Forceps, Suture and Membrane..... | G.22 |
| Forceps, Tissue..... | G.20 |
| Forceps, Tissue Grasping..... | G.89 |
| Forceps, Tongue Holding..... | G.34 |
| Forceps, Vascular..... | G.23 |
| Frahm , Modelling Instrument..... | B.11 |

| | |
|--|------|
| Freer , Periosteal Elevator..... | G.41 |
| Friedmann , Bone Rongeur..... | G.58 |
| Furcation Currettes..... | C.15 |
| Furcation Probes..... | A.03 |
| Furrer , Copper Ring Removal Forceps..... | H.14 |

G

| | |
|--|-------------------|
| Gauze Packers..... | H.05 |
| GES LINE GENTLE EXTRACT SYSTEM..... | E.24 - E.26 |
| Gingival Cord Packers..... | B.03 |
| Gingival Margin Trimmers..... | B.12 |
| Gingival Retractor..... | B.03 |
| Gingivectomy Knives..... | F.02 |
| Glass Mixing Plate..... | B.14 |
| Glickmann , Periosteal Elevator..... | G.42 |
| Goldman-Fox , Gingival Retractor..... | B.03 |
| Goldman-Fox , Operating Scissors..... | G.14 |
| Goldman-Fox , Scaler..... | C.05 |
| Goldman-Fox Standard , Curette..... | C.14 |
| GPM Container, Aluminium..... | I.14 - I.16 |
| GPM Paper Filter..... | I.16 |
| GPM Permanent Filter..... | I.16 |
| GPM Stainless Steel Trays..... | I.08, I.11 - I.13 |
| GPM Stainless Steel Tray System..... | I.03 - I.13 |
| Gracey , Currettes..... | C.11 |
| Gracey „Large“ , Currettes..... | C.12 |
| Gracey „Small“ , Currettes..... | C.13 |
| Gracey „Standard“ , Currettes..... | C.11 |
| Granulate Plugger..... | G.80 |
| Grinding Stone..... | C.16 |
| Gross-Maier , Dressing and Cotton Swab Forceps..... | G.27 |
| Gutta-Percha Applicator..... | D.04 |

H

| | |
|---|------------|
| Haemostatic Forceps..... | G.26 |
| Halsey , Needle Holder..... | G.49 |
| Halstead-Mosquito , Haemostatic Forceps..... | G.26 |
| Hand and Nail Brush..... | I.17 |
| Handle, Palti..... | G.77 |
| Handle for Photographic Mirror..... | G.37 |
| Heidbrink , Root Fragment Elevator..... | E.23 |
| Heidemann , Filling Instrument..... | B.05 |
| Hemingway , Sharp Spoons..... | G.39 |
| Hirschfeld , Periosteal Elevator..... | G.41 |
| Hock Chisels..... | C.06 |
| Holding Forceps for Screw Drivers..... | G.88, H.05 |
| Hollenback , Modelling Instrument..... | B.11 |
| How , Band Pinching and Ligature Locking Pliers..... | H.17 |
| Howard , Bone File..... | G.40 |
| Hygienist , Scaler..... | C.02 |
| Hygienist-Jacquette , Scaler..... | C.02 |
| Hygienist-Towner , Scaler..... | C.02 |
| Hösel , Needle Holder..... | G.51 |

I

| | |
|--|-------------|
| Identification Label..... | I.16 |
| Iglhaut , Micro Periosteal Elevator..... | G.44 |
| Implant Spatula..... | G.45 |
| IMPLANT MAINTENANCE SET..... | F.06 |
| Impression Trays Ehricke..... | H.02 - H.03 |
| India , Grinding Stone..... | C.16 |
| Indiana-Univ. Standard , Universal Curette..... | C.08 |
| Inlay Forceps..... | H.13 |
| Insert for Crown Remover..... | H.10 |
| Instrument Maintenance Oil..... | I.17 |
| Instruments for Crown Work..... | H.08, H.13 |
| Instruments for Tunnel Technique..... | F.08 - F.09 |
| Interdental Periosteal Elevator..... | G.45 |
| Interdental Scaler..... | C.06 |
| Iris , Operating Scissors..... | G.13 |
| Ivory , Bone Carrier..... | B.12 |
| Ivory , Scaler..... | C.04 |

Index

J

| | |
|--|------|
| Jacquette , Scaler | C.03 |
| Joseph , Operating Scissors | G.16 |
| Jovanovic , Granulate Plugger | G.80 |

K

| | |
|---|-------------|
| Kelly , Operating Scissors..... | G.15 |
| Key for Crown Remover | H.09 - H.10 |
| Khoury , Periosteal Elevator | G.43 |
| Kids-Extract..... | E.54 - E.57 |
| Kids Extracting Forceps | E.29 |
| Kids GES LINE Gentle Extract System..... | E.28 |
| Kirkland , Gingivectomy Knife..... | F.02 |
| Kirkland , Periodontal Chisels | F.05 |
| Klein , Extracting Forceps..... | E.54 - E.56 |
| KLING-EX , Scalpel Blade Remover..... | G.06 |
| KLING-EX , Scalpel Blade Removing Forceps..... | G.05 |
| Kocher , Haemostatic Forceps..... | G.26 |
| Kocher , Retractor | G.28 |
| Kocher-Langenbeck , Retractor..... | G.28 |
| Kocher-Langenbeck Mod. , Retractor | G.29 |
| Kopp , Root Elevator..... | E.15 |
| Kramer-Nevins , Bone Chisel..... | G.67 |
| Kuhhorn , Probe..... | A.02 |

L

| | |
|---|-------------|
| Labels with Indicator | I.16 |
| La Grange , Operating Scissors | G.14 |
| Langenbeck , Retractor | G.28 |
| Langer Standard , Titanium Curettes..... | F.07 |
| Langer Standard , Universal Curettes | C.07 |
| Langer Universal Curettes | C.07 |
| Lichtenberg-Ryder , Needle Holder | G.50 |
| Ligature Locking Pliers | H.17 |
| Ligature Scissors..... | G.17 |
| Lip Retractors..... | G.33 - G.34 |
| Locklin , Operating Scissors | G.16 |
| London-College , Endodontic-Tweezers | D.03 |
| Lucas , Bone Curette..... | G.38 |
| Luer , Bone Rongeur..... | G.58 |
| Luniatschek , Gauze Packers | H.05 |

M

| | |
|--|-------------|
| Mac Farlane , Scaler | C.05 |
| Maintainance Oil..... | I.17 |
| Mallets | G.71 |
| Mathieu , Needle Holder..... | G.51 |
| Mathieu-Olsen , Needle Holder | G.51 |
| Matrix Forceps with Tungsten Carbide Inserts..... | B.13 |
| Mayo-Hegar , Needle Holder | G.48 |
| Mazor , Depth Gauge | G.90 |
| Mazor , Flat Plugger / Applicator..... | G.79 |
| Mazor , Membrane Lift / Granulate Plugger | G.79 |
| Mazor , Periosteal Elevator..... | G.44 |
| Mazor , Sinus-Lift Elevator | G.64 |
| Mazor , Universal Instrument..... | G.80 |
| Mazor Mod. , Periosteal Elevator | G.44 |
| Mazouch , Crown Spreader..... | H.07 |
| Mccall , Scaler | C.05 |
| Mccall Standard , Universal Curette | C.09 |
| Mc Kellops , Bar Bending Pliers..... | H.18 |
| Mckesson , Suction Cannula | G.35 |
| Mead , Mallet | G.71 |
| Measuring Instruments..... | G.91 - H.11 |
| Mega , Mirrow | A.07 |
| Megaduo , Mirrow | A.07 |
| Membrane Lift / Granulate Plugger | G.79 |
| Membrane Positioning Instrument | G.82 |
| Meriam , Anatomic Forceps | G.24 |
| Meriam , Tweezers..... | A.05 |
| Meriam-Diamant , Tweezers..... | A.05 |
| Metal Guides for Silicone Band..... | I.10 |
| Metzenbaum , Operating Scissors..... | G.15 |

| | |
|---|-------------------|
| Micro Blade Holder..... | G.07 |
| Micro Dissecting Forceps..... | G.20 - G.23 |
| Micro Forceps | G.20 |
| Micro Mouth Mirrors..... | D.03 |
| Micro Needle Holder | G.52 - G.54 |
| Micro Periosteal Elevator..... | G.44 |
| Micro Plugger | D.04 |
| Micro Resection Mirrors | D.03 |
| Micro Scalpel Blades..... | G.07 |
| Middeldorpf , Cheek and Lip Retractors..... | G.29 |
| Miller , Articulating Paper Forceps | B.13, H.04 |
| Miller , Bone File | G.40 |
| Miller-Colburn , Bone Curette..... | G.38 |
| Miller-Colburn , Bone File | G.39 |
| Mini Dental Caliper | G.91 |
| Mini Periolut | E.07 |
| Mini Friedmann Mod. , Bone Rongeur | G.58 |
| Minimal invasive Needle Holders | G.53 |
| Modelling Instruments..... | B.11 |
| Molt , Bone Curette..... | G.38 |
| Molt , Mouth Gags | G.35 |
| Molt , Periosteal Elevator..... | G.42 |
| Molt Mod. , Mouth Gags | G.35 |
| Mortonson , Filling Instrument Compactor | B.07 |
| Mouth Gags..... | G.35 |
| Mouth Mirror Handles..... | A.06, D.02 - D.03 |
| Mouth Mirrors | A.07 - A.08 |
| Mucosal Flap Preparers | F.08 |
| Mucosa Punches – Circular Knives..... | G.88 |
| Müller , Retractor..... | G.29 |

N

| | |
|--|-------------|
| Needle Holder, Micro..... | G.52 - G.54 |
| Needle Holder, minimal invasive..... | G.53 - G.53 |
| Needle Holder..... | G.48 - G.54 |
| Nerve-Broaches pliers..... | D.06 |
| Neumann , Operating Scissors | G.17 |
| Nevius , Extracting Forceps..... | E.50 |
| North Carolina , Periodontal Probe..... | A.03 |

O

| | |
|---|-------------|
| Obwegeser , Retractor | G.28 |
| Occlusion Former ECI | B.08 |
| Occlusion Trays | H.03 |
| Ochsenbein , Periodontal Chisels | F.04 |
| Oil for Instruments..... | I.17 |
| Oil for Sharpening Stones | C.17 |
| Ointment Cannula | G.10 |
| Old Dominion University , Probe..... | A.02 |
| Old Dominion University Large , Probe..... | A.02 |
| Olsen-Hegar , Needle Holder..... | G.49 |
| Operating Scissors..... | G.13 - G.17 |
| Orban , Gingivectomy Knife..... | F.02 |
| Osteotoms and Bone Pusher..... | G.76 |
| Osteotom and Bone Pusher Tips | G.77 |

P

| | |
|--|-------------|
| Palti , Applicator..... | G.78 |
| Palti , Bone Ring Holding Forceps..... | G.89 |
| Palti , Depth Gauge..... | G.90 |
| Palti , Handle..... | G.77 |
| Palti , Interdental Periosteal Elevator | G.45 |
| Palti , Periosteal Elevator | G.41, G.44 |
| Palti , Sinus-Lift Curettes | G.66 |
| Palti / Molt , Periosteal Elevator | G.43 |
| Paper Filter | I.16 |
| Papilla Elevator..... | G.45 |
| Parmly , Extracting Forceps | E.49 |
| Pean , Haemostatic Forceps..... | G.26 |
| Pear-Shaped Filling Instruments | B.09 |
| Peet , Splinter Instrument | D.08 |
| Periodontal Chisels | F.04 - F.05 |
| Periodontal Files..... | F.03 |
| Periodontal Periosteal Elevator | F.05 |

Index

| | |
|---|-------------|
| Periodontal Pocket Marking Tweezers..... | F.02 |
| Periodontal Probe..... | A.03 - A.04 |
| Periodontal Probe Tips..... | F.07 |
| Periodontal Surgical Curettes..... | F.04 |
| Periolux..... | E.06 - E.10 |
| PERIO PLASTIC SURGERY SET..... | F.10 - F.11 |
| Periosteal Elevators..... | G.41 - G.45 |
| Periosteal Elevators PPS..... | F.08 - F.09 |
| Periotoms..... | E.10 |
| Photographic Mirror..... | G.37 |
| Pin Holding and Nerve-Broaches Pliers..... | D.06 |
| Pivot-Point Instruments..... | E.12 - E.14 |
| Planators..... | B.07 |
| Plastic Inserts for Mallet..... | G.71 |
| Plastic Test Sticks..... | C.17 |
| Plate Curette..... | D.04 |
| Pliers, Band Pinching..... | H.17 |
| Pliers, Bar Bending..... | H.18 |
| Pliers, Bending, for Plates..... | H.17 |
| Pliers, Clasp Bending..... | H.18 |
| Pliers, Copper Ring Removing..... | H.14 |
| Pliers, Crown..... | H.11 - H.13 |
| Pliers, Flat..... | H.18 |
| Pliers, Ligature Locking..... | H.17 |
| Pliers, Nerv-Broaches..... | D.06 |
| Pliers, Pin Bending..... | H.18 |
| Pliers, Pin Holding..... | D.06 |
| Pliers, Technical..... | H.18 |
| Pliers, Telescopic Crown..... | H.11 |
| Pliers, Universal Crampon..... | H.18 |
| Pliers, Wire..... | H.18 |
| Pliers, Wire Seizing..... | H.12 |
| Plugger Pusher..... | G.81 |
| PPS Gingivectomy Knife..... | F.08 |
| PPS Instruments for Tunnel Technique..... | F.08 - F.09 |
| PPS Mucosal Flap Preparer..... | F.08 |
| PPS Perio Plastic Surgery Set..... | F.10 - F.11 |
| PPS Periosteal Elevator..... | F.08 - F.09 |
| PPS Tissue Graft Plug..... | F.09 |
| PPSU Holder..... | I.10 |
| Pressure Syringe..... | G.10 |
| Prichard , Periodontal Surgical Curette..... | F.04 |
| Prichard , Periosteal Elevator..... | G.43 |
| Probes..... | A.02, D.02 |
| PROPHYLAXIS SET..... | C.18 |
| Prosthesis Elevator..... | H.06 |
| Protective Covers for Mouth Gags..... | G.35 |

Q

| | |
|---|------|
| Quetin , Curette..... | C.15 |
| Quetin , Furcations Curette..... | C.15 |

R

| | |
|--|-------------|
| Rack for Mucoas Punches..... | G.88 |
| Rack for Trephine Burrs..... | G.85 |
| Racks..... | G.85, G.88 |
| Ralk , Splinter Instrument..... | D.07 |
| Ramus-Obwegeser , Retractor..... | G.29 |
| Removing Forceps for Temporary Inserts..... | H.08 |
| Replacement Silicone Plate..... | H.13 |
| Retractor..... | G.28 - G.33 |
| Rhodes , Periodontal Chisels..... | F.05 |
| RIDGE SPLITTING SET..... | G.72 |
| Root Elevators..... | E.15 - E.22 |
| Root Fragment Elevator..... | E.23 |
| Rubber Dam Instrument..... | B.13 |
| Rubber Insert for Crown Pliers..... | H.13 |
| Rubber Insert for Tongue Holding Forceps..... | G.34 |
| Ruskin , Bone Rongeur..... | G.59 |
| Ruskin-Liston , Bone Cutting Forceps..... | G.59 |

S

| | |
|--|-------------|
| Safety Seal..... | I.16 |
| Salama , PPS Gingivectomy Knife..... | F.08 |
| Salama , PPS Mucosal Flap Preparer..... | F.08 |
| Salama , PPS Periosteal Elevator..... | F.08 - F.09 |
| Salama , PPS Tissue Graft Plug..... | F.09 |
| Scaler..... | C.05 - C.06 |
| Scaler File..... | C.06 |
| Scaler Hock..... | C.06 |
| Scaler Sickle Scalers..... | C.02 - C.04 |
| Scalpel Blade Remover..... | G.06 |
| Scalpel Blade Removing Forceps..... | G.05 |
| Scalpel Blades..... | G.06 |
| Scalpel Bladeholder..... | G.04 - G.05 |
| Scheufele , Occlusion Trays..... | H.03 |
| Schluger , Periodontal File..... | F.03 |
| Schmelzeisen/Cooley , Atraumatic Micro Forceps..... | G.21 |
| Schäfershaken , Probe..... | D.02 |
| Scissors, Ligature..... | G.17 |
| Scissors, minimal invasive..... | G.18 |
| Scissors, Operating..... | G.12 - G.1 |
| Scissors with fine Tower Point..... | G.13 |
| Semken , Surgical Forceps..... | G.25 |
| Separating Plate..... | I.10 |
| Set Of Trephine Burrs..... | G.87 |
| SET AUGMENTATION, ALL IN ONE..... | G.86 - G.87 |
| SET BASIC IMPLANTOLOGY SURGICAL..... | G.02 - G.03 |
| SET BONE CONDENSING..... | G.74 - G.75 |
| SET BONE SPREADING..... | G.69 |
| SET FLAP DESIGN..... | G.57 |
| SET IMPLANT MAINTENANCE SET..... | F.06 |
| SET PERIO PLASTIC SURGERY..... | F.10 - F.11 |
| SET PROPHYLAXIS..... | C.18 |
| SET RESTORATIVE..... | B.15 |
| SET RIDGE SPLITTING..... | G.72 |
| SET SINUS LIFT, LARGE..... | G.62 - G.63 |
| SET SINUS LIFT, SMALL..... | G.61 |
| SET SOCKET PRESERVATION..... | G.47 |
| SET TREPHINE BURRS..... | G.85 |
| SET TUNNEL TECHNIQUE..... | F.12 |
| SET ZALEX® START UP..... | E.30 - E.31 |
| Sharpening Stone Oil..... | C.17 |
| Sharp Spoons..... | G.39 |
| Sicard , Periosteal Elevator..... | G.41 |
| Side Cutter..... | H.15 |
| Silicone Band for Tray Cover..... | I.10 |
| Silicone Filter..... | G.83 |
| Silicon Inserts for Mouth Gags..... | G.35 |
| Simas , Depth Gauge..... | G.90 |
| Sinus-Lift Curettes..... | G.66 |
| Sinus-Lift Elevators..... | G.64 - G.65 |
| SINUS-LIFT SET – Large..... | G.62 - G.63 |
| SINUS-LIFT SET – Small..... | G.61 |
| Sinus Curettes..... | G.87 |
| Socket Preservation Set..... | G.47 |
| Soft Tissue and Bone Forceps..... | G.55 |
| Sorter for Identification Labels..... | I.16 |
| Spatulated Filling Instruments..... | B.05 |
| Spatula ECI..... | B.08 |
| Special , Universal Curette..... | C.09 |
| Special Extract, Extracting Forceps..... | E.46 - E.48 |
| Special Oil Spray..... | I.17 |
| Spencer , Ligature Scissors..... | G.17 |
| Spherical Plugger Spatula..... | D.04 |
| Splinter Instruments..... | D.07 - D.08 |
| Spoon Curette..... | D.04 |
| Spoons, Sharp..... | G.39 |
| Sprengel , Modelling Instrument..... | B.11 |
| Standard Mouth Mirrors..... | A.08 |
| Steigmann , Bone Scraper..... | G.78 |
| Steigmann , Compactor Applicator..... | G.78 |
| Sternberg , Lip Retractors..... | G.33 |
| Stieglitz , Splinter Instrument..... | D.08 |
| Stolley , Impression Trays..... | H.03 |
| Straight Cut Files..... | G.40 |
| Suction and Irrigation Cannula..... | G.10 |

| | |
|---|-------------------|
| Suction Cannula..... | G.08 - G.09 |
| Suction Cannula Tip..... | G.09, G.82 - G.83 |
| Suction Tube..... | G.09 |
| Sugerman , Periodontal File..... | F.03 |
| Surgery Drape..... | I.16 |
| Surgical Forceps..... | G.20 - G.23, G.25 |
| Suture and Membrane Forceps..... | G.22 |
| Swann Morton (Sm) , Scalpel Blade..... | G.06 - G.07 |
| SWE Periolut..... | E.08 |
| Syndesmotomes..... | E.11 |
| Syringes..... | G.11 |

T

| | |
|--|-------------|
| Tatum/Palti , Sinus-Lift Curette..... | G.66 |
| Taylor , Scaler..... | C.04 |
| Team-Atlanta , Suture and Membrane Forceps..... | G.22 |
| Technical Pliers..... | H.17 - H.18 |
| Teflon® Racks..... | I.09 |
| Teflon® Holder..... | I.10 |
| Teflon® Pusher..... | I.10 |
| Telescopic Crown Pliers..... | H.11 |
| Three-In-One Cheek and Tongue Retractor with Saliva Ejector..... | G.36 |
| Tissue Graft Plug..... | F.09 |
| Tissue Grasping Forceps..... | G.89 |
| Titanium Curettes..... | F.07 |
| Titanium Photographic Mirror..... | G.37 |
| Titan Periodontal Probe Tips..... | F.07 |
| Tongue Depressor..... | G.34 |
| Tongue Holding Forceps..... | G.34 |
| Tongue Retractor..... | G.32 |
| Tongue Spatula..... | G.34 |
| Towel Clamps..... | G.27 |
| Towner-Jacquette , Scaler..... | C.02 |
| Trays and Accessories..... | I.05 - I.14 |
| Trephine Burrs with Internal Cooling..... | G.85 |
| Treyman Mod. , Crown Remover..... | H.06 |
| TUNNEL TECHNIQUE SET..... | F.12 |

U

| | |
|---|-------------|
| Universal , Pin Bending Pliers..... | H.18 |
| Universal Curettes..... | C.08 - C.09 |
| Universal Fixation Tweezers..... | D.06 - G.21 |
| Universal Instrument..... | G.80 |
| University Of Cawood-Minnesota , Retractor..... | G.32 |
| University Of Cawood-Minnesota Modified , Retractor..... | G.32 |

V

| | |
|-----------------------------------|------|
| Vascular Forceps..... | G.23 |
| Vickers , Mallet..... | G.71 |
| Vivas , Bone Spreader..... | G.68 |

W

| | |
|--|-------------|
| Wachtel , Papilla Elevator..... | G.45 |
| Wash Trays, Stainless Steel..... | I.05 - I.14 |
| Water Syringes & Syringes..... | G.10 |
| Wedelstaedt , Chisels..... | B.12 |
| Weingart , Crown Pliers/Wire Seizing and Ligature Locking Pliers..... | H.12 |
| Westcott , Burnisher..... | B.10 |
| White , Burnisher..... | B.10 |
| Who , Periodontal Probe..... | A.04 |
| Wieder , Tongue Depressor..... | G.34 |
| Wiland , Modelling Instrument..... | B.11 |
| Williger , Periosteal Elevator..... | G.41 |
| Winter , Root Elevator..... | E.20 |
| Wire and Clasp Bending Pliers..... | H.18 |
| Wire Cutter, Front Action..... | H.16 |
| Wire Cutter..... | H.16 |
| Wire Seizing and Ligature Locking Pliers..... | H.18 |
| Witzel , Splinter Instrument..... | D.07 |
| Wynmann , Removing Forceps for Temporary Inserts..... | H.08 |

Y

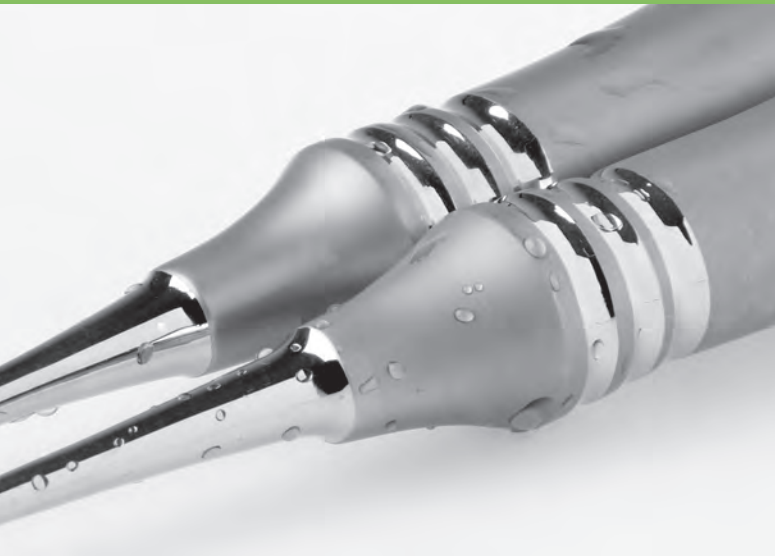
| | |
|---|------|
| Young , Tongue Holding Forceps..... | G.34 |
| Younger-Good Standard , Universal Curette..... | C.09 |

Z

| | |
|--|------|
| ZALEX , Holding Forceps..... | E.31 |
| ZALEX , Lifting-Forceps..... | E.31 |
| ZALEX , Self-Adhesive Absorption Pads..... | E.31 |
| ZALEX® START UP SET in Accordance to Dr. Helke..... | E.31 |

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