

The compact centrifugal casting machine with induction melting and integrated power cooling

- Compact benchtop casting machine with very high-performance induction heating guarantees short melting cycles, minimises oxidation and thus facilitates subsequent finishing
- Compact dimensions and design give the new Fornax® T a very small footprint
- · Integrated power cooling enables more than 50 casts in a row, even with high ambient temperatures
- Casting temperature of up to 1550°C: ideal for all commercially available dental alloys (except titanium)
- Quick adjustment to different sizes of casting rings using a simple mechanism
- · Very quick adjustment to different casting mould sizes by means of a simple mechanism ensures fast working



Fornax® T – From dental technicians for dental technicians

In consultation with our customers it was our main aim to further optimise our compact centrifugal casting machine Fornax® T systematically while maintaining the Fornax® T concept that has been used successfully worldwide.

The design - ergonomic and compact

The Fornax® T functions have been valued by users for many years: an ergonomic, compact centrifugal casting machine with all the technical and economic advantages.

The advantages for you:

- The low overall height means that a lower workbench is no longer required
- There is sufficient space on any laboratory bench for the unit
- A standard, fused Schuko socket is all that is required to make the machine immediately ready to use

The technology in detail

Fornax® T has a very powerful induction heating of 3,600 VA.

The advantages for you:


- Rapid heating and melting of all commercially available precious and non-precious alloys (except titanium)
- Sufficient power reserves even with critical alloys
- Oxidation is minimised to the absolute necessary due to rapid melting to the casting temperature and facilitates trimming at a later stag


BEGO has developed a special crucible safety base for optimal protection of your machine during the casting procedure.

The advantages for you:

- If the crucible fails, the molten metal will be safely collected in the ceramic safety base
- This effectively prevents any serious damage to the machine

In addition to timing the casting point, the multi-functional display also shows the service notification and enables help to be obtained quickly when servicing is required.

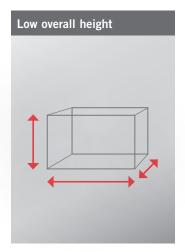

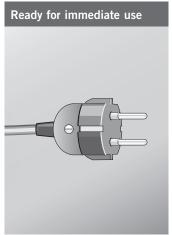



Fornax® T – ergonomically compact with all the technical advantages

Clear display

Integrated power cooling system

The Fornax® boasts integrated power cooling.

The advantages for you:

- More than 50 casts in a row are possible, even if ambient temperatures are high
- Does not require a water supply or water outlet can be used immediately anywhere
- No water consumption, no water damage and no reliance on local water quality
- Furthermore, the costs of installing an expensive water system are saved
- Since the unit does not require a water supply, condensation forming on the crucible coil in the winter months is no longer a concern; "explosions", mould cracks and the resulting flawed casts are a thing of the past



Simple handling

The counterweight is very easily set using an adjustment mechanism with a toothed segment that has predefined markings for different sizes of casting rings.

The advantages for you:

- Adjustment to the correct size of casting ring in a matter of seconds ensures effective operation
- The tare unit is self locking and does not have to be fixed in position separately

Constant further development of the ceramic crucibles ensures optimal castings

The ceramic crucibles for Fornax® casting machines in original BEGO quality provide you with **many advantages:**

- Excellent thermal shock resistance up to 1,500°C, which minimises the risk of splitting
- Special geometry ensures that alloys flow optimally
- The integrated crucible protection reduces casting residue in the crucible for cost-effective casting
- The smooth, high-grade surfaces also emphasise the high quality of the crucible and provide it with a perfect finish





Fornax® crucibles

 $C \in$

Fornax® T	
Technical data	
Height	450 mm
Height with the lid open	965 mm
• Width	650 mm
• Depth	625 mm
Depth with the lid open	640 mm
Rated voltage	230 V, 50/60 Hz
Special voltage	200 – 240 V, 50/60 Hz
Power consumption	approx. 16 A
Induction melting power	3.6 kVA, 65 kHz
• Weight	approx. 70 kg

Products and accessories					
Products		Presentation	Contents	REF	
• Fornax® T 230 V, 50/60 Hz				26425	
Ceramic crucible		1 pack	6	52482	
Graphite inserts		1 pack	6	52454	
Ceramic inserts for ceramic crucible		1 pack	6	52455	
• Crucible former, size 3, 6 and 9			1 of each		
Accessories					
Crucible former	Size 3 Size 6 Size 9	1 set 1 set 1 set	4 4 4	52627 52628 52629	
Furnace tongs, 64 cm long		1	11599		
• Furnace tongs, 55 cm long		1	39754		
Fornax® holder for 4 crucibles and 1 x tongs			1	25337	
Wiromelt flux (for non-precious metal)		80 g container	1	52526	
Auromelt HF flux		65 g dispenser	1	52525	

We reserve the right to make changes in the design, pack contents and composition. Statements and recommendations on technique are based on our experience and tests and should be regarded as guidelines.

