



SUPRAGINGIVAL SODIUM BICARBONATE

(PROPHYLAXIS POWDER) Instructions

1 PRODUCT NAME

SUPRAGINGIVAL SODIUM BICARBONATE (PROPHYLAXIS POWDER)

2 INDICATIONS

- 1) Used in the oral cavity to grind and polish the dental tissue or prosthesis to make its surface smooth and even.
- 2) The powders can be used with polishing devices.

3 COMPOSITION

This product is white powder, sodium bicarbonate content > 90%.

Composition: sodium bicarbonate, silicon dioxide, edible essence.

4 PRECAUTIONS FOR USE

Before using the powder, we suggest you establish the general state of health of your patient.

- 1) The patient should wear goggles.
- 2) The practitioner should wear appropriate protective accessories (goggles, masks, gloves).
- 3) Use the high-speed evacuator of your dental unit.
- 4) The patient should rinse the mouth using an antiseptic solution before and after treatment.
- 5) Apply Vaseline® to the lips before polishing.
- 6) Wash patient face after use (e.g. with a wet wipes).
- 7) Do not use the powder beyond the maximum amount specified in the instrument.
- 8) In the case of accidental contact with the skin or eyes, rinse thoroughly with water.
- 9) Do not swallow.
- 10) The powder expired treatment according to local regulations for disposal and destruction, can not be abandoned in drains or rivers, and can not be mixed with household garbage discarded.
- 11) Except for contraindications, this product is suitable for adults and children who need dental treatment.
- 12) The product is disposable and cannot be reused.

5 CONTRAINDICATIONS

- 1) Patients suffering from upper respiratory tract infections, from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.
- 2) Predisposed persons may be sensitive to Chlorhexidin. If allergic reactions are observed, stop using the product and completely remove the powder from mouth.

3) The treatment of deep periodontal pockets can cause bacteraemia. Please apply appropriate restrictions for the treatment of risk patients (endocarditis, pregnancy, breast feeding, contagious disease), immune deficiency (neutropenia, agranulocytosis, diabetes, hemophilia), patients under treatment (radiotherapy, chemotherapy, antibiotics).

4) Do not direct the jet of powder toward fillings, crowns or bridgework as this could damage these restorations.

5) Patients suffering from severe periodontal disease are advised to eliminate inflammation before polishing.

6 INSTRUCTIONS FOR USE

Follow the instructions for use delivered with your polisher. In general:

- 1) Maintain a <jet-surface> angle of between 45° and 60°.
- 2) Maintain a <spray-surface> distance of about 5 mm.
- 3) Make small circular movements on the tooth.
- 4) Do not spray on the same position for a long time.
- 5) Direct the jet towards the occlusal rim.

7 STORAGE AND SHELF-LIFE

- 1) The powder should not be stored near acid or heat sources.
- 2) Transport Conditions: Temperature not exceeding 40°C and relative humidity within 10% to 95%.
- 3) When stored at room temperature, cool, ventilated, dry, non-polluting, sealed and preserved, the powder has a shelf life of 2 years.
- 4) The powder should not be exposed to the air for a long time. Please use it up within 3 months after opening the bottle. The powder should not be stored more than one week inside the powder chamber.

8 DISPOSAL OF UNUSED POWDER

- 1) Do not discard in drains or rivers.
- 2) Do not discard with household waste.
- 3) Expired powder should be disposed of in accordance with local regulations.

9 EXPLANATIONS FOR SYMBOLS ON THE LABEL



Upper Temperature



Manufacturer



Consult instructions for use



Fragile



Lot number



Date of manufacture




Use-by date



Keep dry



Upward

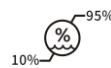

Humidity limitation



Shake well before use



Stack limit



Manufacturer:

URIT Medical Electronic Co., Ltd.

Address: No.4 East Alley, Jiuhua Road, Guilin, Guangxi 541001, P. R. China

Exclusive Distributor and After-Sales Service:

Guilin Veirun Medical Technology Co., Ltd

Address: No.D-07 Information Industry District, High-Tech Zone, Guilin, Guangxi 541004, P.R.China

Tel: +86-773-2260599 2260566

Web: www.veirun.com

V1.00(n)