

■ Direction for Use

1. Select the Accurate Guide of the appropriate size by visually checking the interference with the adjacent tooth of the implant placement site and appropriate gap.

2. Fasten up to the first step of Guide Drill in Accurate Guide.



3. Attach the fastened Accurate Guide and Guide Drill to the hand-piece.

4. Hand-piece set to 45 ~ 55 Ncm / 500 ~ 1200RPM


5. Check the position of the alveolar bone to be drilled and gently close the side of the Accurate Guide to the side of the adjacent tooth or Accurate Pin.




6. Use both hands to prevent deviation from the target point and path. (Hold the Accurate Guide with the other hand)



7. Begin drilling with irrigation.

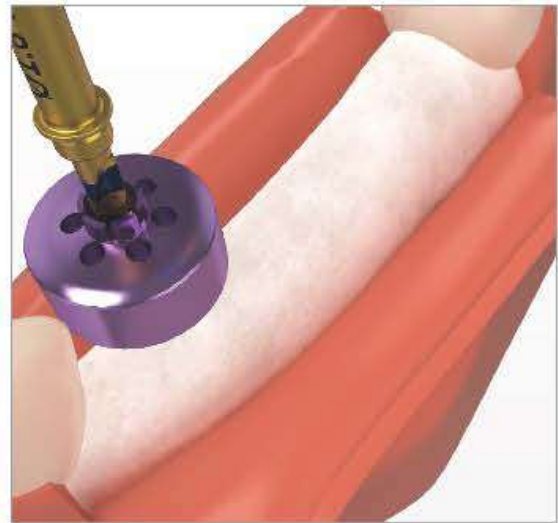

8. Insert Accurate Pin with the same size into the hole created after drilling.

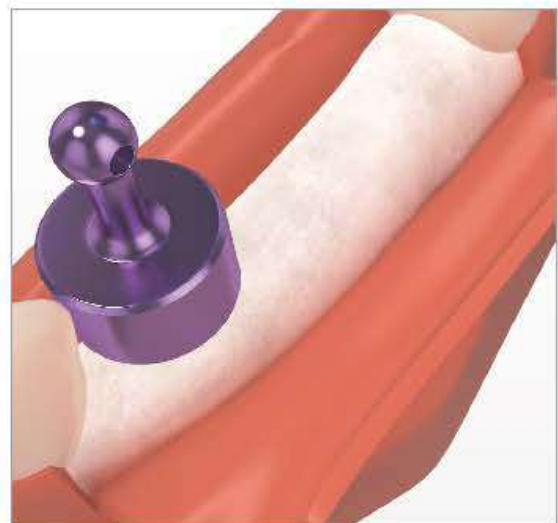

9. Repeat steps 1 through 7 as needed.

10. Be careful not to separate Accurate Guide and Guide Drill during use.