Hersteller: Kaltenbach & Voigt GmbH, Bismarckring 39, D-88400 Biberach, www.kavo.com

KaVo. Dental Excellence.

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
KaVo. Dental Excellence.

Intended use:


The PHYSIO Evo/Evo F/One is an operator stool for dentists in a dental practice in accordance with DIN EN ISO 7493. All versions of the PHYSIO operator stool are tested and approved in accordance with the currently applicable guidelines. All designs of the PHYSIO operator stool are compliant with DIN EN 1335 - 2 and 3, DIN 4550 and DIN EN ISO 7493.

Installation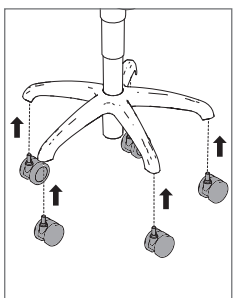


Mounting the castors

- ▶ Insert the castors into the stool base and press until the castors engage.
- ▶ Pull the castors lightly to ensure that they have securely engaged.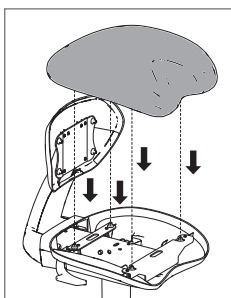

Mounting the upholstery


- ▶ Engage the seat upholstery at the 4 ball pins.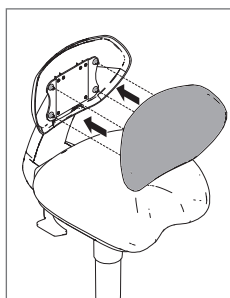

Mounting the backrest upholstery


- ▶ Engage the backrest upholstery at the 4 ball pins.

Operation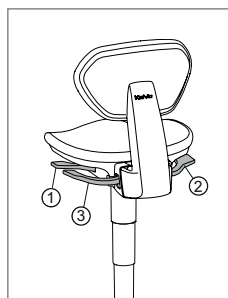


PHYSIO Evo

- ① Adjust the seat height
- ② Adjust the seat angle
- ③ Adjust the backrest angle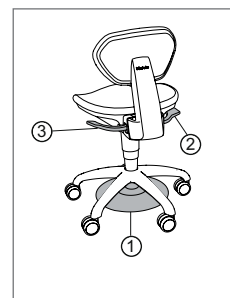

PHYSIO Evo F

- ① Adjust the seat height
- ② Adjust the seat angle
- ③ Adjust the backrest angle

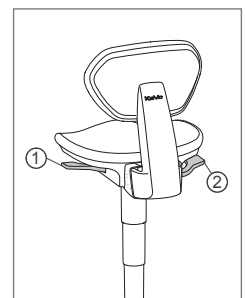PHYSIO One

- ① Adjust the seat height
- ② Adjust the backrest/seat angle

CAUTION

Damage due to working on the pressure vessel.

Forceful opening of the pressure vessel or extensive heat exposure can lead to injury and damage.

- ▶ Only instructed specialists may work on or exchange parts in the vicinity of the seat height adjusting element.

CAUTION

Screws are pre-tensioned.

Inappropriate loosening of screws on the support plate beneath the seat upholstery can lead to injury and damage.

- ▶ Only instructed specialists may dismantle screws from the support plate.

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KaVo Dental GmbH · D-88400 Biberach/Riß · Telefon +49 7351 56-0 · Fax +49 7351 56-1488 · www.kavo.com