User Manual

Osmo

Demineralizer
Demineralisierer
Demineralisateur
Demineralizzatore
Desmineralizador

Osmo

DEMINERALIZER

USER MANUAL

Dear Owner,

*Thank you for purchasing our demineralizer **Osmo**.*

The manufacturer is pleased to provide information concerning the use of this equipment herewith.

We remind you that in order to use the demineralizer correctly, it is necessary to read this manual carefully. All our products comply with the current safety norms and do not involve any risk for the operator when used according to the instructions provided.

Whilst wishing you every success in your work, please note that the reproduction of this manual is illegal and that, due to continuous research and technological development, the equipment specifications could change without prior notice.

W&H STERILIZATION S.r.l.

TABLE OF CONTENTS

INTRODUCTION	3
GENERAL WARNINGS	3
RESIN–SECURITY PRECAUTIONS.....	4
ACTIVE CARBON FILTERS – SAFETY WARNINGS*	4
INFORMATION ON THE WARRANTY.....	5
PACKAGE CONTENTS.....	6
GENERAL NOTES ON DELIVERY	6
PACKAGE SIZE AND WEIGHT.....	6
DELIVERY CONTENT	7
INSTALLATION AND USE OF THE DEMINERALIZER	8
SYMBOLS ON THE NAMEPLATE	8
SYSTEM COMPONENTS	8
PRELIMINARY INSTALLATION INSTRUCTIONS.....	9
INSTALLATION.....	10
PRIMING.....	12
EXAMPLES OF USAGE	13
USE AND OPERATION OF THE DEMINERALIZER	14
PRECAUTIONS TO AVOID WATER CONTAMINATION	14
IDENTIFICATION OF THE CARTRIDGES.....	14
REPLACEMENT OF FILTER CARTRIDGES.....	15
REPLACEMENT OF THE OSMOTIC MEMBRANE	16
MAINTENANCE	17
EXTERNAL CLEANING.....	17
MAINTENANCE OF THE WATER BLOCK® AND ONE-WAY VALVE	17
SPARE PARTS.....	18
TECHNICAL SPECIFICATIONS.....	20
MAINTENANCE ISTRUCTIONS FOR THE SAFETY VALVE (WATER BLOCK®).....	21

INTRODUCTION

This manual provides instructions for:

- Safe and efficient operation of the equipment
 - The correct installation
 - Continuous and scheduled maintenance.
-
- The dimensions mentioned in this manual are subject to change without notice.
 - Drawings and any other documents delivered with the equipment belong to the manufacturer, who reserves all rights; they cannot be made available to third parties.
 - The reproduction, either total or in part, of the texts and the illustrations in this manual is forbidden.

Note: If the Osmo demineralizer is used to supply sterilizers with demineralized water, it is imperative to monitor the water quality. If the supplied sterilizers do not feature a water conductivity sensor to monitor the water quality, we recommend using the manual water conductivity meter **Primo 5**.

GENERAL WARNINGS

- The equipment should be used in compliance with the procedures described in the manual and never for purposes other than those detailed in the manual.
- The user is responsible for installing, operating and servicing the equipment in accordance with the instructions listed in this manual.
- Only use genuine manufacturer resin cartridges dedicated for this product.
- **Do not drink** the water produced by Osmo.
- **Do not block or crush** the inlet or outlet pipes of the equipment or the sterilizer.
- If the **spray gun is not used** (manual filling) we recommend **closing the spiral tube's connecting tap**.
- If the equipment is not operated according to the instructions in this manual or is not properly maintained, the manufacturer cannot be held responsible for any fault, damage or malfunctioning of the equipment.
- If the water shows high quantities of turbidity, chlorine, iron and manganese, silicon, or a level of TDS (Total Dissolved Solids) above 2,000 ppm, the effectiveness and above all the life-span of the osmotic membrane will be significantly affected.
- Do not use the device with water over 20°C
- If the device is not used continuously it may emanate unpleasant smells.

Intended use: This manual is an integral part of the product and must be kept close to the equipment for easy and quick reference. The equipment is to be used only for the purpose for which it is designed.

Osmo: Equipment designed for the demineralization of tap water

RESIN–SECURITY PRECAUTIONS

	Contact Hazards	
	Eye contact	Irritation of the eyes (R36).
	Skin contact	Slight irritation of the skin.
	First Aid procedures	
	Eye contact	Flush immediately with large amounts of water and remove the resin beads.
	Skin contact	Take off the contaminated clothes. Remove the resin beads and flush the contaminated area with water.
	Precautions in case resin material is spilled	
	Precautions for People	Keep people away from the area of spillage. Resin beads on the floor may cause it to be slippery. Take care not to slip.
	Cleaning methods	Sweep the floor and transfer the resin beads to a plastic container for collection or disposal according to the appropriate guidelines.
	Notes for disposal	
The used product is a special non hazardous waste.		
The product must be disposed of in accordance with current local, regional and national regulations.		
The CER number for saturated or used ion-exchange resins used for the production of drinking or service water is: 19 09 05.		

ACTIVE CARBON FILTERS – SAFETY WARNINGS*

	Contact hazards	
	Eye contact	Possible temporary irritation due to the abrasive effect of the granules.
	Skin contact	Possible temporary irritation due to the abrasive effect of the granules.
	First aid procedures	
	Skin contact	The product cannot be absorbed by the skin. Can provoke dehydration, or mild irritation due to the abrasive effect of the granules. Remove contaminated clothes and wash skin with water and soap. Consult your doctor should symptoms of dehydration appear.
	Contatto gli occhi	Do not rub the eyes. Flush immediately with large amounts of water, and keep eyes open for at least 15 minutes (so as to eliminate dust). If necessary consult your optician.
	Precautions in case of accidental spillage	
	Precautions for People	Keep people away from the area of spillage. Granules on the floor may cause it to be slippery. Take care not to slip..
	Cleaning methods	Collect using mechanical methods or a vacuum cleaner so as to avoid the formation of dust. Use appliances that don't create electric charges or sparks.
	Notes for disposal	
If the product is unused: normal disposal is allowed		
If the product has been (with presence of absorbed substances): method of disposal varies according to the nature of the absorbed substance and to the current applicable national or regional norms. The number is CER 19 09 04.		

* extracted from the safety data sheet supplied by the manufacturer. For further information, please request the safety data sheet directly from the manufacturer of distributor of the product.

INFORMATION ON THE WARRANTY

The manufacturer guarantees the quality of your product, if used in accordance with the instructions described in this manual, for a period of **12 months** for mechanical components.

The warranty starts from the date of delivery to the customer, which must be proven by returning the warranty form duly filled in and signed.

In the case of disputes, the date on the delivery note relating to the equipment serial number will be valid. The warranty does not cover defects or damage caused by:

- Incorrect installation
- Inadequate maintenance
- Tampering or unauthorized modifications
- Incorrect use of the equipment
- Non-compliance with the instructions included in this manual

The replacement of equipment or the extension of the warranty period because of breakdown is excluded.

Repair under warranty is carried out at the discretion of the manufacturer and does not include packing and shipment costs.

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PACKAGE CONTENTS

GENERAL NOTES ON DELIVERY

Upon delivery of the equipment, check the condition of the package and keep it in case of future needs.

Open the package and check that:

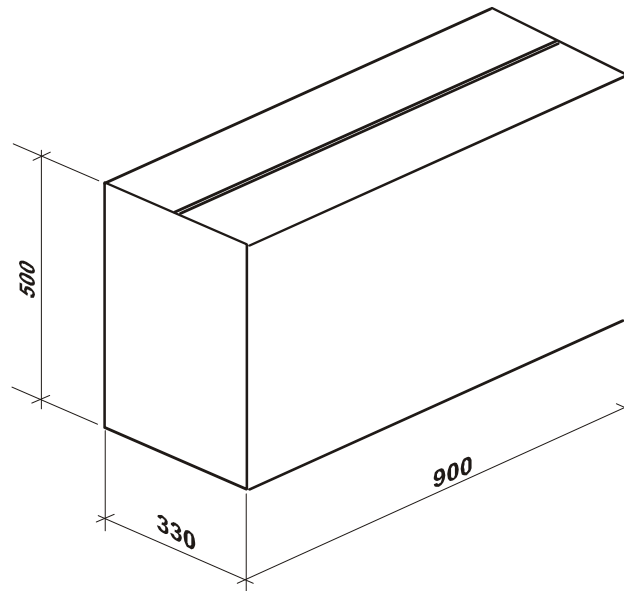
- the contents correspond to the delivery note
- there is no evidence of any damage.

In the event of damage or missing items, inform the carrier and the supplier immediately.

PACKAGE SIZE AND WEIGHT

Gross weight: 18 kg

Dimensions (mm):

DELIVERY CONTENT


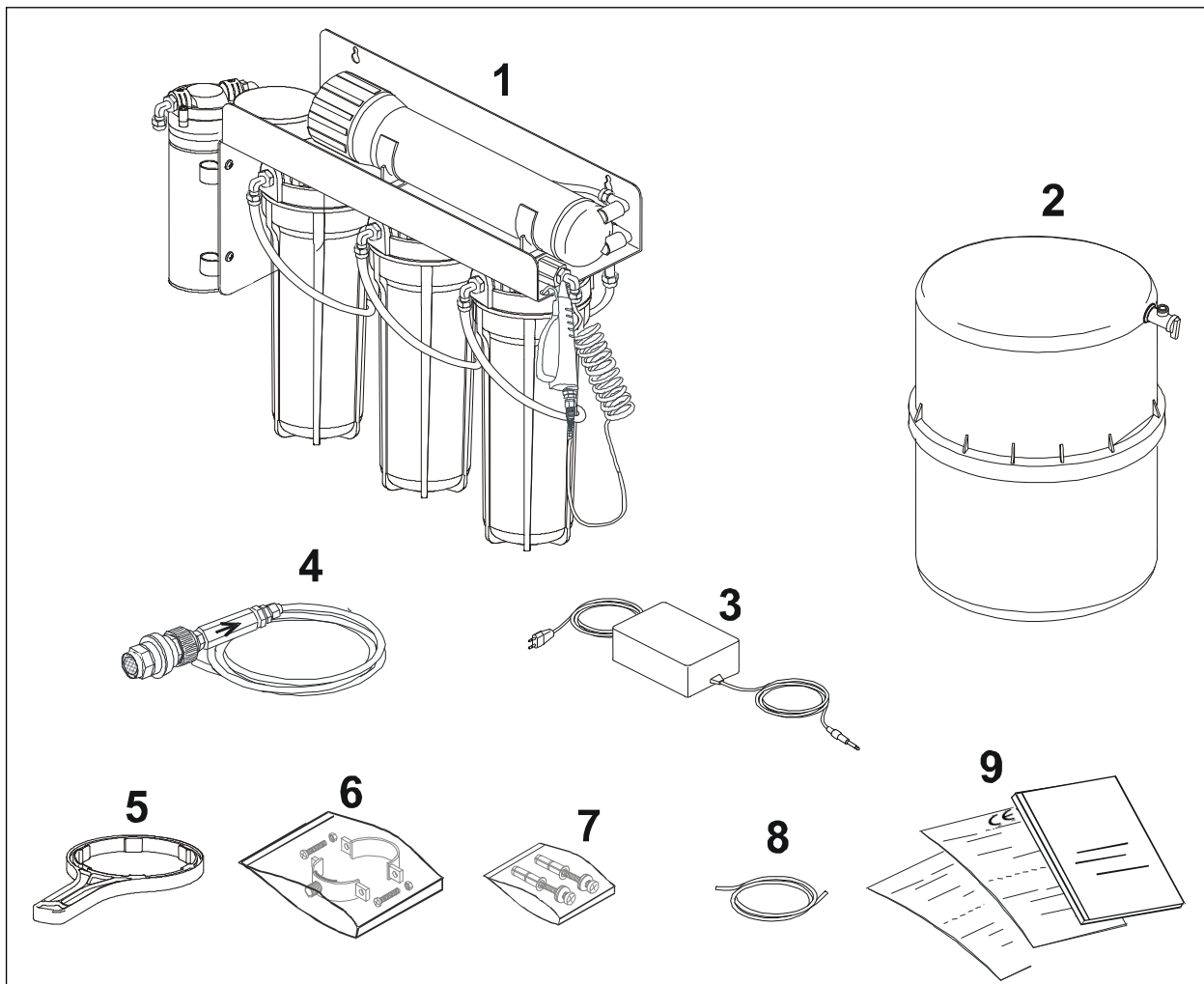
Figure 1 – Delivery Content

<u>Item</u>	<u>Description</u>
1	Complete osmotic subset with mounting brackets composed of: 3 filter cartridges, 1 osmotic membrane, 1 pump, 1 one-way valve, 1 pressure switch and a variety of internal and external connecting tubes and spiral tube with spray gun.
2	9 litre storage tank with a tap (theoretic volume 13/19 litres - actual volume 9/10 litres)
3	Power Supply transformer 240Vac / 24 Vdc
4	Water Block® valve with filter and setting key with one-way valve and hydraulic system tube (5m, Ø1/4")
5	Special key for cartridge holders
6	drain tube installation kit
7	Dowels and screws for fixing the system to a wall
8	Drain tube for carbon block cartridge
9	Warranty certificate , CE declaration of conformity and user manual

INSTALLATION AND USE OF THE DEMINERALIZER

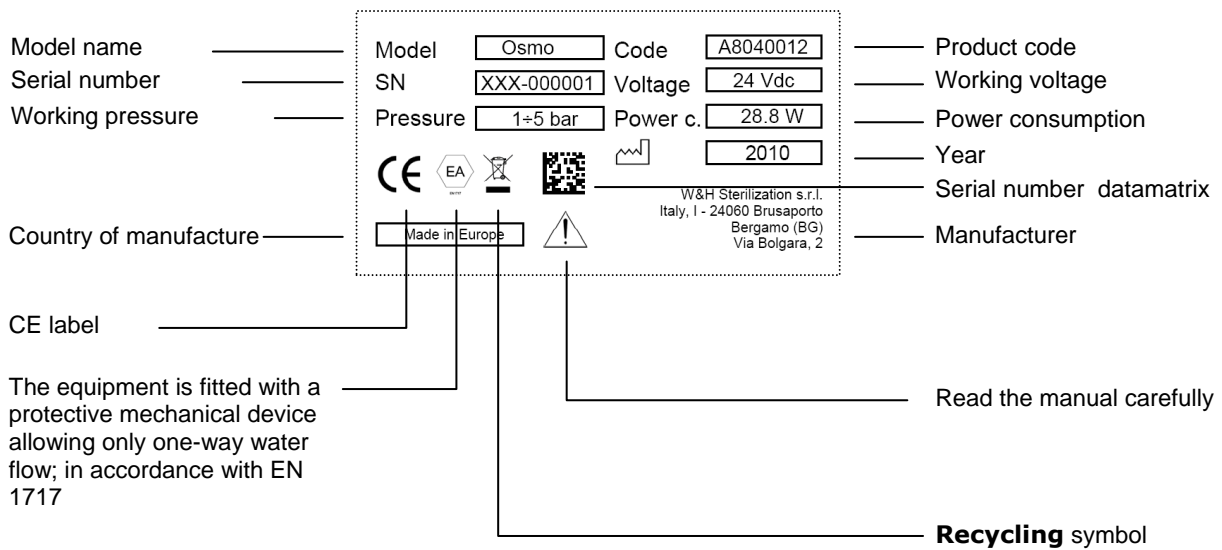SYMBOLS ON THE NAMEPLATE

SYSTEM COMPONENTS

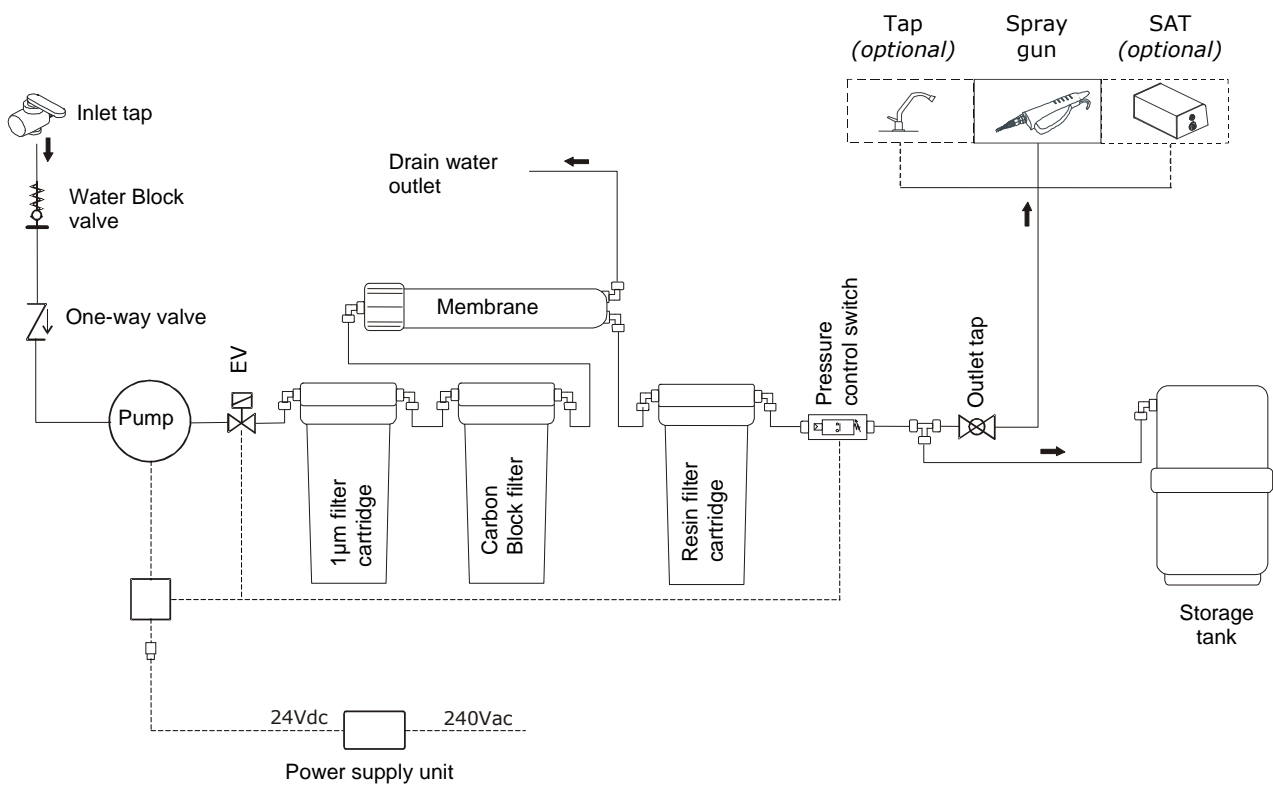
Figure 2 – Composition of the device

PRELIMINARY INSTALLATION INSTRUCTIONS

Wall-mounted installation

The water demineralizer should be fitted in a vertical position either beneath a sink or in a cabinet. For mounting, use the supplied dowels and screws. Install on the wall leaving a space of at least 10 cm from the floor or the bottom surface so that the cartridge holders can be dismantled.

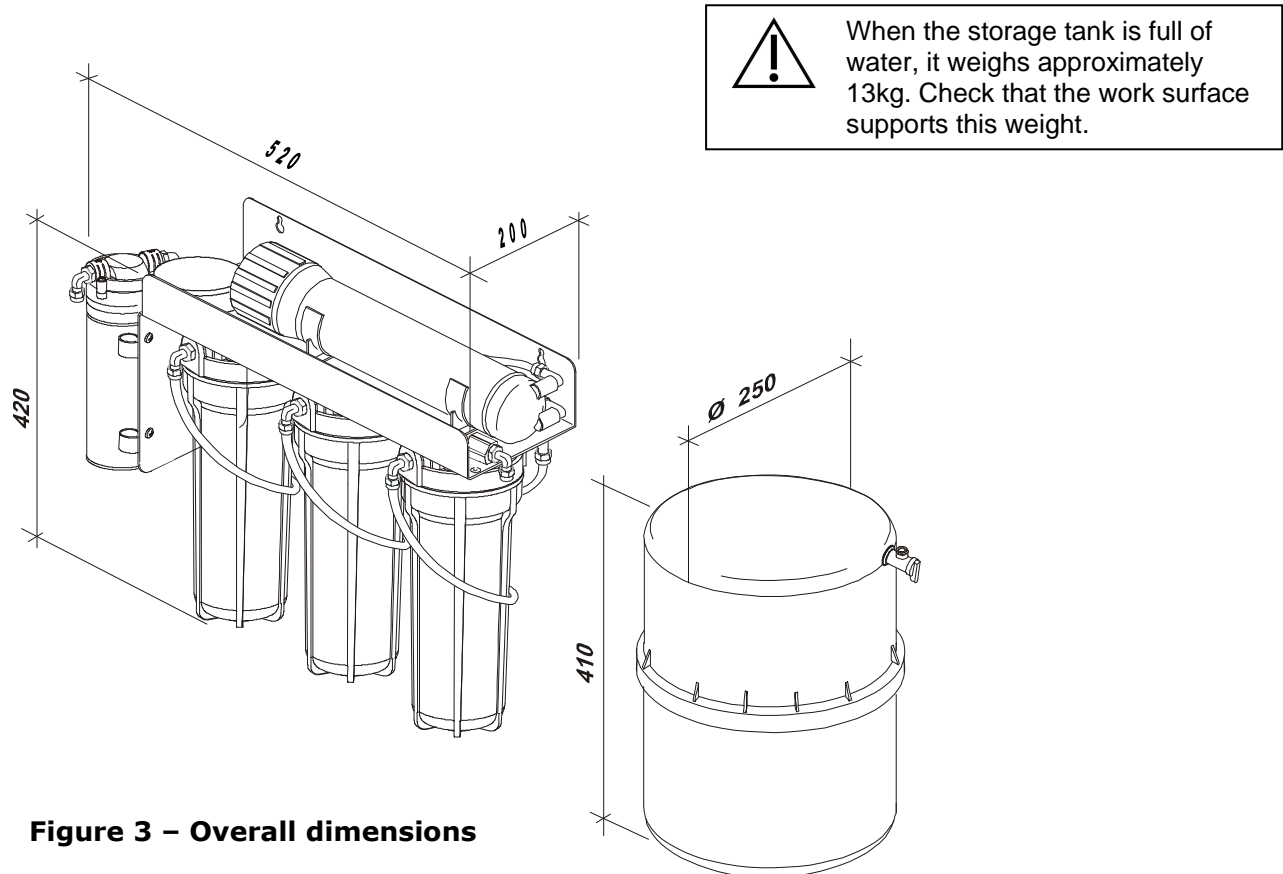
Figure 3 – Overall dimensions

Power Supply

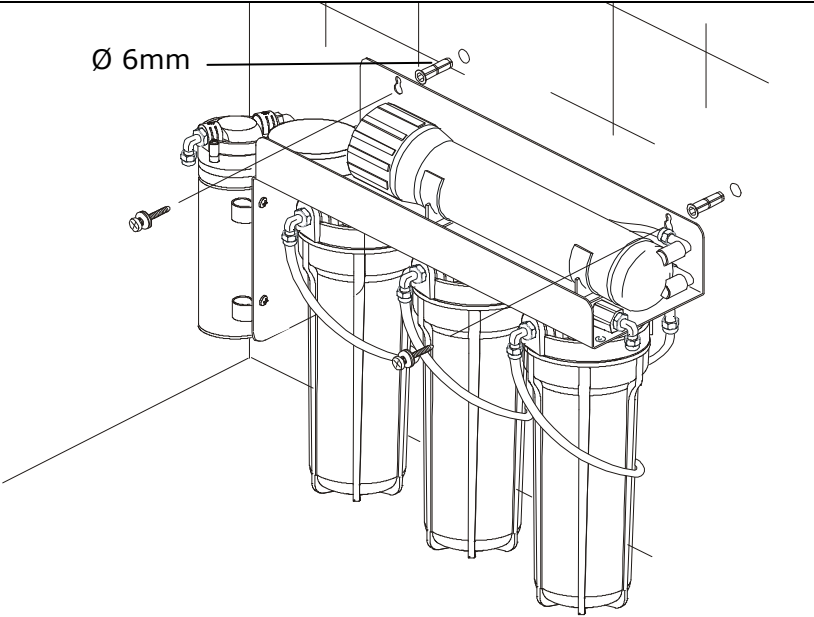
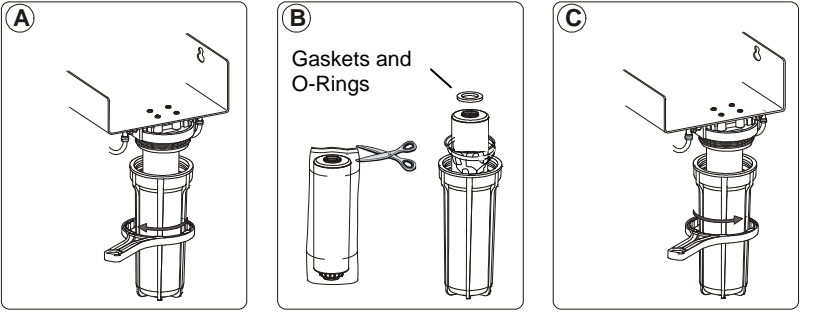
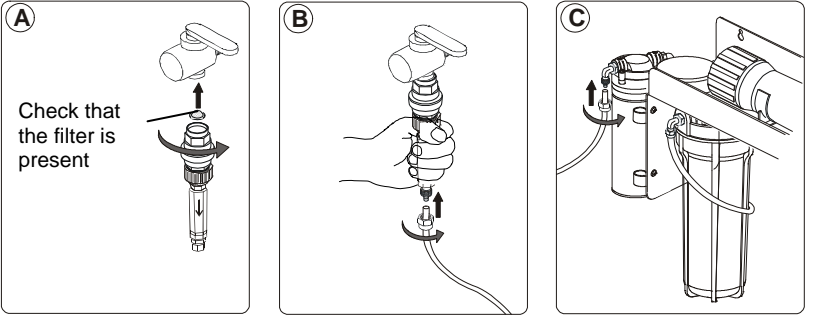
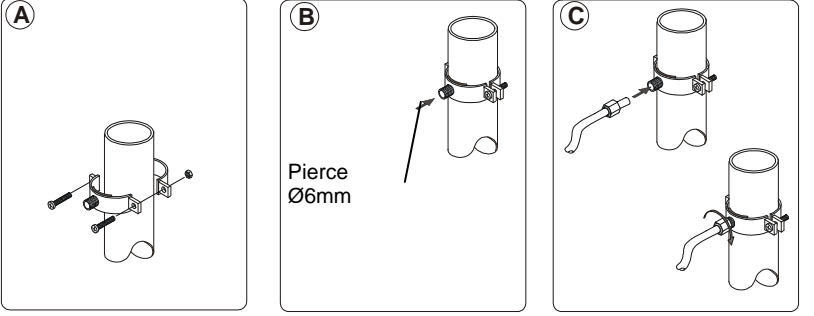
The unit is supplied with 240 Vac by means of a transformer which complies with the applicable electrical safety requirements of the Standards Institutes.

The maximum power absorbed is 40W.

Hydraulic installation

- Drain** Install a drain nearby to dispose the wastewater discarded by the osmotic membrane (max 1.5m)
- WB Valve** The safety valve (Water Block[®]) is fitted with a special one-way valve which prevents the return of water into the hydraulic system and possible consequent contamination. **For this purpose, the WB valve must be fitted in a vertical position (refer to the instructions at the end of this manual).**
- Charge** The installation of the product with incoming water pressure above 3 bars, can drastically reduce the lifespan of all the cartridges and especially of the osmotic membrane. If this is the case, the installation of a reducer to bring the pressure to less than 3 bars is recommended.

INSTALLATION

<p>1</p> <p>Install the osmotic group using the supplied dowels and screws</p>	 <p>Ø 6mm</p>
<p>2</p> <p>Unscrew the cartridge holders. Remove the cartridges and take off the wrapping (resin and carbon block cartridges). Reposition the cartridges in the holder taking care of the centring rings, the gaskets and O-rings. Tighten all the holders.</p>	 <p>A</p> <p>B Gaskets and O-Rings</p> <p>C</p>
<p>3</p> <p>Connect the water block group and one-way valve to the water network tap. Insert the tube into the fitting and tighten with a ring nut. Connect the tube to the pump's inlet fitting and tighten with a ring nut.</p>	 <p>A Check that the filter is present</p> <p>B</p> <p>C</p>
<p>4</p> <p>Mount the brackets from the assembling kit on the drain tube. Pierce the drain tube with a bit Ø 6mm. Insert the black drain tube from the back of the osmotic membrane and tighten with a ring nut.</p>	 <p>A</p> <p>B Pierce Ø6mm</p> <p>C</p>

<p>5</p> <p>Connect the white tube from the T fitting from the osmotic group to the storage tank.</p> <p>This is a quick-connector fitting: insert the tube until it hits the back of the fitting and pull slightly to check it is correctly inserted.</p> <p>To take the tube out of the fitting, press the fitting ring with your fingers and pull out the tube.</p>	
<p>6</p> <p>Disconnect the tube from the outlet fitting of the carbon block cartridge head (middle holder).</p> <p>Insert the supplied red draining tube on the fitting and tighten with a ring nut.</p> <p>Position the tube in a draining sink or a collecting basin.</p>	
<p>7</p> <p>Open the device's tap.</p> <p>Connect the transformer's 24V outlet wire to the osmotic group's power supply connector.</p> <p>Insert the plug into a plug hole.</p>	
<p>8</p> <p>Leave the water that comes out of the active carbon filter to drain until the water has no more black bits.</p>	<p style="text-align: center;">5÷15 min</p>
<p>9</p> <p>Remove the plug from the plug-hole.</p> <p>Close the device tap</p> <p>Reset the original connection on the carbon block's cartridge head (disconnect the red tube, reconnect the white tube and tighten with the ring-nut).</p>	

PRIMING

- Open the water network tap
- Insert the plug into a plug-hole

ATTENTION

After setting up the system, it takes 5 minutes to produce demineralized water. It takes approx. 140 minutes to completely fill up the system (including the storage tank).



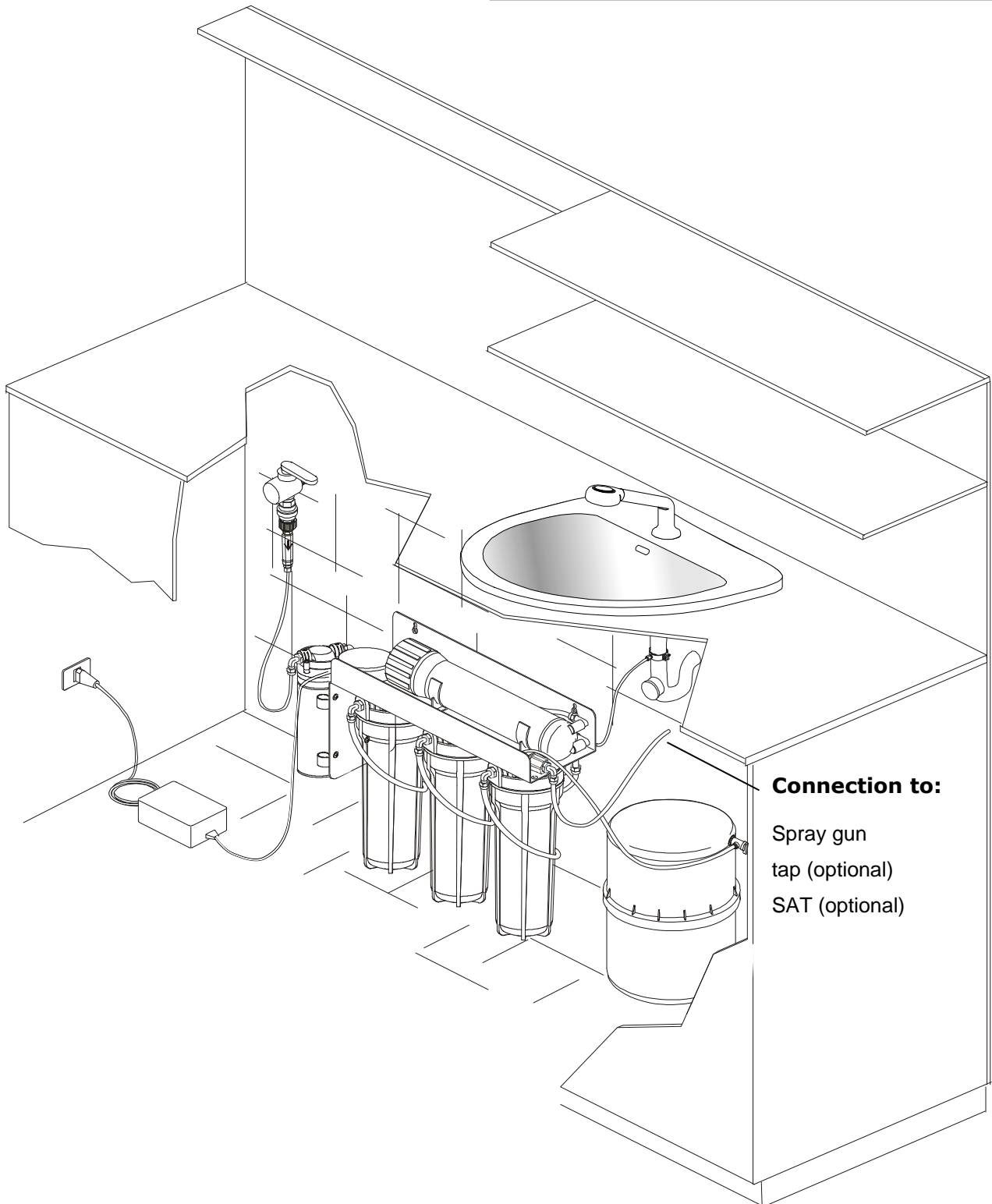
Figure 4 – Example of installation

EXAMPLES OF USAGE

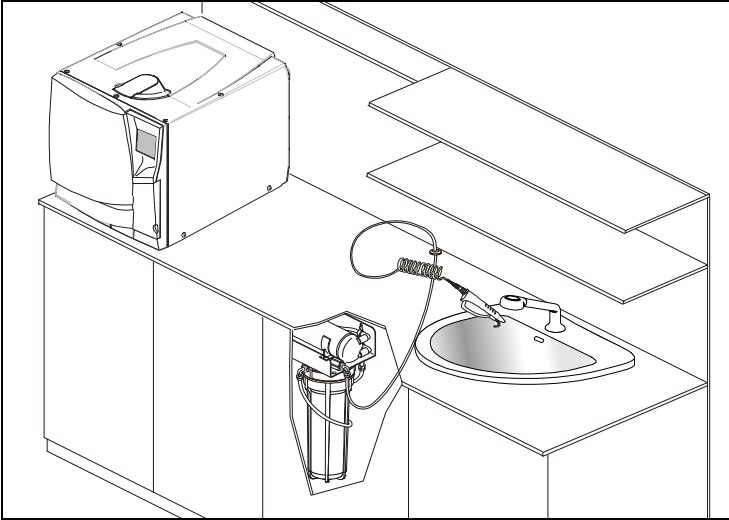Manual use with spray gun (standard) for manual filling of the sterilizer tank and washing of instruments

Manual usage with tap service tank

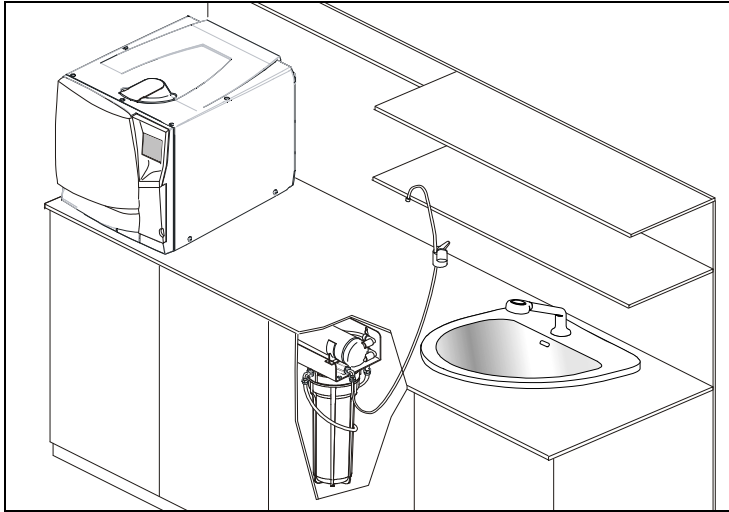Automatic use to fill the sterilizer using the SAT (option)

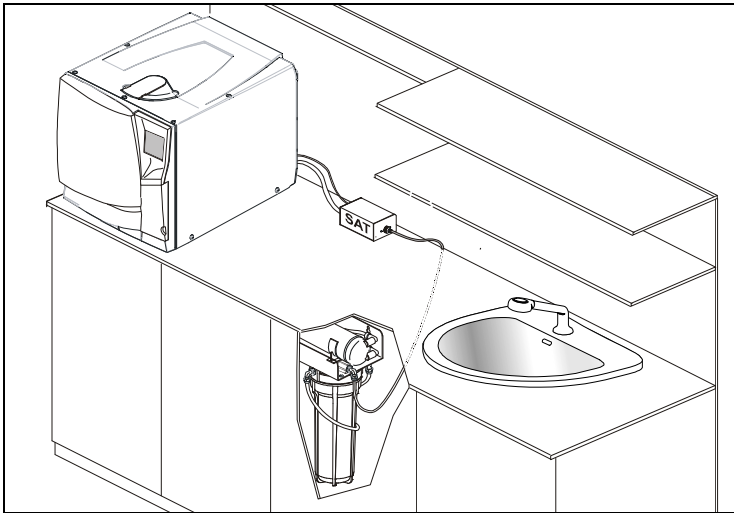
Figure 5 – Possible uses of the device

USE AND OPERATION OF THE DEMINERALIZER

USE OF DEMINERALIZER

Once the system has been correctly installed and connected, the demineralizer is able to immediately supply purified and demineralized water.

If the demineralized water is used to run a sterilizer, you have the option of manually filling the sterilizer tank with the use of the water gun, or have it filled automatically by using an external filling device (e.g. Sat) connected to the Osmo device.

The quantity of water produced by the cartridges depends on the quality and hardness of the city water supply in the area where you live.

The average time it takes to fill the storage tank is approximately 140 minutes (with the pressure control switch set to 3 bar).

PRECAUTIONS TO AVOID WATER CONTAMINATION

CAUTION

In accordance with subsection 4.5 of EN1717, in order to avoid water quality degradation and bacterial growth due to a tailback of water in the tubings, the entire system must be drained before leaving it unused for a longer period of time.

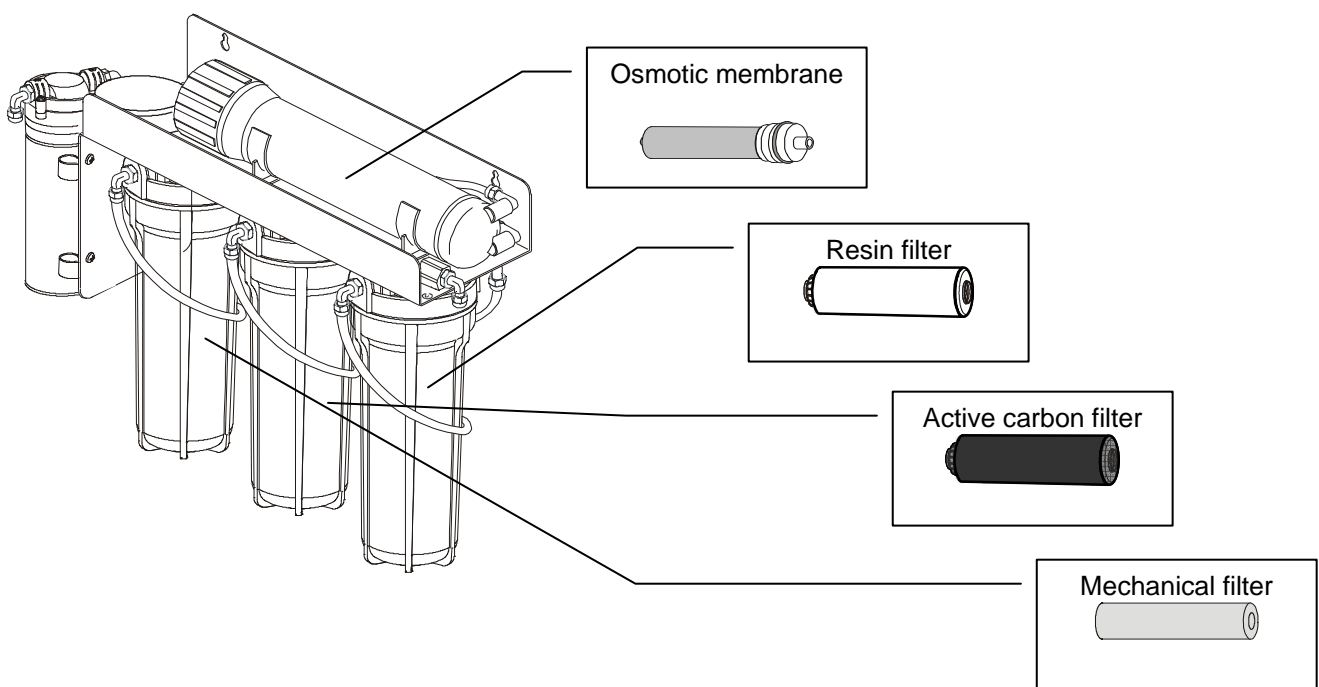
WARNING

Re-using the system after a long period of non-use

To restart the system when it hasn't been used for a long time or after the system has been emptied, water needs to circulate round the entire hydraulic circuit and the water collected in the circuit needs to be drained for a few minutes (see chapter **installation, points 6-9**).

It is also recommended not to leave the storage tank filled with demineralised water for a long period of time.

IDENTIFICATION OF THE CARTRIDGES

REPLACEMENT OF FILTER CARTRIDGES

Only use genuine filter cartridges as they have been specifically designed for this product.

The need to replace cartridges depends largely on the operating conditions and, above all, on the quality of the city water in your area. Annual filter cartridge replacement is recommended for average use.


Complete Kit	Intermediate Kit	Debris Cartridge 1µm	3/6 months
		Carbon Cartridge	3/6 months
		Resin Cartridge	3/6 months
		Osmotic membrane	2 years

Notes:

- Using the intermediate kit maintains the life-span and performance of the osmotic membrane.
- If the water has a conductivity value of over 15 µS after the intermediate kit has been replaced, also replace the osmotic membrane
- All the cartridges can vary slightly, which in combination with unfavourable operational conditions (e.g. high temperature, high presence of chemical particles in the water supply, high pressure) can cause a high reduction in the performance of the product.
- In the event of unexpected contamination of the water coming into the system (for example the presence of soil), all the cartridges can get damaged. Replace the intermediate kit and check if the osmotic membrane also needs changing.
- The installation of the product with incoming water pressure above 3 bars, can drastically reduce the lifespan of all the cartridges and especially of the osmotic membrane. If this is the case, the installation of a reducer to bring the pressure to less than 3 bars is suggested.

To replace cartridges follow these steps:

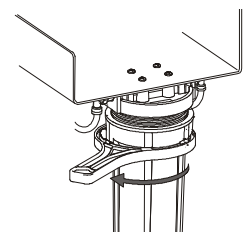
1. Disconnect the power supply
2. Close the tap water inlet valve and the storage tank valve
3. Release the water pressure with the use of the water gun (or tap, if connected) and/or disconnect the tubing leading to an external automatic filling device (e.g. Sat).
4. Unscrew the cartridge holder counter-clockwise with the use of the supplied special tool.

5. Remove used cartridges, remove and clean the centring rings, replace the cartridges using the rings. Make sure to insert the cartridge in the correct holder (if it says UP. place the cartridge with the writing at the top).

Securing bracket

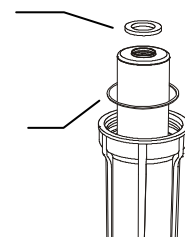
O-R

6. Retighten the cartridge holders by hand, taking care to correctly position the gaskets and o-rings. Tighten the cartridge holders with the use of the special tool.

7. Follow steps 6 to 9 in the “**Installation**” section.

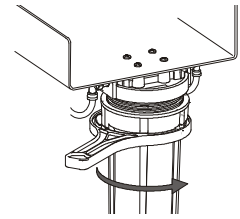8. Slowly reopen the hydraulic circuit tap.

REPLACEMENT OF THE OSMOTIC MEMBRANE

The service life of an osmotic membrane is normally over 2 years (depending on the quality of the city water in your area) but the use in unsuitable conditions can drastically reduce this.

In order to replace the membrane, first unplug the power supply and close the water inlet and the storage tank valve. Proceed as shown in the images below.

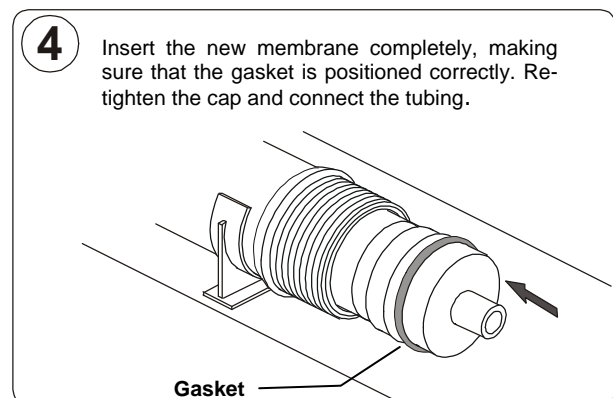
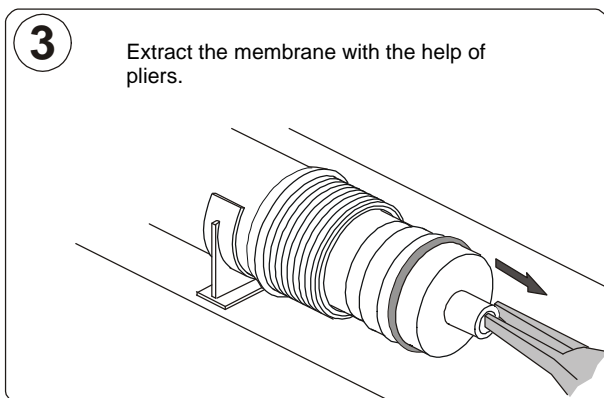
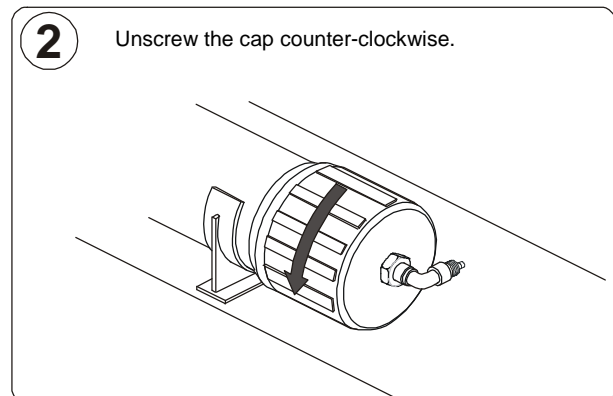
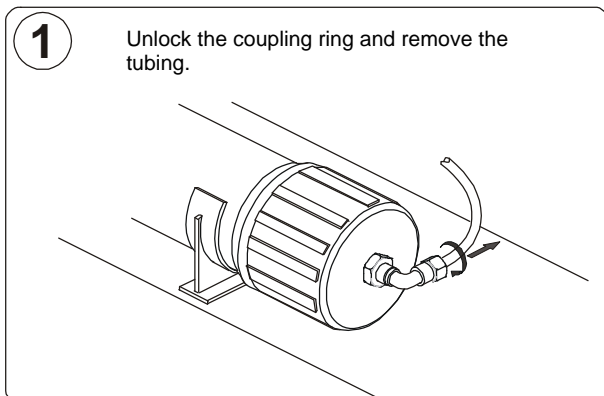
Figure 6 – Membrane replacement

After membrane replacement, the system needs to be primed! Refer to chapter “**Priming**” for further instructions.

MAINTENANCE

EXTERNAL CLEANING

Clean with a damp cloth using non-abrasive and non-corrosive detergents (neutral pH value).

MAINTENANCE OF THE WATER BLOCK® AND ONE-WAY VALVE

Depending on the city water quality in your area and the frequency of use of the equipment, the valve inlet filter should be regularly cleaned as explained in the instruction leaflet attached to the end of this manual.

CAUTION

Regular control of the Water Block and one-way valve

To ensure compliance with subsection 4.6 of EN1717, have the Water Block and the one-way valve checked at least once a year; in case a sterilizer is attached to the system, this could be done during the scheduled maintenance visits for the sterilizer.

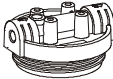
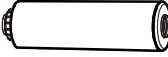
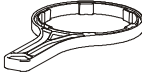
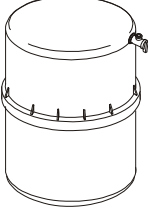
It is further recommended to check all related devices fitted to the sterilizer to ensure that the hydraulic connection between demineralizer and sterilizer corresponds to category 2, in accordance with EN 1717.

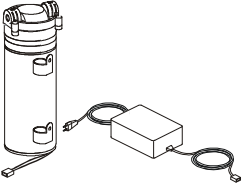
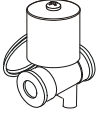
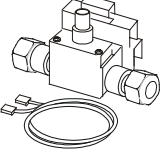
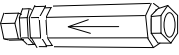
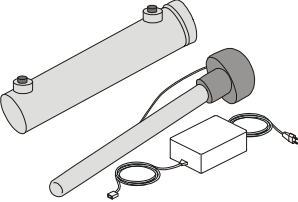
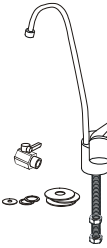
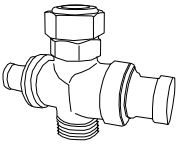
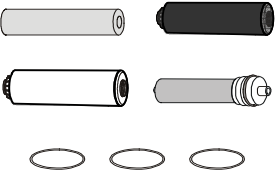
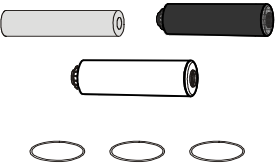
WARNING

The Water Block serves as an anti-flooding device. Blocking only takes place if there is considerable leakage (such as a cut in the tube between the Water Block and the demineralizer). Blocking does not take place if there is a small leak in the product's hydraulic system.

SPARE PARTS

Ref.	Description		Code
1		CARTRIDGE HOLDER	A812103X
2		FILTER HEAD	A813005X
3		DEBRIS CARTRIDGE CPE 1µM	A813015X
4		"CARBON BLOCK" CARTRIDGE	A813016X
5		RESIN CARTRIDGE	A813017X
6		OSMOTIC MEMBRANE 75 GPM, 95-98% REJECTION	A813018X
7		O-RING	A813014X
8		PLASTIC KEY FOR CARTRIDGE HOLDER	A812101X
9		STORAGE TANK INCLUDING TAP	A813007X
10		WATER BLOCK® VALVE (INCLUDING FILTER AND SETTING KEY)	A812005X
11		SPIRAL TUBE ¼"	A812105X
12		WATER GUN	A812106X
13		CENTERING RING	A813006X

Ref.	Description	Code	
14		PUMP WITH TRANSFORMER 24Vdc	A813024X
15		ELECTRIC-VALVE 24VDC	U382016X
16		MANOSTAT AND WIRING KIT	A813011X
17		ONE-WAY VALVE	A812116X
18		UV KIT	A813012X
19		KIT WATER TAP	A813013X
20		WATER PIPE (5 m, Ø 6.35 mm)	A813020X
21		PRESSURE REDUCER $\frac{3}{4}$ " X $\frac{3}{4}$ " 3 BAR	A813025X
22		CATRIDGE KIT COMPLETE (REF. 3+ 4 + 5+ 6 + 7x3)	A813010X
23		CARTRIDGE KIT INTERMEDIATE (REF. 3 + 4+ 5 + 7x3)	A813019X

TECHNICAL SPECIFICATIONS

Operating temperature	+5°C ÷ +30°C
Storage temperature	+5°C ÷ +35°C Protect from sunlight or other sources of heat
Size	520x420x200 mm
Weight with cartridges (empty storage tank)	11 Kg
Use	interior
Pollution degree	2
IP	21
Weight of the storage tank	3,5 kg
Tank - nominal volume	19 liters
Tank - storable volume	ca. 9 liters
Electric power supply	100-240 VAC, 50/60 Hz, 1.0A (with separate voltage transformer)
Max. absorbed power	40 W
Class of insulation	III
Tap water pressure	minim 1 bar (14,7 psi) maximum 5 bar (73,5 psi)
Quantity of produced water with conductivity < 15µS ¹	10,000 liters

¹ Test conditions: T=25°C, tap water quality 450 µS and water pressure of 3 bar; with continuous flow;

MAINTENANCE ISTRUCTIONS FOR THE SAFETY VALVE (WATER BLOCK[®])


WATER BLOCK (WB) SAFETY VALVE

Indispensable device for washing machines, dishwashers and demineralizers as, under the conditions defined in the technical features herewith described, it prevents a flow of water greater than the one for which it has been set by the adjustment pointer. This avoids continuous leaking. The Water Block also has a particular non-return valve (antipollution). The device should be installed vertically (see fig. 2) and following the instructions carefully.

TECHNICAL FEATURES

- Burst pressure: 500 N/cm²
- Max. static pressure: 100 N/cm²
- Max. controllable volume: about 50 lt.
- Min. operating capacity: 2 lt/min.
- Max. operating capacity: 30 lt/min
- Max. water temperature: 70°C

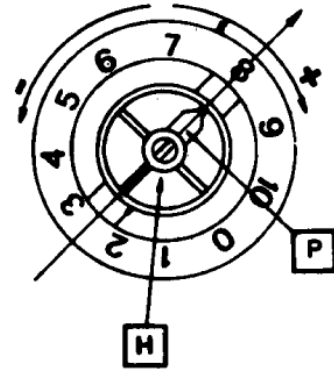Fig. 1

USER INSTRUCTIONS

WATER BLOCK should be installed between the tap and the supply hose of the demineralizer (fig. 2). Using the enclosed plastic key, turn pointer 'P' (fig. 1) anticlockwise to the required setting (example, position 1,5).

CAUTION:

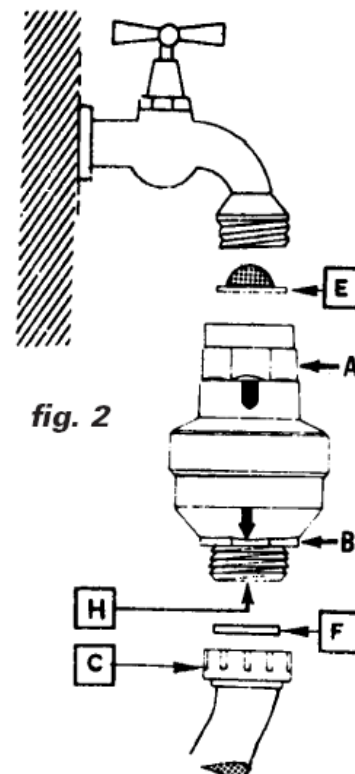
For use on other fixtures, set the pointer to a position suitable for that particular application, considering that each number on the scale corresponds to a controlled volume of approx. 4-6 litres.
In case of installation on fixtures which foresee dispensing of drinking water for alimentary use it is advisable to let flow at least 10 litres of water inside the device before the connection, so as remove eventual impurities banked during the storage.

INSTALLATION

1. Manually screw the WB to the tap having suitable thread (3/4" gas), inserting the filter 'E' with its convex side facing upwards (fig. 2). The WB will not operate properly if the filter is not installed correctly. If the WB is tightened or loosened with the aid of a wrench, the same must be applied to the appropriately octagonally shaped part of the housing 'A'.
2. Screw the supply hose 'C' to the lower end of the WB. If the supply hose is equipped with a filter, the latter must be substituted by a flat rubber washer 'F'. If the said hose has to be tightened or loosened with the aid of a wrench, the same must be applied to the appropriately octagonally shaped part of the WB housing 'B' so as to hold it firmly while performing this operation.

If a failure occurs causing a continuous water leak, the WB will shut off when the volume of water flowing through equals the amount to which it was previously set with the pointer 'P'.

To reset the Water Block again after it has shut off, turn the tap off and carry out the necessary repair work. Remove the supply hose 'C' from the Water Block and press the coloured plunger 'H' which is located inside the lower housing of the device itself (fig. 1). It may be difficult to press this plunger if there is high pressure at the inlet side of the WB. In this case, slacken the device from the tap: the plunger 'H' can then be pressed with ease. Repeat the operation as described in the section headed 'Installation'.

CAUTION:

To avoid any possible damage that may be caused by water freezing inside the WB after its installation, the ambient temperature should never be below 2°C. We decline all responsibility deriving from the incorrect use of the WB or from its use for applications and/or solutions not expressly foreseen by the above instructions.

MAINTENANCE:

Depending on the conditions and frequency of usage of the water block, it is recommended to clean out the inlet filter 'E' periodically and to replace the seal 'F'. We reserve the right to modify any features without prior notice.

Produced by
ELTEK s.p.a.
Tel. 0142 335511
Fax 0142 335555
15033 Casale Monf. (Italy)
salesdept@eltekgroup.it

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MAINTENANCE ISTRUCTIONS FOR THE SAFETY VALVE (WATER BLOCK®)

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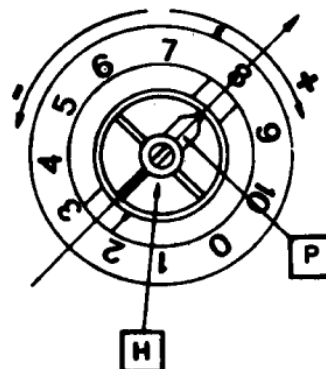
CAUTION:

For use on other fixtures, set the pointer to a position suitable for that particular application, considering that each number on the scale corresponds to a controlled volume of approx. 4-6 litres.

In case of installation on fixtures which foresee dispensing of drinking water for alimentary use it is advisable to let flow at least 10 litres of water inside the device before the connection, so as to remove eventual impurities banked during the storage.

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W&H Sterilization S.r.l